

# CR PHOSPHOR PLATE CLEANER

A dedicated cleaner for CR imaging plates



CR PHOSPHOR PLATE CLEANER IS AN EASY-TO-USE, ETHANOL-BASED CLEANER THAT SUPPORTS LONGER LIFE AND HIGHER PERFORMANCE FOR CR IMAGING PLATES.

## Protect your investment

Dust and dirt build up on CR phosphor plates, leading to image artifacts. To avoid this, plates need cleaning at regular intervals. As CR imaging plates are highly sensitive to water-based cleaning fluids, Agfa HealthCare has developed an ethanol-based plate cleaner. Using this cleaner, specially designed for CR phosphor plates, will help support them the longest service life.

## Simple to use

CR Phosphor Plate Cleaner is simple to use: moisten a piece of gauze with the cleaner, wipe gently across the plate's surface and wait for any remaining solvent to evaporate.

CR Phosphor Plate Cleaner is suitable for use with CR imaging plates only, not with conventional plates.

## Cleaning frequency

Plates subject to a normal frequency of use should be cleaned at least once every month or whenever there is doubt about particles visible on the X-ray images.

# Technical

## SPECIFICATIONS

### Solvent

- Ethanol 99.7%

### Additives

- Catanac SN (Cyastat SN 50) 0.3%

### Contains

- 0.5 l

### Safety

- Highly flammable
- Keep container closed
- Keep away from sources of ignition

Safety datasheets are available on request

Agfa and the Agfa rhombus are trademarks of Agfa-Gevaert N.V., Belgium, or its affiliates. All other trademarks are held by their respective owners and are used in an editorial fashion with no intention of infringement. The data in this publication are for illustration purposes only and do not necessarily represent standards or specifications, which must be met by Agfa HealthCare. All information contained herein is intended for guidance purposes only, and characteristics of the products and services described in this publication can be changed at any time without notice. Products and services may not be available for your local area. Please contact your local sales representative at [agfa.com](http://agfa.com) for availability information. Agfa HealthCare diligently strives to provide as accurate information as possible, but shall not be responsible for any typographical error.

© Copyright 2016 Agfa HealthCare NV

All rights reserved

Printed in Belgium

Published by Agfa HealthCare NV,

B-2640 Mortsel-Belgium

NGGHP GB 00201601