



1 To perform a full leg full spine exposure


- 1 Add the full leg full spine (FLFS) exposure set to the exam.
- 2 Insert the (FLFS) CR cassettes in the cassette holder, as shown in the illustrations below.
Pay special attention to the orientation of the cassettes:
 - Face the cassette holder and insert the cassettes with the black side facing you.
 - Hold the cassettes with the label at the top.

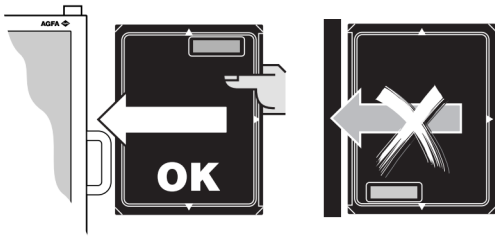

 *Note: Incorrect positioning of the cassettes in the cassette holder will cause artifacts on the images.*

- 3 Perform the full leg full spine exposure.
- 4 Put the cassettes in the digitizer. Make sure to identify the partial images corresponding to the exposures.

After the last image is received on the workstation, an extra image is created in the exam, containing the stitched FLFS image.

 *Note: Check for correct anatomical order of the partial images if the stitched FLFS image is not created automatically.*

For detailed information, refer to:

- NX User manual, section “Stitching Full Leg Full Spine images”
- CR Full Leg Full Spine User manual, section “Basic workflow”