Instructions for Use - Portegra2

Languages:	English	2 - 20
	Deutsch	21 - 39

Dear Customer,

*Thank you for purchasing a MAVIG Portegra2 suspension system or component.
Please read this document carefully and follow all directions, warnings, and notices.
Feel free to contact us with any questions or comments on our products.*

Contact Information

For questions or comments, please see back of manual for nearest MAVIG contact.

Disclaimers

Considerable effort has been made to ensure that the contents of this manual are free from omissions and inaccuracies. However, MAVIG makes no warranties with respect to the contents of this document and disclaims any implied warranties.

Changes

MAVIG products are under continuous development. MAVIG reserves the right to modify the manual, packaging, equipment or technical specifications, and scope of delivery without prior notice.

Translations

The master document is in German. In case of translation, precedence remains with the German version.

Copyright and Trademarks

All rights reserved. These instructions/manuals are protected by copyright law. MAVIG and Portegra2 are registered trademarks of the company MAVIG GmbH. All other trademarks mentioned in the manuals are the property of the corresponding manufacturer.

The logo for Portegra2 features the word "PORTEGRA" in a bold, grey, sans-serif font. The number "2" is significantly larger and rendered in a bright orange color, with a stylized, flowing design that extends to the right.

© 01/2015 MAVIG GmbH

	Introduction	02
	Contents	03
	Use and ready availability of the instructions / Warranty	04
1.0	Symbols and Definitions	05
2.0	Safety Instructions	06
3.0	Operation	07
	Initial use	07
3.1	Identification of the system/device	07
3.2	Intended use	07
3.3.	Standards	07
3.4	Precautionary measures	07
3.5	Working with the system	08-11
4.0	Positioning of extension arm - spring arm	12-13
5.0	Adjusting the tilt tension of center-mounted shields	13
6.0	Installation / Removal	13
7.0	Cleaning / Disinfection	14-15
7.1	Cleaning	14
7.2	Disinfection	15
8.0	Servicing	16-17
9.0	Specifications	18
10.0	Replacement Parts	18
11.0	Disposal	18

Use and ready availability of the instructions for use

This manual describes the use of the various components of the Portegra2 system manufactured by MAVIG. The instructions apply to the 360° and Twin column ranges of the Portegra 2 system both in their stationary and track-guided versions.

The instructions for use are an integral part of the system/device and must be kept close at hand at all times for ease of reference. The Portegra2 installation instructions must also be kept close at hand as certain points raised in the instructions for use refer to them.

Do not pass the system/device on to third parties without the valid instructions for use. Check the identity and version number of the system/equipment to ensure that an up-to-date and valid set of instructions for use has been issued with the system/device.

Warranty

MAVIG guarantees the safety and reliability of the system/device solely on the following basis:

- The system/device is used solely for its intended purpose and its operation complies with the information provided in these instructions for use.
- Only original authorised replacement parts or accessories as specified by MAVIG are used. The use of unauthorised parts holds unknown risks and must be avoided at all times.
- No structural changes are made to the device. Unauthorised changes or modifications to the system/device are prohibited for reasons of safety.
- The system/device is regularly inspected and serviced at the specified times.

1.0 Symbols and Definitions

WARNING

Indicates that the information is a warning. Warnings advise you of circumstances that could result in injury or death to the patient or operator. Read and understand the warnings before installing or operating the system.

CAUTION

Indicates that the information is a caution. Cautions advise you of circumstances that could result in damage to the device or improper functioning of the device. Read and understand the cautions before installing or operating the system.

NOTE:

Provides usage tips and useful information.

WARNING

Indicates a possible electrical hazard, which can lead to severe injury or even death.

This symbol marks the points, where protective ground lines are attached.

This symbol represents to consult accompanying documents.

MAXIMUM WEIGHT LOAD SYMBOL

This symbol is combined with a weight in kilograms that shows the maximal weight that the marked component can hold.

CE MARK

With this symbol, MAVIG declares that the products comply with the relevant regulations set forth in the applicable European Directives.

RECOGNISED UL COMPONENT

This component has been recognised by Underwriters Laboratories Inc. Representative samples of this product have been reviewed by UL and comply with the applicable requirements.

COLLISION - WARNING

Collisions with other equipment can cause structural failure of the suspension arm, resulting in operator or patient injury. If collision occurs, immediately have the arm inspected by a qualified service technician.

2.0 Safety Instructions

All electrical, assembly and dismantling work on the system must be carried out by qualified technicians specifically trained to perform this work.

Only appropriately trained personnel may operate, clean, and disinfect the device.

WARNING – Electric shock.

When installing and adjusting the Portegra2 and/or associated components, ensure to cut off the power supply to the entire system. ***Danger of Death!***

CAUTION – Risk of injury or breakage.

The spring arm is always under tension. Before carrying out assembly or dismantling work on the spring arm, always move it slowly to its upper limit.

There is otherwise a risk that the arm will suddenly shoot upwards when components are removed. This could result in personal injury and/or damage to the system!

BEWARE:

The system has not been certified to DIN EN 60601-1 when live. The system must therefore be certified again by an appropriately trained technician after installation with monitors, wiring etc.

MAVIG is not responsible for unauthorised or inappropriate handling of or changes to the system. Changes to the system require written authorisation by MAVIG as inappropriate changes may have adversely affect on the smooth operation, safety, and reliability of the system.

Any changes made to the system render the CE certification awarded to MAVIG invalid.

Thereafter, new approval must be obtained by the operator in accordance with the relevant statutory provisions.

Avoid collisions between the system and other devices, walls, ceilings, etc. as these might seriously damage it.

CAUTION – Risk of injury.

The manufacturer must be immediately informed of any damage caused to the system. The system must not be used until the damage has been rectified. Disconnect the system from the mains. Do not stand or work beneath the system.

CAUTION – Risk of injury.

Do not remove any covers, open any cable ducts, etc.!

Any disregard of these instructions exempts the manufacturer from all liability and/or warranty obligations.

3.0 Operation

Initial use

These instructions for use apply after initial use as specified. The system/device must be thoroughly cleaned and disinfected prior to initial use (see section 7.0 Cleaning/disinfection, page 14).

3.1 Identification of the system/device

These instructions for use are intended solely for systems/devices identified on the model plate attached to the system:

Type: **Portegra2**

Information on other available Portegra2 products can be found on our web page at:
<http://www.mavig.com/system-solutions/portegra2/>

3.2 Intended use

The Portegra2 system is a spring-supported, rotating and swivelling suspension system for use in climate-controlled rooms in medical facilities.

It is the responsibility of the operator to ensure the device is safe for use and to provide the user with due instruction in its operation and its use for the purpose intended.




3.3 Standards

Medical Devices Act (MPG)
MDD 93/42/EEC (2007) – Medical Device Directive
IEC 60601-1:2005-12 (Edition 3.0)


3.4 Precautionary measures

Observe the following when using the system/device after successful installation in accordance with the Portegra2 installation instructions (POR03Oxx):

1. Avoid collisions with other devices, the walls, or the ceiling.
2. Stop using the system if there is a collision or any other damage to the system.
Do not stand or work beneath the system.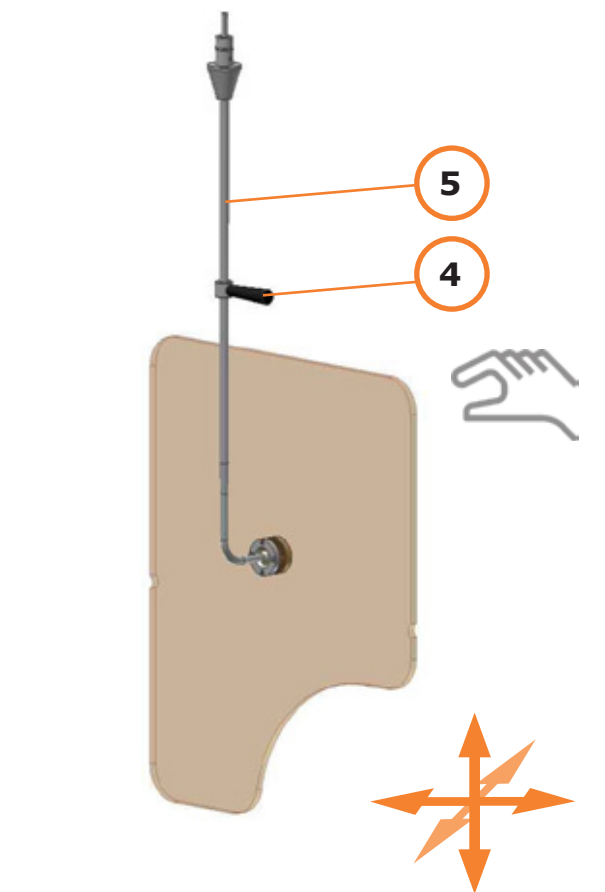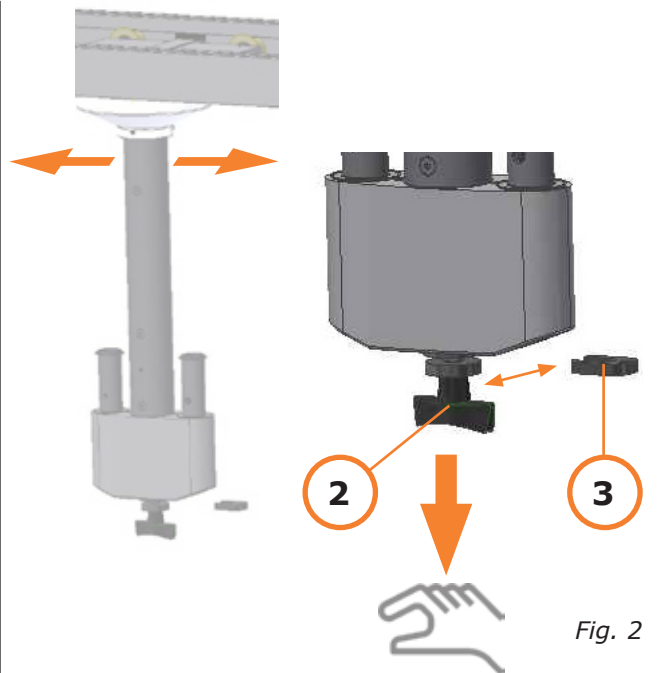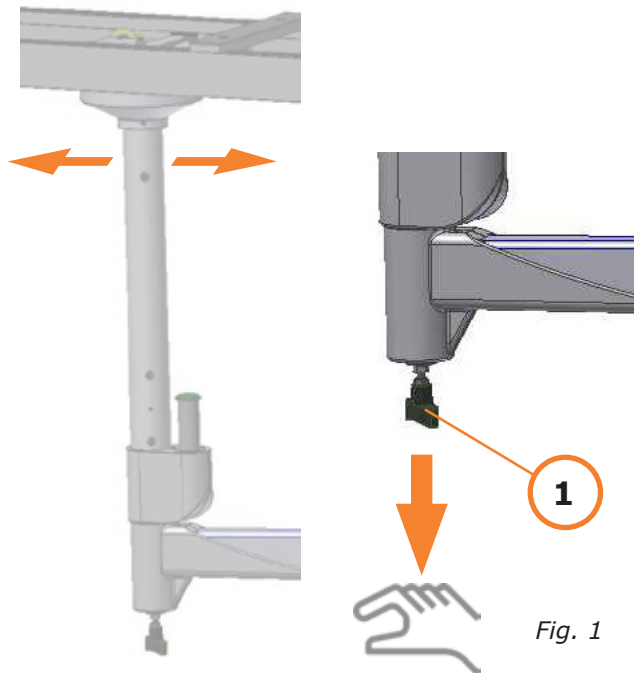

Radiation protective shield
by way of example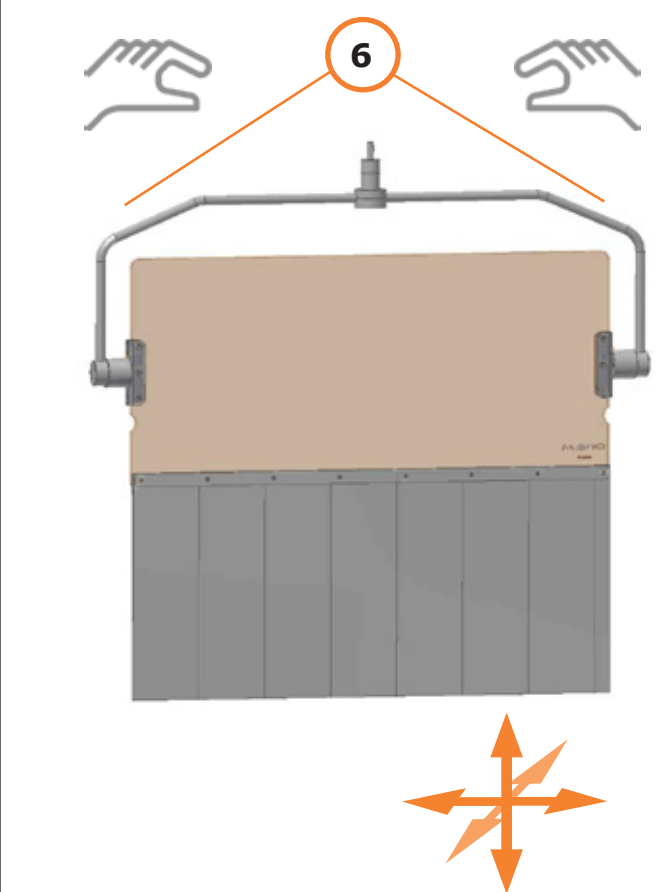

Radiation protective shield
by way of example

3.0 Operation

3.5 Working with the system

**CAUTION – Risk of injury or breakage.**

Only move the system by the defined points/handles. Always hold the system firmly until you have finished moving it. Never allow movement to swing outwards in an uncontrolled manner.

3.5.1 360° column on a ceiling track (Fig. 1)

Release the brake by pulling downwards on the plastic handle (1/Fig. 1). Always release the brake before moving the column on the track. The ceiling column should not be moved along the track by pulling or pushing the arm or end device (shield, light, monitor holder).

To lock the brake, turn the handle to the right. To release the brake again, turn the handle to the left until it locks in place.

Also see chapter 4.0, Positioning of extension arm - spring arm.

3.5.2 TWIN column on a ceiling track (Fig. 2)

Release the brake by pulling downwards on the plastic handle (2/Fig. 2). Always release the brake before moving the column on the track. The ceiling column should not be moved along the track by pulling or pushing the arm or end device (shield, light, monitor holder).

To lock the brake, place the U piece (3/Fig. 2, (attached to the column by a chain)) between the plastic collar and the plastic disc. To release the brake again, pull out the U piece.

Also see chapter 4.0, Positioning of extension arm - spring arm.

3.5.3 Radiation protective shield with ball joint (Fig. 3)

To move the system, always hold it by the handle provided (4/Fig. 3) or by the metal rod (5/Fig. 3).

Also see chapter 4.0, Positioning of extension arm - spring arm.

3.5.4 Radiation protective shield with two-point bar (Fig. 4)

To move the system, always hold it by the bar (6/Fig. 4) and guide it to the desired position.

Also see chapter 4.0, Positioning of extension arm - spring arm.

Operating light by way
of example


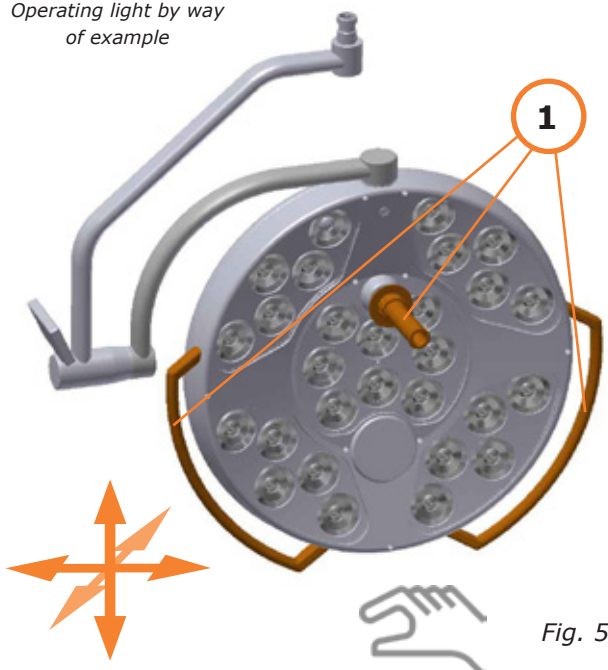
Fig. 5

Fig. 6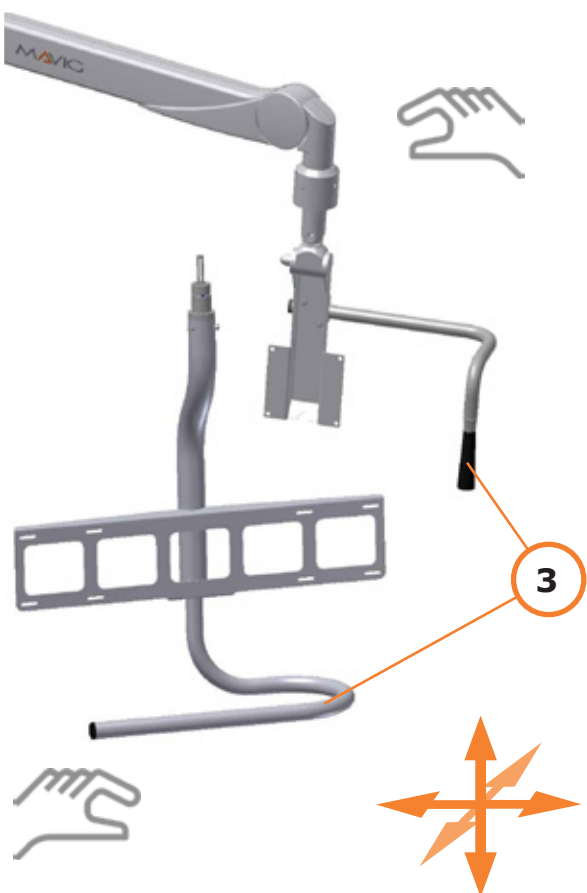

Monitor holder by way
of example

Fig. 7

Injector holder by way
of example

Fig. 8

3.0 Operation

3.5 Working with the system

Move the system in any direction by holding it firmly at the places provided.

CAUTION – Risk of injury or breakage.

Always hold the system firmly until you have finished moving it.
Never allow movement to swing outwards in an uncontrolled manner!

3.5.5 Lighting systems (Fig. 5)

Always move the lamp by the handle or handlebar provided. Highlighted in colour, see 1/Fig. 5.

Also see chapter 4.0, Positioning of extension arm - spring arm.

3.5.6 Monitor holder without handle (Fig. 6, GD4020)

The system should only be moved at the point where the monitor holder is mounted to the spring arm (2/Fig. 6).

Also see chapter 4.0, Positioning of extension arm - spring arm.

3.5.7 Monitor holder with handle (Fig. 7, GD4021 / GD4210 / GD4220)

Move the system only by the handle provided. (3/Fig. 7)

Also see chapter 4.0, Positioning of extension arm - spring arm.

3.5.8 Injector holder bar (Fig. 8)

Move the system only by the injector holder bar (4/Fig. 8).

Also see chapter 4.0, Positioning of extension arm - spring arm.

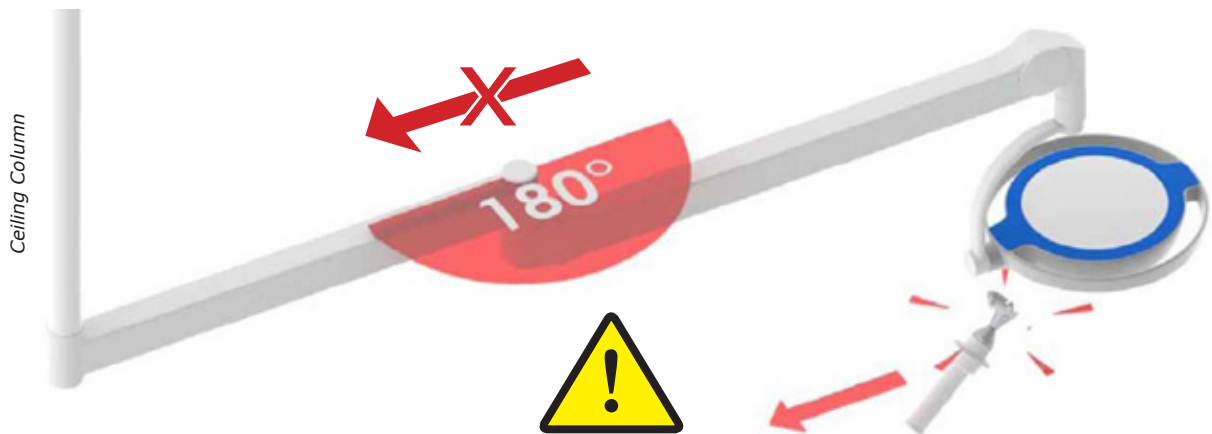
Fig. 9a

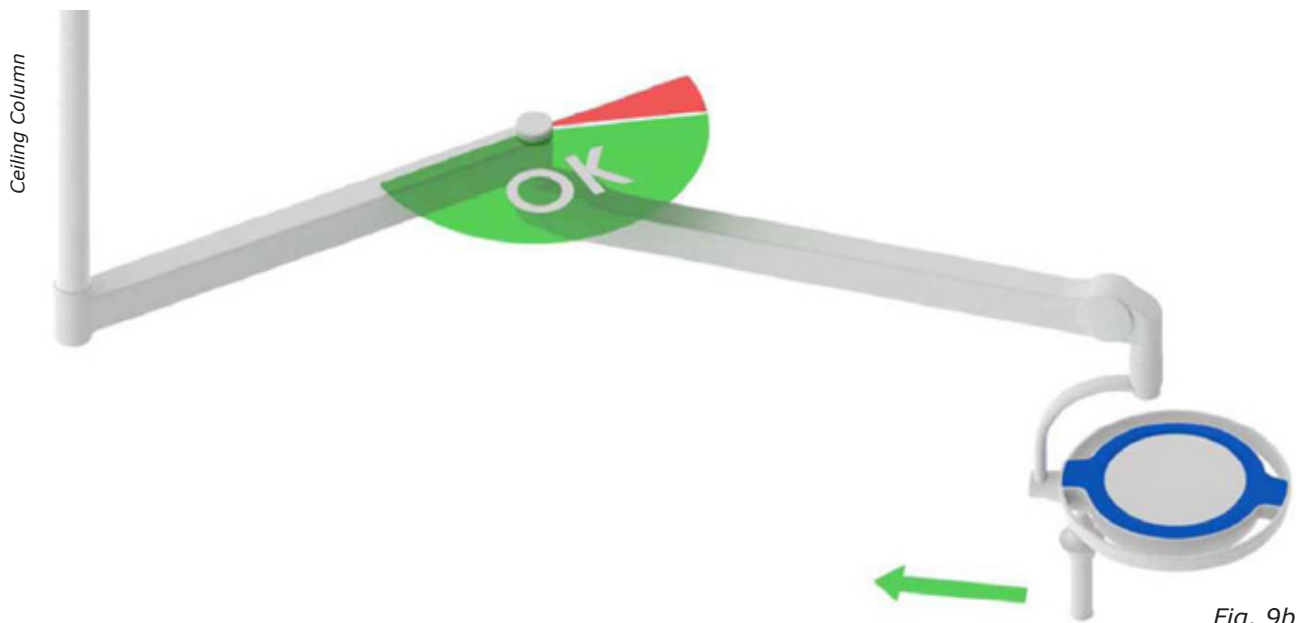
Fig. 9b

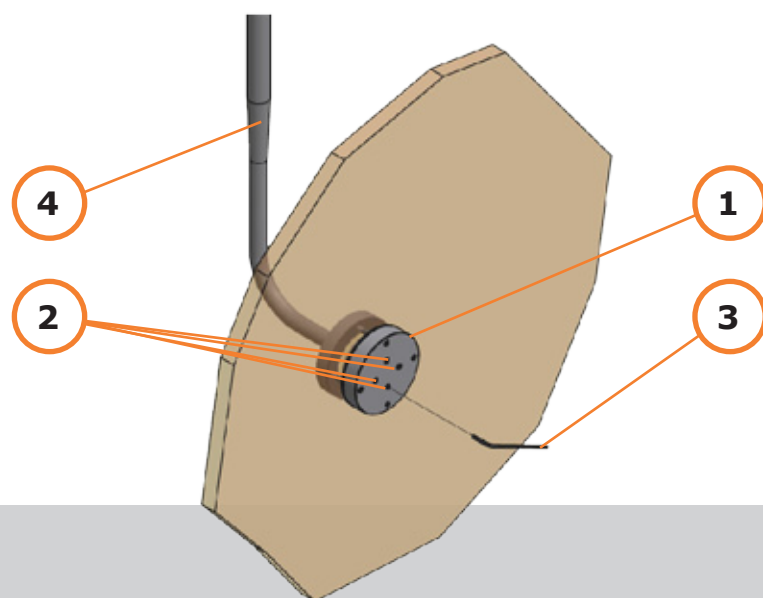
Fig. 10

4.0 Positioning of extension arm - spring arm

- 4.0.1 When the spring arm and the extension arm are positioned at 180° to each other (Fig. 9a), the attachment cannot be moved in the direction of the ceiling column. Repeated exertion of force of > 25 N (acc. to DIN EN 60601-2-41) may cause damage to the attachment.
- 4.0.2 To move the attachment in the direction of the axis of suspension, the angle between the extension and spring arms must be less than 180° (Fig. 9b).

This applies to all attachments to ceiling and wall mountings (illustration is just one example).

5.0 Adjusting the tilt tension of center-mounted shields

- 5.0.1 Move the shield, around this ball joint to validate its movability. The shield should remain in each position without further movement. If the shield moves out of position in relation to its center mount (1/Fig. 10), adjust the tilt tension of the center mount by tightening the 4 inside screws (2/Fig. 10) with the 2 mm Allen wrench (3/Fig. 10).

NOTE: A 2 mm Allen Key (3/Fig. 10) comes delivered with the shield.

WARNING – Risk of injury and breakage.

1. When loosening the 4 inside screws (2/Fig. 10), make sure to secure the shield against tilting. Otherwise, the shield could move unexpectedly and become damaged from coming in contact with other objects.
2. Never grease the ball connection joint of the shield's center mount.
3. Never loosen the screws of the shield on the side of the connection element (4/Fig. 10), otherwise the center shield joint may become instable.

6.0 Installation / Removal

The information required for installation and removal can be found in the Portegra2 installation instructions provided (POR03Oxx). Only qualified personnel may carry out this work.

7.0 Cleaning / Disinfection

General safety instructions for cleaning and disinfection

WARNING – Electric shock.

Extreme care must be taken when cleaning electrical devices. ***Danger of death!***

1. Disconnect the device from the mains electricity before cleaning.
2. Wipe only. Use cleaning liquid and/or use wipe disinfectants sparingly.
Do not spray clean or use spray disinfectants.
3. No moisture should penetrate the system, e.g. the sockets, gas outlets, or device apertures.

ATTENTION: Only trained personnel can clean and disinfect the system/equipment. The methods used to clean and disinfect must comply with the applicable statutory provisions and guidelines in regards to hygiene, disinfection, and explosion protection.

ATTENTION: The Portegra2 system/devices cannot be sterilised.

7.1 Cleaning

7.1.1 Follow the general safety instructions as stated in section 2.0 on page 6.

7.1.2 ***Cleaning agents/aids and use***

- I. Clean the surfaces with a mild soap solution or a commercially available detergent.
To avoid damage to surfaces, do not use any scouring agents, bleaches, or abrasive, corrosive, or acidic cleaning agents or polishes.
- II. To wipe surfaces, use a soft cloth free of particles or debris of any kind. Use liquid sparingly when cleaning and ensure that no moisture penetrates the system.

Only slightly dampen the cloth. If necessary, use a mild soap solution or commercially available detergent. Rub the cleaned areas dry with a cotton cloth.

7.0 Cleaning / Disinfection

7.2 Disinfection

**CAUTION – Risk of explosion.**

If disinfectants are used that form explosive gas mixtures, they pose a risk to life and health because of the risk of explosion.

7.2.1 Follow the general safety instructions as stated in section 2.0 on page 6.

7.2.2 ***Disinfection procedures***

- I. Only use wipe disinfectants; do not use any spray disinfectants.
- II. The procedures adopted must comply with the applicable statutory provisions and guidelines. The operator must define hygiene guidelines and the corresponding safety measures for the disinfection procedures used.
- III. If the system is contaminated with infectious or potentially infectious material (e.g. blood, bodily fluids, excrement), the relevant surfaces must be disinfected immediately.
- IV. Surfaces should be routinely disinfected in accordance with the hygiene/disinfection system established by the operator.
- V. The combined use of different disinfectants and/or disinfectants and cleaning agents may impair product performance or damage system surfaces.
- VI. Always consider the material compatibility (e.g. stainless steel, aluminium, plastics, lead acrylic glass, lead glass, PVC, silicon, natural or synthetic rubber, as well as chromed, painted or coated surfaces).
- VII. If unsure, apply the disinfectant on a small test area.

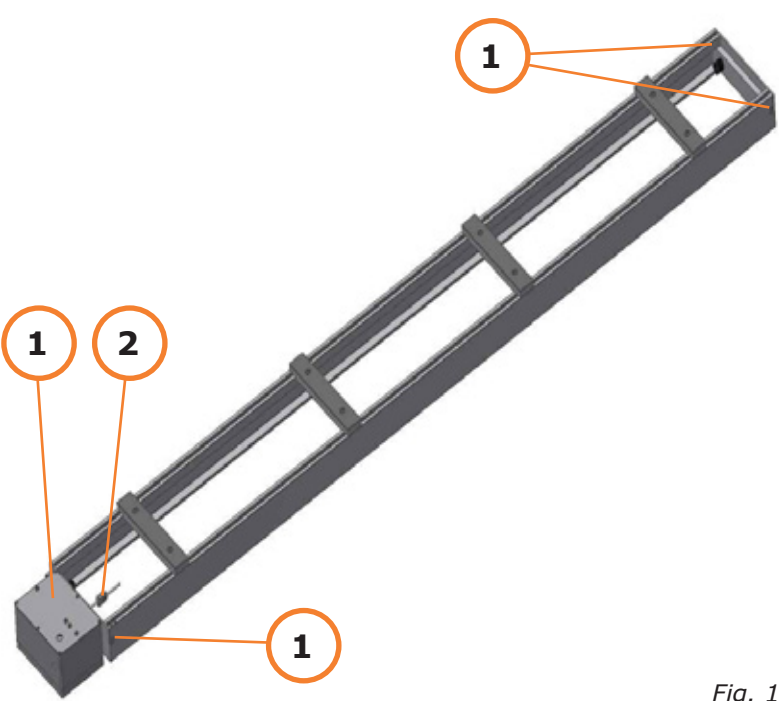
Fig. 11

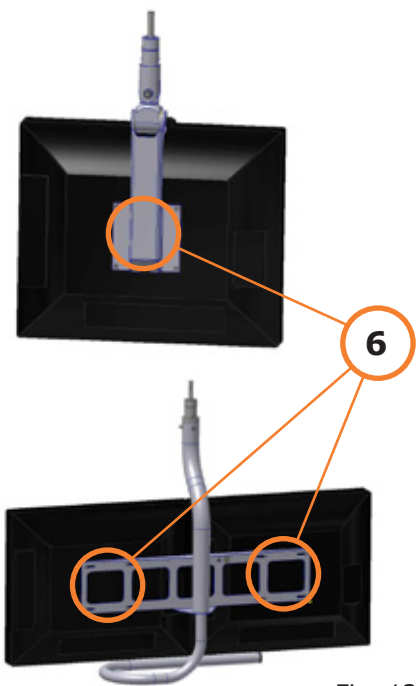
Fig. 12

Twin column

360° column

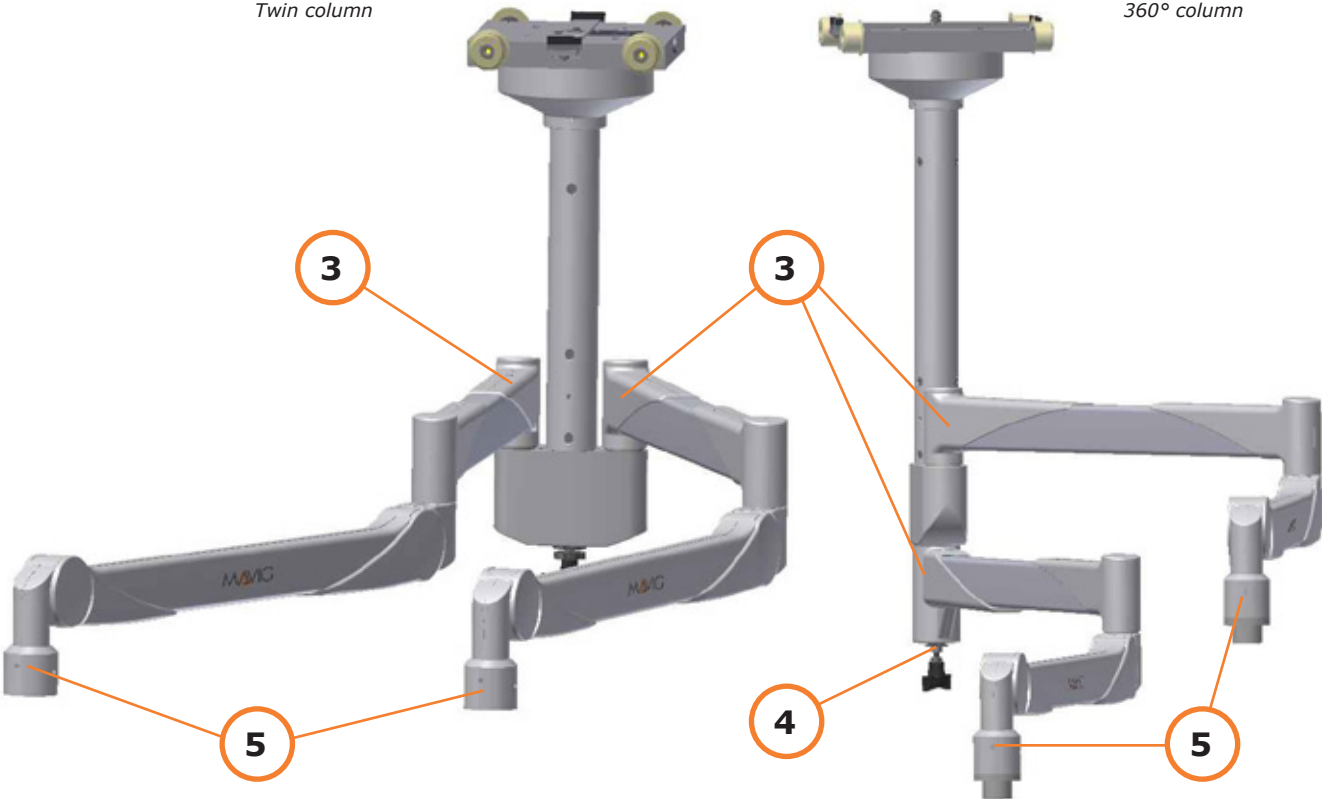
Fig. 13

8.0 Servicing

CAUTION – Damage to threadlocking adhesive.

Do not use any tools to check screws fastened with Loctite. Doing so could damage the threadlocking adhesive.

CAUTION – Risk of injury.

If components have loosened or you are not sure whether the components are still in good order, please advise the relevant service technician of the company responsible for the installation.

Stop all use of the device until the damage is rectified:

Disconnect the system from the mains. Do not stand or work beneath the system.

8.0.1 Check the system for external damage prior to use. Stop using a system/device if it is damaged.

8.0.2 Visual and performance checks for damage or stiffness in the system must be carried out by the user **every six months**.

8.0.3  The following tests must be carried out by a qualified service technician every **2 years**. Refer to the information in the Portegra2 installation instructions (POR03Oxx) where needed.

8.0.4 Check all connections of movable parts for performance and smooth movement.

8.0.5 Perform a purely visual inspection of the items below. All the screw heads must be firmly seated in the components.

Ceiling track: Check the screws on the end cap(s) and/or cable reel (1/Fig. 11).

Cable reel: (if present) Check the cable (2/Fig. 11) for damage. Check by moving the column to see if the cable is still reeled in properly.

360° - column: *Check all swivel joints for looseness.*
Check that the locking ring on the mounting pin (3/Fig. 13) is still entirely in the groove. (beneath the covering). Check the screws (4/Fig. 13) at the 360° mounting (beneath the covering).
Check the screws in the safety collars (5/Fig. 13).

TWIN - column: *Check all swivel joints for looseness.*
Check that the locking rings on the mounting pins on both arms (if present) (3/Fig. 13) are still entirely in the groove (beneath the covering).
Check the screws in the safety collars (5/Fig. 13).

Monitor support: The screws between the VESA mounting and the monitor must be checked on all monitor supports (6/Fig. 12).

To adjust the settings, such as the spring tension and the height of the spring arm, please refer to the information in the Portegra2 installation instructions (POR03Oxx).

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**Ti-Ba Enterprises, Inc.