

# AGFA HEALTHCARE FHIR Conformance Statement



## Enterprise Imaging 8.2.1

Document No. 001655 - Revision: 4

Livelihood NodeID: 79390343

**When printed, this is NOT a controlled copy**

## Document Information

---

<b>Service-related contact information worldwide</b>	All service-related contact information is available on this URL→	<a href="https://global.agfahealthcare.com/main/contact/">https://global.agfahealthcare.com/main/contact/</a>

Issued by:  
Agfa HealthCare  
V&V Connectivity  
Septestraat 27  
B-2640 Mortsel  
Belgium

email : [connectivity@agfa.com](mailto:connectivity@agfa.com)

Agfa HealthCare shall not be liable for errors contained herein or for incidental or consequential damages in connection with the furnishing, performance or use of this publication. Agfa reserves the right to revise this publication and to make changes to its content at any time, without obligation to notify any person or entity of such revisions and changes. This publication may only be used in connection with the promotion, sales, installation and use of Agfa equipment by Agfa personnel.

**Copyright © 2022**  
**Agfa HealthCare**  
**All rights reserved**

## Table of Contents

---

1	Introduction .....	7
1.1	Revision Record .....	7
1.2	Purpose and Intended Audience of this Document .....	7
1.3	About FHIR .....	7
1.4	General Remarks .....	8
1.4.1	Integration and Validation Activities .....	8
1.4.2	Future Evolution .....	8
1.5	Acronyms and Abbreviations .....	9
1.6	Related Documents .....	9
2	Capability Statement .....	10
2.1	RESTful Interactions .....	10
2.2	Search Parameters & Operations per resource type .....	10
2.2.1	CodeSystem .....	10
2.2.2	DocumentReference .....	10
2.2.3	List .....	11
2.2.4	Patient .....	11
2.2.5	Practitioner .....	12
2.2.6	Procedure .....	12
2.2.7	ServiceRequest .....	12
2.2.7.1	Search Parameters .....	12
2.2.7.2	Operation Definition .....	14
2.2.8	StructureDefinition .....	14
2.2.9	Task .....	14
2.2.9.1	Search Parameters .....	14
2.2.9.2	Operation Definition .....	15
2.2.10	ValueSet .....	28
2.3	Operations Overview .....	28
3	Profiles .....	32
3.1	EIActivitiesOverview .....	32
3.2	EIAddress .....	32
3.3	EIAssignmentGroup .....	33
3.4	EIAttachment .....	33
3.5	EIBlob .....	35
3.6	EIContactPoint .....	36
3.7	EIDepartment .....	36
3.8	EIFacility .....	37
3.9	EIIDentifier .....	38
3.10	EIMeasurementQuantity .....	39
3.11	EIMultiMediaReference .....	39
3.12	EIPatient .....	41
3.13	EIPatientAdmission .....	43
3.14	EIPatientAllergy .....	45
3.15	EIPatientPregnancy .....	46
3.16	EIPatientSafetyHazard .....	47
3.17	EIPerformedProcedureStep .....	48
3.18	EIPersonName .....	49

3.19	EIProfessional.....	49
3.20	EIQueryList.....	51
3.21	EIReportBlob.....	51
3.22	EIReportMeasurements.....	52
3.23	EIReportResultAsDiagnosticReport.....	52
3.24	EIReportResultAsObservation.....	53
3.25	EIReportSection.....	58
3.26	EIReportSoundFile.....	59
3.27	EIRequestedProcedureAsProcedure.....	59
3.28	EIRequestedProcedureAsServiceRequest.....	62
3.29	EIRoom.....	64
3.30	EIScheduledProcedureStep.....	64
3.31	EIServiceRequest.....	65
3.32	EIStructuredReport.....	67
3.33	EIStudy.....	68
3.34	EITaskInstance.....	70
3.35	EITransfer.....	72
4	Extensions.....	75
4.1	EIADTBasedExtension.....	75
4.2	EIAnnotationCommentKeyExtension.....	75
4.3	EIAttachmentContentExtension.....	75
4.4	EIAttachmentExtension.....	76
4.5	EIAttachmentLinkExtension.....	76
4.6	EIClassExtension.....	77
4.7	EIConfigLabelExtension.....	77
4.8	EICountersExtension.....	77
4.9	EIDataRetrievalStatusExtension.....	78
4.10	EIDigitalDictationExtension.....	78
4.11	EIInternalIdExtension.....	79
4.12	EILocationDetailsExtension.....	79
4.13	EIMatchingIssuerExtension.....	79
4.14	EIModifiedByExtension.....	80
4.15	EIMultiMediaReferenceContentExtension.....	80
4.16	EIOnlineRecognitionExtension.....	80
4.17	EIOriginalPatientExtension.....	81
4.18	EIParticipationTypeExtension.....	81
4.19	EIPatientExtension.....	82
4.20	EIPregnancyLinkExtension.....	82
4.21	EIPrimaryBodyPartExtension.....	82
4.22	EIQuantityValueExtension.....	83
4.23	EIReportResultExtension.....	83
4.24	EIReportResultStatisticsExtension.....	84
4.25	EIReportResultStatusExtension.....	84
4.26	EIReportSectionExtension.....	84
4.27	EIReportSeverityExtension.....	85
4.28	EIRequestedProcedureAsProcedureExtension.....	86
4.29	EIRequestedProcedurePriorityExtension.....	87
4.30	EIScheduledProcedureStepStatusExtension.....	87
4.31	EISecUserExtension.....	87
4.32	EIServiceRequestOriginalPatientIDExtension.....	88

4.33	EIServiceRequestRequesterExtension .....	88
4.34	EIServiceTypeCodeExtension .....	89
4.35	EISourceExtension .....	89
4.36	EIStudyExtension .....	89
4.37	EITaskInfoExtension .....	89
4.38	EITaskInstanceAdditionRemovalExtension .....	90
4.39	EITaskInstancePriorityExtension .....	90
4.40	EITaskInstanceResponsiblesExtension .....	90
4.41	EITaskInstanceSubtypeExtension .....	91
4.42	EITextSectionExtension .....	91
4.43	EITransferLinkExtension .....	91
4.44	EITypeCodeExtension .....	91
4.45	EIVolumeTypeExtension .....	92
<b>5</b>	<b>Codesystems &amp; Valuesets .....</b>	<b>93</b>
5.1	EIAttachmentLevelCodeSystem .....	93
5.1.1	EIAttachmentLevelValueSet .....	93
5.2	EIAttachmentScopeCodeSystem .....	93
5.2.1	EIAttachmentScopeValueSet .....	93
5.3	EIAttachmentTypeCodeSystem .....	94
5.3.1	EIAttachmentTypeValueSet .....	94
5.4	EICommentLogCommentKeyCodeSystem .....	94
5.4.1	EICommentLogCommentKeyValueSet .....	95
5.5	EIDataRetrievalStatusCodeSystem .....	95
5.5.1	EIDataRetrievalStatusValueSet .....	95
5.6	EIDocumentReferenceTypeCodeSystem .....	96
5.6.1	EIDocumentReferenceTypeValueSet .....	96
5.7	EIDocumentTypeCodeSystem .....	96
5.7.1	EIDocumentTypeValueSet .....	96
5.8	EIIDentifierTypeCodeSystem .....	97
5.8.1	EIIDentifierTypeValueSet .....	97
5.9	EIIInConflictReasonCodeSystem .....	97
5.9.1	EIIInConflictReasonValueSet .....	98
5.10	EILateralityCodeSystem .....	98
5.10.1	EILateralityValueSet .....	98
5.11	EIListTypeCodeSystem .....	98
5.11.1	EIListTypeValueSet .....	99
5.12	EIOrganizationTypeCodeSystem .....	99
5.12.1	EIOrganizationTypeValueSet .....	99
5.13	EIPatientClassCodeSystem .....	99
5.13.1	EIPatientClassValueSet .....	99
5.14	EIPerformerFunctionCodeSystem .....	100
5.14.1	EIPerformerFunctionValueSet .....	100
5.15	EIPriorityLevelCodeSystem .....	100
5.15.1	EIPriorityLevelValueSet .....	101
5.16	EIProcedureTypeCodeSystem .....	101
5.16.1	EIProcedureTypeValueSet .....	101
5.17	EIQueryListTypeCodeSystem .....	101
5.17.1	EIQueryListTypeValueSet .....	102
5.18	EIReportResultStatusCodeSystem .....	102
5.18.1	EIReportResultStatusValueSet .....	102
5.19	EIReportSectionContentTypeCodeSystem .....	102

5.19.1	EIReportSectionContentTypeValueSet .....	103
5.20	EIScheduledProcedureStepStatusCodeSystem .....	103
5.20.1	EIScheduledProcedureStepStatusValueSet .....	103
5.21	EIServiceRequestTypeCodeSystem .....	104
5.21.1	EIServiceRequestTypeValueSet .....	104
5.22	EIStudyAvailabilityCodeSystem .....	104
5.22.1	EIStudyAvailabilityValueSet .....	104
5.23	EITaskInputDataCodeSystem .....	105
5.23.1	EITaskInputDataValueSet .....	106
5.23.2	EITaskInputMapsValueSet .....	106
5.24	EITaskInputRequestedProceduresCodeSystem .....	106
5.24.1	EITaskInputRequestedProceduresValueSet .....	107
5.25	EITaskInputVariablesCodeSystem .....	107
5.25.1	EITaskInputVariablesValueSet .....	108
5.26	EITaskInstanceSubtypeCodes .....	108
5.26.1	EITaskInstanceSubtypeCodesValueSet .....	109
5.27	EITaskInstanceTypeCodes .....	109
5.28	EIUnverifiedTypeCodeSystem .....	110
5.28.1	EIUnverifiedTypeValueSet .....	110

# 1 INTRODUCTION

## 1.1 Revision Record

FHIR Conformance Statement Template, Livelink Node ID: 65788844		
Version	Date	Reason for Change
2	August 2018	Initial revision
3	February 2019	all corrections made in review cycle
4	February 2019	Chapter 2 changes

FHIR Conformance Statement Enterprise Imaging 8.2.1		
Version	Date	Reason for Change
2-3-4	July 2022	Initial Revision and corrections made in review cycle

## 1.2 Purpose and Intended Audience of this Document

### **DISCLAIMER:**

This document is a Technology Preview of the FHIR capabilities of Enterprise Imaging 8.2.1. Some profiles, extensions or parameters may be deprecated without notice in future versions of Enterprise Imaging. Therefore, Agfa HealthCare shall not be liable for loss of functionalities with non Agfa products which implemented a FHIR interface with EI 8.2.1, in case Enterprise Imaging is upgraded.

This document is a FHIR Conformance Statement for the FHIR Services of Enterprise Imaging 8.2.1.

The user of this document is involved with system integration and/or software design. We assume that the reader is familiar with the terminology and concepts that are used in FHIR standard and the IHE Technical Framework.

Readers not familiar with FHIR terminology should first read the appropriate parts of the FHIR standard itself, prior to reading this conformance statement.

Although the use of this conformance statement in conjunction with the FHIR standard is intended to facilitate communication with Enterprise Imaging 8.2.1, it is not sufficient to guarantee, by itself, the inter-operation of the connection between Enterprise Imaging 8.2.1 and the 3rd party FHIR-based system.

The integration of any device into a system of interconnected devices goes beyond the scope of the FHIR standard and this conformance statement when *interoperability* is desired. The responsibility for analyzing the applications requirements and developing a solution that integrates the Agfa equipment with other vendors' systems is the user's responsibility and should not be underestimated.

## 1.3 About FHIR

Fast Healthcare Interoperability Resources (FHIR, pronounced "Fire") defines a set of "Resources" that represent granular clinical concepts. The resources can be managed in isolation, or aggregated into complex documents. This flexibility offers coherent solutions for a range of interoperability problems. The simple direct definitions of the resources are based on thorough requirements gathering, formal analysis and extensive cross-mapping to other relevant standards.

Each resource carries a master id. The id is never changed or reused, and it identifies the resource permanently. Resources may refer to other resources by id knowing that this is stable reference. Each resource has a URL which is derived from the id, the type, and the local base URL. Given one resource address, the address of any other resource can be automatically determined.

Resources contain references to other resources. While each resource can be read and/or changed without explicit reference to these other resources, the presence of these references influences the behaviour of the system: implementations are required to maintain system and data integrity at all times.

Each resource supports the same list of transactions - read, update, delete, etc. One particularly important transaction supported by every resource type is the provision of a conformance statement which specifies what parts of the defined content model are supported by the system, and what other transactions or interactions are supported. If any of the other interactions are supported, the conformance interaction must be supported. (i.e. if the conformance interaction returns an error, no operations are supported).

The exchange specifications are simple and straight forward and based around direct description of the XML representation of the resource. Each resource is described separately, though there are some common patterns used across all the resources (called "data types"). In addition to the simple XML definitions, a schema and UML class diagram are available for each resource. The UML class diagram represents the same logical model as the XML format (though because of UML issues, implementors should not expect the UML model to lead to interoperable implementations).

Further, each xml element (or matching UML class, attribute and composition association) is associated with a reference into a single integrated ontology that serves as a data dictionary. As well as more precisely defining the element, the data dictionary specifies the mappings from the element into other standards.

Technically, FHIR is designed for the web; the resources are based on simple XML or JSON structures, with an http-based RESTful protocol where each resource has predictable URL. Where possible, open internet standards are used for data representation.

## 1.4 General Remarks

### 1.4.1 Integration and Validation Activities

The integration of any device into a system of interconnected devices goes beyond the scope of the FHIR standard and this conformance statement when interoperability is desired. The responsibility for analyzing the applications requirements and developing a solution that integrates the Agfa equipment with other vendors' systems is the user's responsibility and should not be underestimated.

In some circumstances it might be necessary to perform a validation to make sure that functional interoperability between the Agfa equipment and non-Agfa devices works as expected. The user should ensure that any non-Agfa provider accepts responsibility for any validation required for their connection with the Agfa equipment.

### 1.4.2 Future Evolution

As the FHIR standard evolves to meet the user's growing requirements and to incorporate new features and technologies, Agfa will follow the evolution of the standard. This evolution of the standard may require changes to devices that have implemented FHIR. The user should ensure that any non-Agfa provider, who connects with Agfa devices, also plans for



future evolution of the FHIR standard. A refusal to do so may result in the loss of functionality and/or connectivity between the different products.

## 1.5 Acronyms and Abbreviations

Definitions, terms and abbreviations used in this document, many of which are defined within the FHIR standard. Abbreviations and terms are as follows:

FHIR	Fast Healthcare Interoperability Resources
HL7	Health Level 7
IHE	Integrating the Healthcare Enterprise
RESTful	Representational State Transfer ( <b>REST</b> )
UML	Unified Modeling Language
URI	Uniform Resource Identifier
URL	Uniform Resource Locator
XML	Extensible Markup Language

## 1.6 Related Documents

- FHIR Standard v4.0.1 - see [www.hl7.org/fhir](http://www.hl7.org/fhir)
- HL7 Standard v2.3.1 - see [www.hl7.org](http://www.hl7.org)
- [IHE Radiology Technical Framework](#)
- [IHE IT Infrastructure Technical Framework](#)

## 2 CAPABILITY STATEMENT

### 2.1 RESTful Interactions

Resource Type	Profile	Create	Read	Update	Delete	SearchType
AllergyIntolerance	AllergyIntolerance		√			
Binary	Binary		√	√		
CodeSystem	CodeSystem		√			√
DiagnosticReport	DiagnosticReport		√			
DocumentReference	DocumentReference	√	√	√	√	√
Encounter	Encounter		√			
ImagingStudy	ImagingStudy		√			
List	List		√			√
Location	Location		√			
Observation	Observation		√	√		
OperationDefinition	OperationDefinition		√			
Organization	Organization		√			
Patient	Patient		√			√
Practitioner	Practitioner		√			√
Procedure	Procedure		√			√
ServiceRequest	ServiceRequest		√			√
StructureDefinition	StructureDefinition		√			√
Task	Task		√			√
ValueSet	ValueSet		√			√

### 2.2 Search Parameters & Operations per resource type

#### 2.2.1 CodeSystem

Name	Type	Documentation
url	uri	The uri that identifies the code system

#### 2.2.2 DocumentReference

Name	Type	Documentation
related-identifier	token	
subject.name	string	Who/what is the subject of the document
subject.identifier	token	Who/what is the subject of the document
category	token	Categorization of document
multimedia-mime-type	string	

Supported Includes:  
DocumentReference:author  
DocumentReference:related

### 2.2.3 List

Name	Type	Documentation
listConsumer	string	
code	token	What the purpose of this list is
status	string	current   retired   entered-in-error
mode	string	
_id	reference	The logical id of the resource, WORKSCHEMA#ID with prefix "ao-", QUERY_LIST#ID with prefix "ql-"
item.status	string	Actual entry

Supported Includes:  
 Basic:activity  
 DiagnosticReport:result  
 DocumentReference:author  
 List:entry.item  
 Observation:has-member  
 Observation:report-blob  
 Patient:current-transfer  
 Procedure:attachments  
 Procedure:pregnancy  
 Procedure:report  
 Procedure:transfer-at-acquisition  
 ServiceRequest:attachment  
 ServiceRequest:based-on  
 ServiceRequest:imagingStudies  
 ServiceRequest:performer  
 ServiceRequest:requester

ServiceRequest:requesting-department  
 ServiceRequest:requesting-facility  
 ServiceRequest:subject  
 Task:excluded-responsibles  
 Task:input-requested-procedure  
 Task:owner  
 Task:responsibles  
 Task:subject  
 Task:taskType

Supported Reverse Includes:  
 ImagingStudy:basedon  
 Procedure:based-on  
 ServiceRequest:based-on

### 2.2.4 Patient

Name	Type	Documentation
_id	string	The ID of the resource
identifier	token	A patient identifier
name	string	A server defined search that may match any of the string fields in the HumanName, including family, give, prefix, suffix, suffix, and/or text
birthdate	date	The patient's date of birth
active	token	Whether the patient record is active
gender	string	Gender of the patient
family	string	A portion of the family name of the patient
given	string	A portion of the given name of the patient
address-postalcode	string	A postalCode specified in an address
address-city	string	A city specified in an address
address	string	A server defined search that may match any of the string fields in the Address,

		including line, city, district, state, country, postalCode, and/or text
assigner.display-value-identifier	composite	

Supported Includes:  
Patient:current-transfer  
Patient:link

## 2.2.5 Practitioner

Name	Type	Documentation
userId	string	
domain	string	

## 2.2.6 Procedure

Name	Type	Documentation
_id	reference	The logical id of the resource, support the Read with the special security setup, REQUESTED_PROCEDURE#ID with prefix "rqp-", PERFORMED_PROCEDURE_STEP#ID with prefix "pps-"

Supported Includes:  
Procedure:attachments  
Procedure:based-on

Supported Reverse Includes:  
DocumentReference:related

## 2.2.7 ServiceRequest

### 2.2.7.1 Search Parameters

Name	Type	Documentation
all-comparison-active-study-id	number	
_id	reference	The logical id of the resource, support the Read with the special security setup, REQUESTED_PROCEDURE#ID with prefix "rpq-", SCHEDULED_PROCEDURE_STEP#ID with prefix "sps-", SERVICE_REQUEST#ID with prefix "srq-"
identifier	string	Identifiers assigned to this order
non-prefetched-comparison-active-study-id	number	
non-prefetched-comparison-prefetched-comparisons	number	
non-prefetched-comparison-taskid	number	
relevant-comparison-studies-task-id	number	

Name	Type	Documentation
relevant-comparison-studies-studies-to-include	string	
relevant-comparison-studies-allowed-studies	number	
relevant-comparison-studies-patient-id	number	
category	token	Classification of service
subject.name	string	Search by subject
subject.family	string	Search by subject
subject.given	string	Search by subject
subject.address-postalcode	string	Search by subject
subject.address-city	string	Search by subject
subject.address	string	Search by subject
subject.identifier	token	Search by subject
subject.assigner.display-value-identifier	composite	Search by subject
subject.active	string	Search by subject
subject.birthdate	date	Search by subject
subject.gender	string	Search by subject
performer.name	string	Requested performer
performer.family	string	Requested performer
performer.given	string	Requested performer
performing-department	reference	
performing-facility	reference	
based-on.requester._id	reference	What request fulfills
performer._id	reference	Requested performer
based-on.requester.name	string	What request fulfills
based-on.requester.family	string	What request fulfills
based-on.requester.given	string	What request fulfills
based-on.requesting-department	reference	What request fulfills
based-on.requesting-facility	reference	What request fulfills
occurrence	date	When service should occur
occurrence-period	string	
based-on.authored-period	string	What request fulfills
priority	token	routine   urgent   asap   stat
status	string	draft   active   on-hold   revoked   completed   entered-in-error   unknown
code	token	What is being requested/ordered
_has:Procedure:based-on:body-site	string	
_has:Procedure:based-on:performed-period	string	
_has:Procedure:based-on:performed	date	
_has:Procedure:based-on:report-available	string	

Name	Type	Documentation
_has:Procedure:based-on:report-status	string	
_has:Task:input-requested-procedure:status	string	
_has:Task:input-requested-procedure:type	string	
_has:Procedure:based-on:modality-type	string	
_has:ImagingStudy:basedon:volume-type	string	
_has:Task:input-requested-procedure:responsibles-with-group	string	

Supported Includes:  
 DiagnosticReport:result  
 DocumentReference:author  
 Observation:dictation-soundfile  
 Observation:has-member  
 Observation:recognition-smdfile  
 Observation:recognition-srrfile  
 Observation:recognition-wavfile  
 Observation:report-blob  
 Patient:current-transfer  
 Patient:pregnancy  
 xProcedure:attachments  
 Procedure:pregnancy  
 Procedure:report  
 Procedure:transfer-at-acquisition

ServiceRequest:attachment  
 ServiceRequest:based-on  
 ServiceRequest:performer  
 ServiceRequest:requester  
 ServiceRequest:requesting-department  
 ServiceRequest:requesting-facility  
 ServiceRequest:subject  
 Task:note.author

Supported Reverse Includes:  
 DocumentReference:related  
 ImagingStudy:basedon  
 Procedure:based-on  
 ServiceRequest:based-on

## 2.2.7.2 Operation Definition

addToConference

Parameter Name	Use	Min	Max	Type
conferenceCode	in	0	1	Coding
identifier	in	0	*	Coding

## 2.2.8 StructureDefinition

Name	Type	Documentation
url	uri	The uri that identifies the structure definition

## 2.2.9 Task

### 2.2.9.1 Search Parameters

Name	Type	Documentation
_id	token	The logical id of the resource, support the Read with the special security setup
patient-id	string	
input-requested-procedure	string	

Supported Includes:  
 DiagnosticReport:result

ServiceRequest:performer  
 ServiceRequest:requester

DocumentReference:author  
 Observation:dictation-soundfile  
 Observation:has-member  
 Observation:recognition-smdfile  
 Observation:recognition-srrfile  
 Observation:recognition-wavfile  
 Observation:report-blob  
 Patient:current-transfer  
 Patient:pregnancy  
 Procedure:attachments  
 Procedure:pregnancy  
 Procedure:report  
 Procedure:transfer-at-acquisition  
 ServiceRequest:attachment  
 ServiceRequest:based-on  
 ServiceRequest:requesting-department  
 ServiceRequest:requesting-facility  
 ServiceRequest:subject  
 Task:excluded-responsibles  
 Task:input-requested-procedure  
 Task:input.valueReferenceowner  
 Task:note.author  
 Task:owner  
 Task:responsibles  
 Supported Reverse Includes:  
 ImagingStudy:basedon  
 Procedure:based-on  
 ServiceRequest:based-on

## 2.2.9.2 Operation Definition

### assignTasksToMe

Parameter Name	Use	Min	Max	Type
tasks	in	0	1	Reference
priorityLevel	in	0	1	Coding
comment	in	0	1	

### cancelAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reason	in	0	1	

### cancelReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reason	in	0	1	

## changePriority

Parameter Name	Use	Min	Max	Type
tasks	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
comment	in	0	1	

## changeTasksAssignment

Parameter Name	Use	Min	Max	Type
tasks	in	0	1	Reference
priorityLevel	in	0	1	Coding
responsibles	in	0	1	Reference
comment	in	0	1	

## completeAcquisition

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## createAddendumWorkflow

Parameter Name	Use	Min	Max	Type
requestedProcedures	in	0	*	decimal
newTaskAssignees	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
newTaskComment	in	0	1	

## createAndStartAddendumTask

Parameter Name	Use	Min	Max	Type
requestedProcedures	in	0	*	decimal
newTaskAssignees	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
newTaskComment	in	0	1	
externalReporting	in	0	1	boolean



## createReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
requestedProcedures	in	0	*	Reference
newTaskAssignees	in	0	1	Reference

## noReportNeededReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## pause

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
actionProvider	in	0	1	

## pauseAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date

Parameter Name	Use	Min	Max	Type
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary

## pauseReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
isRecoveredTask	in	0	1	boolean
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary

## pauseReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reportResult	in	0	1	Observation

Parameter Name	Use	Min	Max	Type
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary

## removePreliminaryReportAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## removePreliminaryReportReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## reset

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
actionProvider	in	0	1	

## resetAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## resetReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## resetReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## resume

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	

Parameter Name	Use	Min	Max	Type
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
actionProvider	in	0	1	

## resumeAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## resumeReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## resumeReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## revertAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## revertReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## revertReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## sendToCorrectionistAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	

Parameter Name	Use	Min	Max	Type
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary
newTaskAssignees	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
newTaskComment	In	0	1	

## sendToCorrectionistReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary
newTaskAssignees	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
newTaskComment	in	0	1	

## sendToCorrectionistReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	

Parameter Name	Use	Min	Max	Type
selectClause	in	0	1	
newTaskAssignees	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
newTaskComment	in	0	1	

## signOffAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary

## signOffLaterAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
requestedProcedures	in	0	*	decimal
newTaskAssignees	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
newTaskComment	in	0	1	
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary



Parameter Name	Use	Min	Max	Type
wavFile	in	0	1	Binary

## signOffLaterReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
requestedProcedures	in	0	*	decimal
newTaskAssignees	in	0	1	Reference
newTaskPriorityLevel	in	0	1	Coding
newTaskComment	in	0	1	
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary

## signOffReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
reportResult	in	0	1	Observation
dictationFile	in	0	1	Binary
srrFile	in	0	1	Binary
smdFile	in	0	1	Binary
wavFile	in	0	1	Binary

## startAcquisition

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## startAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
externalReporting	in	0	1	boolean

## startOrResumeAddendum

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
activityContext	in	0	1	
selectClause	in	0	1	
activeStudies	in	0	*	decimal
autoMerge	in	0	1	
externalReporting	in	0	1	boolean

## startOrResumeReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
activityContext	in	0	1	
selectClause	in	0	1	
activeStudies	in	0	*	decimal
autoMerge	in	0	1	
externalReporting	in	0	1	boolean

## startOrResumeReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
activityContext	in	0	1	
selectClause	in	0	1	
activeStudies	in	0	*	decimal
autoMerge	in	0	1	
externalReporting	in	0	1	Boolean

## startReading

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	
activeStudies	in	0	*	decimal
autoMerge	in	0	1	
externalReporting	in	0	1	boolean

## startReportApproval

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## startVerifyReport

Parameter Name	Use	Min	Max	Type
priorityLevel	in	0	1	Coding
expirationDate	in	0	1	date
dueDate	in	0	1	date
scheduledStartDate	in	0	1	date
comment	in	0	1	

Parameter Name	Use	Min	Max	Type
commentKey	in	0	1	
activityContext	in	0	1	
selectClause	in	0	1	

## 2.2.10 ValueSet

Name	Type	Documentation
url	uri	The uri that identifies the value set

## 2.3 Operations Overview

Operation	Reference
addMeasurementsToSection	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Observation-i-addMeasurementsToSection">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Observation-i-addMeasurementsToSection</a>
followup	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Observation-i-followup">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Observation-i-followup</a>
getCurrentProfessional	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Practitioner--getCurrentProfessional">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Practitioner--getCurrentProfessional</a>
userDomainQuery	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Practitioner-query-userDomainQuery">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Practitioner-query-userDomainQuery</a>
addToConference	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ServiceRequest--addToConference">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ServiceRequest--addToConference</a>
get-linked-studies	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ServiceRequest-i-get-linked-studies">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ServiceRequest-i-get-linked-studies</a>
assignTasksToMe	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--assignTasksToMe">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--assignTasksToMe</a>
cancelAddendum	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-cancelAddendum">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-cancelAddendum</a>
cancelReading	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-cancelReading">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-cancelReading</a>
changePriority	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--changePriority">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--changePriority</a>
changeTasksAssignment	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--changeTasksAssignment">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--changeTasksAssignment</a>
completeAcquisition	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-completeAcquisition">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-completeAcquisition</a>
createAddendumWorkflow	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--createAddendumWorkflow">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--createAddendumWorkflow</a>
createAndStartAddendumTask	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--createAndStartAddendumTask">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--createAndStartAddendumTask</a>
createReading	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--createReading">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task--createReading</a>
noReportNeededReading	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-noReportNeededReading">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-noReportNeededReading</a>

Operation	Reference
pause	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pause">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pause</a>
pauseAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pauseAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pauseAddendum</a>
pauseReading	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pauseReading">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pauseReading</a>
pauseReportApproval	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pauseReportApproval">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-pauseReportApproval</a>
removePreliminaryReportAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-removePreliminaryReportAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-removePreliminaryReportAddendum</a>
removePreliminaryReportReportApproval	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-removePreliminaryReportReportApproval">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-removePreliminaryReportReportApproval</a>
reset	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-reset">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-reset</a>
resetAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resetAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resetAddendum</a>
resetReading	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resetReading">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resetReading</a>
resetReportApproval	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resetReportApproval">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resetReportApproval</a>
resume	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resume">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resume</a>
resumeAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resumeAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resumeAddendum</a>
resumeReading	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resumeReading">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resumeReading</a>
resumeReportApproval	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resumeReportApproval">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-resumeReportApproval</a>
revertAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-revertAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-revertAddendum</a>
revertReading	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-revertReading">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-revertReading</a>
revertReportApproval	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-revertReportApproval">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-revertReportApproval</a>
sendToCorrectionistAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-sendToCorrectionistAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-sendToCorrectionistAddendum</a>
sendToCorrectionistReading	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-sendToCorrectionistReading">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-sendToCorrectionistReading</a>
sendToCorrectionistReportApproval	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-sendToCorrectionistReportApproval">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-sendToCorrectionistReportApproval</a>
signOffAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-signOffAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-signOffAddendum</a>
signOffLaterAddendum	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-signOffLaterAddendum">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-signOffLaterAddendum</a>
signOffLaterReading	<a href="https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-signOffLaterReading">https://&lt;Elservice&gt;/fhir/r4/OperationDefinition/Task-i-signOffLaterReading</a>

Operation	Reference
signOffReading	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-signOffReading">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-signOffReading</a>
signOffReportApproval	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-signOffReportApproval">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-signOffReportApproval</a>
startAcquisition	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startAcquisition">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startAcquisition</a>
startAddendum	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startAddendum">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startAddendum</a>
startOrResumeAddendum	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startOrResumeAddendum">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startOrResumeAddendum</a>
startOrResumeReading	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startOrResumeReading">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startOrResumeReading</a>
startOrResumeReportApproval	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startOrResumeReportApproval">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startOrResumeReportApproval</a>
startReading	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startReading">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startReading</a>
startReportApproval	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startReportApproval">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startReportApproval</a>
startVerifyReport	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startVerifyReport">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-startVerifyReport</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Encounter-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Encounter-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Organization-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Organization-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/DiagnosticReport-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/DiagnosticReport-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Observation-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Observation-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Task-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Procedure-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Procedure-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Practitioner-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Practitioner-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/DocumentReference-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/DocumentReference-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ImagingStudy-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ImagingStudy-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/List-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/List-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Patient-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Patient-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Location-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/Location-i-graphql</a>
graphql	<a href="https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ServiceRequest-i-graphql">https://&lt;EIservers&gt;/fhir/r4/OperationDefinition/ServiceRequest-i-graphql</a>



## 3 PROFILES

### 3.1 EIActivitiesOverview

Profile based on the Resource List: <http://hl7.org/fhir/StructureDefinition/List>

Resource Elements	support	Card	StructureDefinition
List.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
List.identifier	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Identifier">http://hl7.org/fhir/StructureDefinition/Identifier</a>
List.identifier:ActivitiesOverviewCode		0..*	
List.identifier:ActivitiesOverviewCode.type.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
List.identifier:ActivitiesOverviewCode.type.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
List.identifier:ActivitiesOverviewCode.value	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
List.status	√	0..*	
List.title	√	0..*	
List.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
List.code.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
List.code.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
List.entry	√	0..*	
List.entry.item	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Resource">http://hl7.org/fhir/StructureDefinition/Resource</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIQueryList">http://www.agfa.com/ei/fhir/StructureDefinition/EIQueryList</a>
List.entry.item.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.2 EIAddress

Profile based on the Resource Address: <http://hl7.org/fhir/StructureDefinition/Address>

Resource Elements	support	Card	StructureDefinition
Address		0..*	
Address.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Address.use	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Address.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Address.text	√	0..*	
Address.line	√	0..*	



Resource Elements	support	Card	StructureDefinition
Address.city	√	0..*	
Address.state	√	0..*	
Address.postalCode	√	0..*	
Address.country	√	0..*	
Address.period	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Period">http://hl7.org/fhir/StructureDefinition/Period</a>
Address.period.start	√	0..*	
Address.period.end	√	0..*	

### 3.3 EIAssignmentGroup

Profile based on the Resource Organization:

<http://hl7.org/fhir/StructureDefinition/Organization>

Resource Elements	support	Card	StructureDefinition
Organization.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Organization.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Organization.type.coding.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Organization.type.coding.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Organization.name	√	0..*	

### 3.4 EIAttachment

Profile based on the Resource DocumentReference:

<http://hl7.org/fhir/StructureDefinition/DocumentReference>

Resource Elements	support	Card	StructureDefinition
DocumentReference		0..*	
DocumentReference.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
DocumentReference.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
DocumentReference.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
DocumentReference.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
DocumentReference.extension:AttachmentDetails	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentExtension</a>
DocumentReference.extension:AttachmentDetails.extension:Definition	√	0..*	
DocumentReference.extension:AttachmentDetails.extension:Definition.value[x]	√	1..1	
DocumentReference.extension:AttachmentDetails.extension:Scope	√	0..*	

Resource Elements	support	Card	StructureDefinition
DocumentReference.extension:AttachmentDetails.extension:Scope.value[x]	√	1..1	
DocumentReference.extension:AttachmentDetails.extension:Scope.value[x].system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
DocumentReference.extension:AttachmentDetails.extension:Scope.value[x].code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
DocumentReference.extension:AttachmentDetails.extension:Active	√	0..*	
DocumentReference.extension:AttachmentDetails.extension:Active.value[x]	√	1..1	
DocumentReference.extension:AttachmentDetails.extension:Level	√	0..*	
DocumentReference.extension:AttachmentDetails.extension:Level.value[x]	√	1..1	
DocumentReference.extension:AttachmentDetails.extension:Level.value[x].system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
DocumentReference.extension:AttachmentDetails.extension:Level.value[x].code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
DocumentReference.status		0..*	
DocumentReference.category	√	0..*	
DocumentReference.category.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
DocumentReference.category.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
DocumentReference.category.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
DocumentReference.date	√	0..*	
DocumentReference.author	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/PractitionerRole">http://hl7.org/fhir/StructureDefinition/PractitionerRole</a> <a href="http://hl7.org/fhir/StructureDefinition/Organization">http://hl7.org/fhir/StructureDefinition/Organization</a> <a href="http://hl7.org/fhir/StructureDefinition/Device">http://hl7.org/fhir/StructureDefinition/Device</a> <a href="http://hl7.org/fhir/StructureDefinition/Patient">http://hl7.org/fhir/StructureDefinition/Patient</a> <a href="http://hl7.org/fhir/StructureDefinition/RelatedPerson">http://hl7.org/fhir/StructureDefinition/RelatedPerson</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
DocumentReference.author.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DocumentReference.content	√	0..*	
DocumentReference.content.attachment	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Attachment">http://hl7.org/fhir/StructureDefinition/Attachment</a>
DocumentReference.content.attachment.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>

Resource Elements	support	Card	StructureDefinition
DocumentReference.content.attachment.extension:Details	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentContentExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentContentExtension</a>
DocumentReference.content.attachment.extension:Details.extension:Tag	√	0..*	
DocumentReference.content.attachment.extension:Details.extension:Tag.value[x]	√	1..1	
DocumentReference.content.attachment.extension:Details.extension:Tag.value[x].coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
DocumentReference.content.attachment.extension:Details.extension:Tag.value[x].coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
DocumentReference.content.attachment.extension:Details.extension:Tag.value[x].coding.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DocumentReference.content.attachment.extension:Details.extension:Tag.value[x].text	√	0..*	
DocumentReference.content.attachment.extension:Details.extension:FileFormat	√	0..*	
DocumentReference.content.attachment.extension:Details.extension:FileFormat.value[x]	√	1..1	
DocumentReference.content.attachment.extension:Details.extension:User	√	0..*	
DocumentReference.content.attachment.extension:Details.extension:User.value[x]	√	1..1	
DocumentReference.content.attachment.extension:Details.extension:User.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DocumentReference.content.attachment.extension:Details.extension:TagCatalog	√	0..*	
DocumentReference.content.attachment.extension:Details.extension:TagCatalog.value[x]	√	1..1	
DocumentReference.content.attachment.contentType	√	0..*	
DocumentReference.content.attachment.data	√	0..*	
DocumentReference.content.attachment.url	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
DocumentReference.context.related	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequest">http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequest</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure</a>
DocumentReference.context.related.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.5 EIBlob

Profile based on the Resource Binary: <http://hl7.org/fhir/StructureDefinition/Binary>

Resource Elements	support	Card	StructureDefinition
Binary.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>

Resource Elements	support	Card	StructureDefinition
Binary.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Binary.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Binary.contentType	√	0..*	
Binary.data	√	0..*	

### 3.6 EIContactPoint

Profile based on the Resource ContactPoint:

<http://hl7.org/fhir/StructureDefinition/ContactPoint>

Resource Elements	support	Card	StructureDefinition
ContactPoint		0..*	
ContactPoint.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ContactPoint.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
ContactPoint.value	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ContactPoint.use	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
ContactPoint.rank	√	0..*	
ContactPoint.period	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Period">http://hl7.org/fhir/StructureDefinition/Period</a>
ContactPoint.period.start	√	0..*	
ContactPoint.period.end	√	0..*	

### 3.7 EIDepartment

Profile based on the Resource Organization:

<http://hl7.org/fhir/StructureDefinition/Organization>

Resource Elements	support	Card	StructureDefinition
Organization		0..*	
Organization.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Organization.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Organization.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Organization.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Organization.extension:ConfigLabel	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIConfigLabelExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIConfigLabelExtension</a>
Organization.extension:ConfigLabel.extension:Code	√	0..*	
Organization.extension:ConfigLabel.extension:Code.value [x]	√	1..1	

Resource Elements	support	Card	StructureDefinition
Organization.extension:ConfigLabel.extension:Name	√	0..*	
Organization.extension:ConfigLabel.extension:Name.value[x]	√	1..1	
Organization.extension:ConfigLabel.extension:Active	√	0..*	
Organization.extension:ConfigLabel.extension:Active.value[x]	√	1..1	
Organization.identifier	√	0..*	http://hl7.org/fhir/StructureDefinition/Identifier
Organization.identifier:IdentityCode	√	0..1	http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier
Organization.identifier:IdentityCode.type.coding		0..*	http://hl7.org/fhir/StructureDefinition/Coding
Organization.identifier:IdentityCode.value		1..1	http://hl7.org/fhir/StructureDefinition/string
Organization.identifier:IdentityCode.assigner.display		0..1	http://hl7.org/fhir/StructureDefinition/string
Organization.active	√	0..1	http://hl7.org/fhir/StructureDefinition/boolean
Organization.type	√	0..1	http://hl7.org/fhir/StructureDefinition/uri
Organization.name	√	0..*	
Organization.telecom	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIContactPoint
Organization.address	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIAddress
Organization.partOf	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIFacility http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment
Organization.partOf.reference	√	1..1	http://hl7.org/fhir/StructureDefinition/string

### 3.8 EIFacility

Profile based on the Resource Organization:

<http://hl7.org/fhir/StructureDefinition/Organization>

Resource Elements	support	Card	StructureDefinition
Organization		0..*	
Organization.id	√	0..1	http://hl7.org/fhir/StructureDefinition/id
Organization.meta.versionId	√	0..1	http://hl7.org/fhir/StructureDefinition/id
Organization.extension		0..*	http://hl7.org/fhir/StructureDefinition/Extension
Organization.extension:ConfigLabel	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIConfigLabelExtension

Resource Elements	support	Card	StructureDefinition
Organization.extension:ConfigLabel.extension:Code	√	0..*	
Organization.extension:ConfigLabel.extension:Code.value[x]	√	1..1	
Organization.extension:ConfigLabel.extension:Name	√	0..*	
Organization.extension:ConfigLabel.extension:Name.value[x]	√	1..1	
Organization.extension:ConfigLabel.extension:Active	√	0..*	
Organization.extension:ConfigLabel.extension:Active.value[x]	√	1..1	
Organization.identifier	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Identifier">http://hl7.org/fhir/StructureDefinition/Identifier</a>
Organization.identifier:IdentityCode		0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Organization.identifier:IdentityCode.type.coding		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Organization.identifier:IdentityCode.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Organization.identifier:IdentityCode.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Organization.active	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/boolean">http://hl7.org/fhir/StructureDefinition/boolean</a>
Organization.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Organization.name	√	0..*	
Organization.telecom	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIContactPoint">http://www.agfa.com/ei/fhir/StructureDefinition/EIContactPoint</a>
Organization.address	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAddress">http://www.agfa.com/ei/fhir/StructureDefinition/EIAddress</a>

### 3.9 EIdentifier

Profile based on the Resource Identifier: <http://hl7.org/fhir/StructureDefinition/Identifier>

Resource Elements	support	Card	StructureDefinition
Identifier.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Identifier.extension:IsMatchingIssuer	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIMatchingIssuerExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIMatchingIssuerExtension</a>
Identifier.extension:IsMatchingIssuer.url		0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Identifier.extension:IsMatchingIssuer.value[x]	√	1..1	
Identifier.extension:InternalId	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIInternalIdExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIInternalIdExtension</a>
Identifier.extension:InternalId.url		0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Identifier.extension:InternalId.value[x]	√	1..1	

Resource Elements	support	Card	StructureDefinition
Identifier.use	√	1..1	http://hl7.org/fhir/StructureDefinition/code
Identifier.type.coding.display		0..1	http://hl7.org/fhir/StructureDefinition/string
Identifier.type.text		0..*	
Identifier.system	√	0..1	http://hl7.org/fhir/StructureDefinition/uri
Identifier.value	√	1..1	http://hl7.org/fhir/StructureDefinition/string
Identifier.assigner	√	0..*	
Identifier.assigner.identifier.type.coding.display		0..1	http://hl7.org/fhir/StructureDefinition/string
Identifier.assigner.identifier.type.text		0..*	
Identifier.assigner.identifier.assigner.identifier.type.coding.display		0..1	http://hl7.org/fhir/StructureDefinition/string
Identifier.assigner.identifier.assigner.identifier.type.text		0..*	
Identifier.assigner.identifier.assigner.identifier.assigner.display		0..1	http://hl7.org/fhir/StructureDefinition/string
Identifier.assigner.identifier.assigner.display		0..1	http://hl7.org/fhir/StructureDefinition/string
Identifier.assigner.display	√	0..1	http://hl7.org/fhir/StructureDefinition/string

### 3.10 EIMeasurementQuantity

Profile based on the Resource Quantity: <http://hl7.org/fhir/StructureDefinition/Quantity>

Resource Elements	support	Card	StructureDefinition
Quantity.extension		0..*	http://hl7.org/fhir/StructureDefinition/Extension
Quantity.extension:ValueString	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIQuantityValueExtension
Quantity.extension:ValueString.value[x]	√	1..1	
Quantity.unit	√	0..*	

### 3.11 EIMultiMediaReference

Profile based on the Resource DocumentReference:  
<http://hl7.org/fhir/StructureDefinition/DocumentReference>

Resource Elements	support	Card	StructureDefinition
DocumentReference		0..*	
DocumentReference.id	√	0..1	http://hl7.org/fhir/StructureDefinition/id
DocumentReference.meta	√	0..1	http://hl7.org/fhir/StructureDefinition/Meta

Resource Elements	support	Card	StructureDefinition
DocumentReference.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
DocumentReference.masterIdentifier	√	0..*	
DocumentReference.masterIdentifier.value	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DocumentReference.status	√	0..*	
DocumentReference.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
DocumentReference.type.coding.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
DocumentReference.type.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
DocumentReference.category	√	0..*	
DocumentReference.category.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
DocumentReference.category.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
DocumentReference.category.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
DocumentReference.category.coding.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DocumentReference.subject	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
DocumentReference.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DocumentReference.date	√	0..*	
DocumentReference.author	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Practitioner">http://hl7.org/fhir/StructureDefinition/Practitioner</a> <a href="http://hl7.org/fhir/StructureDefinition/PractitionerRole">http://hl7.org/fhir/StructureDefinition/PractitionerRole</a> <a href="http://hl7.org/fhir/StructureDefinition/Organization">http://hl7.org/fhir/StructureDefinition/Organization</a> <a href="http://hl7.org/fhir/StructureDefinition/Device">http://hl7.org/fhir/StructureDefinition/Device</a> <a href="http://hl7.org/fhir/StructureDefinition/Patient">http://hl7.org/fhir/StructureDefinition/Patient</a> <a href="http://hl7.org/fhir/StructureDefinition/RelatedPerson">http://hl7.org/fhir/StructureDefinition/RelatedPerson</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
DocumentReference.author.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DocumentReference.content	√	0..*	
DocumentReference.content.attachment	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Attachment">http://hl7.org/fhir/StructureDefinition/Attachment</a>
DocumentReference.content.attachment.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>



Resource Elements	support	Card	StructureDefinition
DocumentReference.content.attachment.extension:contentExtension		0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIMultiMediaReferenceContentExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIMultiMediaReferenceContentExtension</a>
DocumentReference.content.attachment.contentType	√	0..*	
DocumentReference.content.attachment.url	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
DocumentReference.content.attachment.size	√	0..*	
DocumentReference.content.attachment.title	√	0..*	
DocumentReference.content.attachment.creation	√	0..*	
DocumentReference.context.related	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Resource">http://hl7.org/fhir/StructureDefinition/Resource</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure</a>
DocumentReference.context.related.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.12 EIPatient

Profile based on the Resource Patient: <http://hl7.org/fhir/StructureDefinition/Patient>

Resource Elements	support	Card	StructureDefinition
Patient.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Patient.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Patient.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Patient.extension:CurrentTransfer	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITransferLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITransferLinkExtension</a>
Patient.extension:CurrentTransfer.value[x]	√	1..1	
Patient.extension:CurrentTransfer.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Patient.extension:CurrentTransfer.value[x].type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.extension:Attachment	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension</a>
Patient.extension:Attachment.value[x]	√	1..1	
Patient.extension:Attachment.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Patient.extension:Attachment.value[x].type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.extension:AdditionalDetails	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientExtension</a>
Patient.extension:AdditionalDetails.extension:VIP	√	0..*	
Patient.extension:AdditionalDetails.extension:VIP.value[x]	√	1..1	
Patient.extension:AdditionalDetails.extension:SocialSecurityNumber	√	0..*	

Resource Elements	support	Card	StructureDefinition
Patient.extension:AdditionalDetails.extension:SocialSecurityNumber.value[x]	√	1..1	
Patient.identifier	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Patient.identifier:ActiveIdentifiers	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Patient.identifier:ActiveIdentifiers.id		0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Patient.identifier:ActiveIdentifiers.use		1..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Patient.identifier:ActiveIdentifiers.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.identifier:ActiveIdentifiers.type.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Patient.identifier:ActiveIdentifiers.type.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.identifier:ActiveIdentifiers.type.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Patient.identifier:ActiveIdentifiers.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.identifier:ActiveIdentifiers.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Patient.identifier:ActiveIdentifiers.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Patient.identifier:InactiveIdentifiers	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Patient.identifier:InactiveIdentifiers.id		0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Patient.identifier:InactiveIdentifiers.use		1..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Patient.identifier:InactiveIdentifiers.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.identifier:InactiveIdentifiers.type.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Patient.identifier:InactiveIdentifiers.type.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.identifier:InactiveIdentifiers.type.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Patient.identifier:InactiveIdentifiers.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Patient.identifier:InactiveIdentifiers.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Patient.identifier:InactiveIdentifiers.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Patient.active	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/boolean">http://hl7.org/fhir/StructureDefinition/boolean</a>
Patient.name	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName">http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName</a>
Patient.name:OfficialName	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName">http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName</a>

Resource Elements	support	Card	StructureDefinition
Patient.name:OfficialName.use		1..1	http://hl7.org/fhir/StructureDefinition/code
Patient.name:MaidenName	√	0..1	http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName
Patient.name:MaidenName.use		1..1	http://hl7.org/fhir/StructureDefinition/code
Patient.telecom	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIContactPoint
Patient.gender	√	0..1	http://hl7.org/fhir/StructureDefinition/code
Patient.birthDate	√	0..1	http://hl7.org/fhir/StructureDefinition/date
Patient.deceased[x]	√	0..1	
Patient.address	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIAddress
Patient.contact		0..*	http://hl7.org/fhir/StructureDefinition/BackboneElement
Patient.generalPractitioner	√	0..*	http://hl7.org/fhir/StructureDefinition/Organization http://hl7.org/fhir/StructureDefinition/PractitionerRole http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional
Patient.generalPractitioner:PrimaryPhysician	√	0..1	http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional
Patient.generalPractitioner:PrimaryPhysician.reference	√	1..1	http://hl7.org/fhir/StructureDefinition/string
Patient.generalPractitioner:PrimaryPhysician.type	√	0..1	http://hl7.org/fhir/StructureDefinition/uri
Patient.generalPractitioner:PrimaryPhysician.display	√	0..1	http://hl7.org/fhir/StructureDefinition/string
Patient.link	√	0..*	http://hl7.org/fhir/StructureDefinition/BackboneElement
Patient.link.other	√	0..*	http://hl7.org/fhir/StructureDefinition/RelatedPerson http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient
Patient.link.other.reference	√	1..1	http://hl7.org/fhir/StructureDefinition/string
Patient.link.type	√	0..1	http://hl7.org/fhir/StructureDefinition/uri

### 3.13 EIPatientAdmission

Profile based on the Resource Encounter: <http://hl7.org/fhir/StructureDefinition/Encounter>

Resource Elements	support	Card	StructureDefinition
Encounter		0..*	
Encounter.id	√	0..1	http://hl7.org/fhir/StructureDefinition/id

Resource Elements	support	Card	StructureDefinition
Encounter.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Encounter.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Encounter.identifier	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Encounter.identifier:AdmissionNumber	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Encounter.identifier:AdmissionNumber.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.identifier:AdmissionNumber.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.status	√	0..*	
Encounter.statusHistory		0..*	
Encounter.class		0..*	
Encounter.class.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.class.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Encounter.classHistory		0..*	
Encounter.serviceType	√	0..*	
Encounter.serviceType.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Encounter.serviceType.extension:Code	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceTypeCodeExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceTypeCodeExtension</a>
Encounter.serviceType.extension:Code.value[x]	√	1..1	
Encounter.serviceType.text	√	0..*	
Encounter.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
Encounter.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.participant		0..*	
Encounter.participant:AdmittingPhysician	√	0..*	
Encounter.participant:AdmittingPhysician.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.participant:AdmittingPhysician.type.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Encounter.participant:AdmittingPhysician.type.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.participant:AdmittingPhysician.type.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Encounter.participant:AdmittingPhysician.individual	√	0..*	

Resource Elements	support	Card	StructureDefinition
Encounter.participant:AdmittingPhysician.individual.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.participant:ReferringPhysician	√	0..*	
Encounter.participant:ReferringPhysician.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.participant:ReferringPhysician.type.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Encounter.participant:ReferringPhysician.type.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.participant:ReferringPhysician.type.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Encounter.participant:ReferringPhysician.individual	√	0..*	
Encounter.participant:ReferringPhysician.individual.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.period	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Period">http://hl7.org/fhir/StructureDefinition/Period</a>
Encounter.period.start	√	0..*	
Encounter.period.end	√	0..*	
Encounter.diagnosis		0..*	

### 3.14 EIPatientAllergy

Profile based on the Resource AllergyIntolerance:

<http://hl7.org/fhir/StructureDefinition/AllergyIntolerance>

Resource Elements	support	Card	StructureDefinition
AllergyIntolerance.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
AllergyIntolerance.clinicalStatus		0..*	
AllergyIntolerance.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
AllergyIntolerance.code.text	√	0..*	
AllergyIntolerance.patient	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
AllergyIntolerance.patient.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
AllergyIntolerance.onset[x]	√	0..1	
AllergyIntolerance.recordedDate	√	0..*	
AllergyIntolerance.recorder	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/PractitionerRole">http://hl7.org/fhir/StructureDefinition/PractitionerRole</a> <a href="http://hl7.org/fhir/StructureDefinition/Patient">http://hl7.org/fhir/StructureDefinition/Patient</a> <a href="http://hl7.org/fhir/StructureDefinition/RelatedPerson">http://hl7.org/fhir/StructureDefinition/RelatedPerson</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>

Resource Elements	support	Card	StructureDefinition
AllergyIntolerance.recorder.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
AllergyIntolerance.note	√	0..*	
AllergyIntolerance.note.text	√	0..*	
AllergyIntolerance.reaction	√	0..*	
AllergyIntolerance.reaction.manifestation	√	0..*	
AllergyIntolerance.reaction.manifestation.text	√	0..*	
AllergyIntolerance.reaction.description	√	0..*	
AllergyIntolerance.reaction.severity	√	0..*	

### 3.15 EIPatientPregnancy

Profile based on the Resource Observation: <http://hl7.org/fhir/StructureDefinition/Observation>

Resource Elements	support	Card	StructureDefinition
Observation		0..*	
Observation.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Observation.extension:ADTBased	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIADTBasedExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIADTBasedExtension</a>
Observation.extension:ADTBased.value[x]	√	1..1	
Observation.status		0..*	
Observation.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.code.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Observation.code.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Observation.code.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.code.coding.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://hl7.org/fhir/StructureDefinition/Device">http://hl7.org/fhir/StructureDefinition/Device</a> <a href="http://hl7.org/fhir/StructureDefinition/Location">http://hl7.org/fhir/StructureDefinition/Location</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
Observation.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.effective[x]	√	0..1	

Resource Elements	support	Card	StructureDefinition
Observation.issued	√	0..*	
Observation.value[x]	√	1..1	
Observation.value[x].coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Observation.value[x].coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Observation.value[x].coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.value[x].coding.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.16 EIPatientSafetyHazard

Profile based on the Resource Condition: <http://hl7.org/fhir/StructureDefinition/Condition>

Resource Elements	support	Card	StructureDefinition
Condition.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Condition.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Condition.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Condition.extension:ModificationExtension	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIModifiedByExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIModifiedByExtension</a>
Condition.extension:ModificationExtension.extension:ModificationDate	√	0..*	
Condition.extension:ModificationExtension.extension:ModificationDate.value[x]	√	1..1	
Condition.extension:ModificationExtension.extension:ModifiedBy	√	0..*	
Condition.extension:ModificationExtension.extension:ModifiedBy.value[x]	√	1..1	
Condition.extension:ModificationExtension.extension:ModifiedBy.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Condition.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Condition.code.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Condition.code.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Condition.code.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Condition.code.coding.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Condition.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>

Resource Elements	support	Card	StructureDefinition
Condition.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Condition.recordedDate	√	0..*	
Condition.recorder	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/PractitionerRole">http://hl7.org/fhir/StructureDefinition/PractitionerRole</a> <a href="http://hl7.org/fhir/StructureDefinition/Patient">http://hl7.org/fhir/StructureDefinition/Patient</a> <a href="http://hl7.org/fhir/StructureDefinition/RelatedPerson">http://hl7.org/fhir/StructureDefinition/RelatedPerson</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
Condition.recorder.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Condition.note	√	0..*	
Condition.note.text	√	0..*	

### 3.17 EIPerformedProcedureStep

Profile based on the Resource Procedure: <http://hl7.org/fhir/StructureDefinition/Procedure>

Resource Elements	support	Card	StructureDefinition
Procedure		0..*	
Procedure.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Procedure.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Procedure.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Procedure.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Procedure.extension:Attachment	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension</a>
Procedure.extension:Attachment.value[x]	√	1..1	
Procedure.extension:Attachment.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.identifier	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Procedure.identifier:ProcedureStepID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Procedure.identifier:ProcedureStepID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.identifier:PerformedProcedureStepSOPUIID	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Procedure.identifier:PerformedProcedureStepSOPUIID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.basedOn		0..*	<a href="http://hl7.org/fhir/StructureDefinition/CarPlan">http://hl7.org/fhir/StructureDefinition/CarPlan</a>



Resource Elements	support	Card	StructureDefinition
			<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIScheduledProcedureStep">http://www.agfa.com/ei/fhir/StructureDefinition/EIScheduledProcedureStep</a>
Procedure.basedOn.reference		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.partOf	√	1..*	<a href="http://hl7.org/fhir/StructureDefinition/Observation">http://hl7.org/fhir/StructureDefinition/Observation</a> <a href="http://hl7.org/fhir/StructureDefinition/MedicationAdministration">http://hl7.org/fhir/StructureDefinition/MedicationAdministration</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure</a>
Procedure.partOf.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.status	√	0..*	
Procedure.category	√	0..*	
Procedure.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
Procedure.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.performed[x]	√	0..1	
Procedure.performed[x].start	√	0..*	
Procedure.performed[x].end	√	0..*	

### 3.18 EIPersonName

Profile based on the Resource HumanName:  
<http://hl7.org/fhir/StructureDefinition/HumanName>

Resource Elements	support	Card	StructureDefinition
HumanName		0..*	
HumanName.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
HumanName.use	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
HumanName.text	√	0..*	
HumanName.family	√	0..*	
HumanName.given	√	0..*	
HumanName.prefix	√	0..*	
HumanName.suffix	√	0..*	

### 3.19 EIProfessional

Profile based on the Resource Practitioner: <http://hl7.org/fhir/StructureDefinition/Practitioner>

Resource Elements	support	Card	StructureDefinition
Practitioner.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Practitioner.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Practitioner.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Practitioner.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Practitioner.extension:SecUserExtension	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EISecUserExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EISecUserExtension</a>
Practitioner.extension:SecUserExtension.extension:UserAccount	√	0..*	
Practitioner.extension:SecUserExtension.extension:UserAccount.extension:LoginName	√	0..*	
Practitioner.extension:SecUserExtension.extension:UserAccount.extension:LoginName.value[x]	√	1..1	
Practitioner.extension:SecUserExtension.extension:UserAccount.extension:Domain	√	0..*	
Practitioner.extension:SecUserExtension.extension:UserAccount.extension:Domain.value[x]	√	1..1	
Practitioner.extension:SecUserExtension.extension:Guid	√	0..*	
Practitioner.extension:SecUserExtension.extension:Guid.value[x]	√	1..1	
Practitioner.identifier	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Identifier">http://hl7.org/fhir/StructureDefinition/Identifier</a>
Practitioner.identifier:IdentityCode	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Practitioner.identifier:IdentityCode.type.coding		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Practitioner.identifier:IdentityCode.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Practitioner.identifier:IdentityCode.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Practitioner.active	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/boolean">http://hl7.org/fhir/StructureDefinition/boolean</a>
Practitioner.name	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName">http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName</a>
Practitioner.name:OfficialName	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName">http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName</a>
Practitioner.name:OfficialName.use		1..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Practitioner.name:MaidenName	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName">http://www.agfa.com/ei/fhir/StructureDefinition/EIPersonName</a>
Practitioner.name:MaidenName.use		1..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>

Resource Elements	support	Card	StructureDefinition
Practitioner.telecom	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIContactPoint">http://www.agfa.com/ei/fhir/StructureDefinition/EIContactPoint</a>
Practitioner.address	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAddress">http://www.agfa.com/ei/fhir/StructureDefinition/EIAddress</a>
Practitioner.gender	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Practitioner.birthDate	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/date">http://hl7.org/fhir/StructureDefinition/date</a>

### 3.20 EIQueryList

Profile based on the Resource List: <http://hl7.org/fhir/StructureDefinition/List>

Resource Elements	support	Card	StructureDefinition
List.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
List.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
List.extension:counters	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EICountersExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EICountersExtension</a>
List.extension:dataRetrievalStatus	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIDataRetrievalStatusExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIDataRetrievalStatusExtension</a>
List.extension:dataRetrievalStatus.value[x]	√	1..1	
List.identifier	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Identifier">http://hl7.org/fhir/StructureDefinition/Identifier</a>
List.identifier:QueryListCode		0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
List.status	√	0..*	
List.title	√	0..*	
List.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
List.code.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
List.code.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
List.code.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
List.entry	√	0..*	
List.entry.item	√	0..*	
List.entry.item.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.21 EIReportBlob

Profile based on the Resource Binary: <http://hl7.org/fhir/StructureDefinition/Binary>

Resource Elements	support	Card	StructureDefinition
Binary		0..*	

Resource Elements	support	Card	StructureDefinition
Binary.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Binary.contentType	√	0..*	
Binary.data	√	0..*	

### 3.22 EIReportMeasurements

Profile based on the Resource Observation: <http://hl7.org/fhir/StructureDefinition/Observation>

Resource Elements	support	Card	StructureDefinition
Observation		0..*	
Observation.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Observation.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.identifier	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Observation.identifier:StudyUID	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Observation.identifier:StudyUID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.identifier:ContainerUid	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Observation.identifier:ContainerUid.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.status	√	0..*	
Observation.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.code.text		0..*	
Observation.component	√	0..*	
Observation.component.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.component.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.component.code.text	√	0..*	
Observation.component.value[x]	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIMeasurementQuantity">http://www.agfa.com/ei/fhir/StructureDefinition/EIMeasurementQuantity</a>

### 3.23 EIReportResultAsDiagnosticReport

Profile based on the Resource DiagnosticReport:  
<http://hl7.org/fhir/StructureDefinition/DiagnosticReport>

Resource Elements	support	Card	StructureDefinition
DiagnosticReport		0..*	

Resource Elements	support	Card	StructureDefinition
DiagnosticReport.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
DiagnosticReport.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
DiagnosticReport.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
DiagnosticReport.basedOn	√	1..*	<a href="http://hl7.org/fhir/StructureDefinition/CarePlan">http://hl7.org/fhir/StructureDefinition/CarePlan</a> <a href="http://hl7.org/fhir/StructureDefinition/ImmunizationRecommendation">http://hl7.org/fhir/StructureDefinition/ImmunizationRecommendation</a> <a href="http://hl7.org/fhir/StructureDefinition/MedicationRequest">http://hl7.org/fhir/StructureDefinition/MedicationRequest</a> <a href="http://hl7.org/fhir/StructureDefinition/NutritionOrder">http://hl7.org/fhir/StructureDefinition/NutritionOrder</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest</a>
DiagnosticReport.basedOn.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DiagnosticReport.status	√	0..*	
DiagnosticReport.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
DiagnosticReport.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://hl7.org/fhir/StructureDefinition/Device">http://hl7.org/fhir/StructureDefinition/Device</a> <a href="http://hl7.org/fhir/StructureDefinition/Location">http://hl7.org/fhir/StructureDefinition/Location</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
DiagnosticReport.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DiagnosticReport.effective[x]	√	0..1	
DiagnosticReport.effective[x].start	√	0..*	
DiagnosticReport.effective[x].end	√	0..*	
DiagnosticReport.issued	√	0..*	
DiagnosticReport.result	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultAsObservation">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultAsObservation</a>
DiagnosticReport.result.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
DiagnosticReport.imagingStudy	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIStudy">http://www.agfa.com/ei/fhir/StructureDefinition/EIStudy</a>
DiagnosticReport.imagingStudy.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.24 EIReportResultAsObservation

Profile based on the Resource Observation: <http://hl7.org/fhir/StructureDefinition/Observation>

Resource Elements	support	Card	StructureDefinition
Observation		0..*	
Observation.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Observation.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Observation.extension:Statistics	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultStatisticsExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultStatisticsExtension</a>
Observation.extension:Statistics.extension:WordCount	√	0..*	
Observation.extension:Statistics.extension:WordCount.value[x]	√	1..1	
Observation.extension:Statistics.extension:WordCountTyped	√	0..*	
Observation.extension:Statistics.extension:WordCountTyped.value[x]	√	1..1	
Observation.extension:Statistics.extension:LetterCount	√	0..*	
Observation.extension:Statistics.extension:LetterCount.value[x]	√	1..1	
Observation.extension:Statistics.extension:LetterCountTyped	√	0..*	
Observation.extension:Statistics.extension:LetterCountTyped.value[x]	√	1..1	
Observation.extension:Statistics.extension:LetterCountNoWhitespace	√	0..*	
Observation.extension:Statistics.extension:LetterCountNoWhitespace.value[x]	√	1..1	
Observation.extension:Statistics.extension:LetterCountTypedNoWhitespace	√	0..*	
Observation.extension:Statistics.extension:LetterCountTypedNoWhitespace.value[x]	√	1..1	
Observation.extension:MainReport	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/observation-sequelTo">http://hl7.org/fhir/StructureDefinition/observation-sequelTo</a>
Observation.extension:MainReport.value[x]	√	1..1	
Observation.extension:MainReport.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.extension:ReportResult	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultExtension</a>
Observation.extension:ReportResult.extension:AwaitingContent	√	0..*	
Observation.extension:ReportResult.extension:AwaitingContent.value[x]	√	1..1	

Resource Elements	support	Card	StructureDefinition
Observation.extension:ReportResult.extension:MsdPrinted	√	0..*	
Observation.extension:ReportResult.extension:MsdPrinted.value[x]	√	1..1	
Observation.extension:ReportResult.extension:External	√	0..*	
Observation.extension:ReportResult.extension:External.value[x]	√	1..1	
Observation.extension:ReportResult.extension:Intermediate	√	0..*	
Observation.extension:ReportResult.extension:Intermediate.value[x]	√	1..1	
Observation.extension:ReportResult.extension:InConflictReason	√	0..*	
Observation.extension:ReportResult.extension:InConflictReason.value[x]	√	1..1	
Observation.extension:ReportResult.extension:ReportBlob	√	0..*	
Observation.extension:ReportResult.extension:ReportBlob.value[x]	√	1..1	
Observation.extension:ReportResult.extension:ReportBlob.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.extension:ReportSeverity	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSeverityExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSeverityExtension</a>
Observation.extension:ReportSeverity.extension:Code	√	0..*	
Observation.extension:ReportSeverity.extension:Code.value[x]	√	1..1	
Observation.extension:ReportSeverity.extension:UpdatedOn	√	0..*	
Observation.extension:ReportSeverity.extension:UpdatedOn.value[x]	√	1..1	
Observation.extension:ReportSeverity.extension:UpdatedBy	√	0..*	
Observation.extension:ReportSeverity.extension:UpdatedBy.value[x]	√	1..1	
Observation.extension:ReportSeverity.extension:UpdatedBy.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.extension:DigitalDictation	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIDigitalDictationExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIDigitalDictationExtension</a>
Observation.extension:DigitalDictation.extension:SoundFile	√	0..*	
Observation.extension:DigitalDictation.extension:SoundFile.value[x]	√	1..1	
Observation.extension:DigitalDictation.extension:SoundFile.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

Resource Elements	support	Card	StructureDefinition
Observation.extension:DigitalDictation.extension:Format	√	0..*	
Observation.extension:DigitalDictation.extension:Format.value[x]	√	1..1	
Observation.extension:DigitalDictation.extension:Purged	√	0..*	
Observation.extension:DigitalDictation.extension:Purged.value[x]	√	1..1	
Observation.extension:DigitalDictation.extension:Duration	√	0..*	
Observation.extension:DigitalDictation.extension:Duration.value[x]	√	1..1	
Observation.extension:OnlineRecognition	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIOnlineRecognitionExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIOnlineRecognitionExtension</a>
Observation.extension:OnlineRecognition.extension:WavFile	√	0..*	
Observation.extension:OnlineRecognition.extension:WavFile.value[x]	√	1..1	
Observation.extension:OnlineRecognition.extension:WavFile.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.extension:OnlineRecognition.extension:SrrFile	√	0..*	
Observation.extension:OnlineRecognition.extension:SrrFile.value[x]	√	1..1	
Observation.extension:OnlineRecognition.extension:SrrFile.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.extension:OnlineRecognition.extension:SmdFile	√	0..*	
Observation.extension:OnlineRecognition.extension:SmdFile.value[x]	√	1..1	
Observation.extension:OnlineRecognition.extension:SmdFile.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.extension:OnlineRecognition.extension:Purged	√	0..*	
Observation.extension:OnlineRecognition.extension:Purged.value[x]	√	1..1	
Observation.extension:OnlineRecognition.extension:ParagraphWhitespace	√	0..*	
Observation.extension:OnlineRecognition.extension:ParagraphWhitespace.value[x]	√	1..1	
Observation.extension:Attachment	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension</a>
Observation.extension:Attachment.value[x]	√	1..1	
Observation.extension:Attachment.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>



Resource Elements	support	Card	StructureDefinition
Observation.identifier	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Observation.identifier:ReportUID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Observation.identifier:ReportUID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.identifier:ReportCode	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Observation.identifier:ReportCode.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.status	√	0..*	
Observation.status.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Observation.status.extension:ReportResultStatus	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultStatusExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultStatusExtension</a>
Observation.status.extension:ReportResultStatus.value[x]	√	1..1	
Observation.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://hl7.org/fhir/StructureDefinition/Device">http://hl7.org/fhir/StructureDefinition/Device</a> <a href="http://hl7.org/fhir/StructureDefinition/Location">http://hl7.org/fhir/StructureDefinition/Location</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
Observation.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.issued	√	0..*	
Observation.performer	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/PractitionerRole">http://hl7.org/fhir/StructureDefinition/PractitionerRole</a> <a href="http://hl7.org/fhir/StructureDefinition/Organization">http://hl7.org/fhir/StructureDefinition/Organization</a> <a href="http://hl7.org/fhir/StructureDefinition/CareTeam">http://hl7.org/fhir/StructureDefinition/CareTeam</a> <a href="http://hl7.org/fhir/StructureDefinition/Patient">http://hl7.org/fhir/StructureDefinition/Patient</a> <a href="http://hl7.org/fhir/StructureDefinition/RelatedPerson">http://hl7.org/fhir/StructureDefinition/RelatedPerson</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
Observation.performer.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Observation.performer.extension:ParticipationType	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIParticipationTypeExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIParticipationTypeExtension</a>
Observation.performer.extension:ParticipationType.extension:Code	√	0..*	

Resource Elements	support	Card	StructureDefinition
Observation.performer.extension:ParticipationType.extension:Code.value[x]	√	1..1	
Observation.performer.extension:ParticipationType.extension:AddedOn	√	0..*	
Observation.performer.extension:ParticipationType.extension:AddedOn.value[x]	√	1..1	
Observation.performer.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.hasMember	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/QuestionnaireResponse">http://hl7.org/fhir/StructureDefinition/QuestionnaireResponse</a> <a href="http://hl7.org/fhir/StructureDefinition/MolecularSequence">http://hl7.org/fhir/StructureDefinition/MolecularSequence</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSection">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSection</a>
Observation.hasMember.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.25 EIReportSection

Profile based on the Resource Observation: <http://hl7.org/fhir/StructureDefinition/Observation>

Resource Elements	support	Card	StructureDefinition
Observation		0..*	
Observation.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Observation.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Observation.extension:TextSection	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITextSectionExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITextSectionExtension</a>
Observation.extension:TextSection.extension:TextFormat	√	0..*	
Observation.extension:TextSection.extension:TextFormat.value[x]	√	1..1	
Observation.extension:TextSection.extension:Online	√	0..*	
Observation.extension:TextSection.extension:Online.value[x]	√	1..1	
Observation.extension:ReportSection	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSectionExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSectionExtension</a>
Observation.extension:ReportSection.extension:SpeechJobUID	√	0..*	
Observation.extension:ReportSection.extension:SpeechJobUID.value[x]	√	1..1	
Observation.extension:ReportSection.extension:Mandatory	√	0..*	

Resource Elements	support	Card	StructureDefinition
Observation.extension:ReportSection.extension:Mandatory.value[x]	√	1..1	
Observation.extension:ReportSection.extension:PriorReport	√	0..*	
Observation.extension:ReportSection.extension:PriorReport.value[x]	√	1..1	
Observation.extension:ReportSection.extension:RequestedProceduresCovered	√	0..*	
Observation.extension:ReportSection.extension:RequestedProceduresCovered.value[x]	√	1..1	
Observation.extension:ReportSection.extension:RequestedProceduresCovered.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.extension:ReportSection.extension:ReportSectionDefinitionCode	√	0..*	
Observation.extension:ReportSection.extension:ReportSectionDefinitionCode.value[x]	√	1..1	
Observation.status	√	0..*	
Observation.category	√	0..*	
Observation.category.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Observation.category.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.code.text		0..*	
Observation.value[x]	√	1..1	
Observation.hasMember	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/QuestionnaireResponse">http://hl7.org/fhir/StructureDefinition/QuestionnaireResponse</a> <a href="http://hl7.org/fhir/StructureDefinition/MolecularSequence">http://hl7.org/fhir/StructureDefinition/MolecularSequence</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportMeasurements">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportMeasurements</a>
Observation.hasMember.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.26 EIReportSoundFile

Profile based on the Resource Binary: <http://hl7.org/fhir/StructureDefinition/Binary>

Resource Elements	support	Card	StructureDefinition
Binary.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Binary.contentType	√	0..*	
Binary.data	√	0..*	

### 3.27 EIRequestedProcedureAsProcedure

Profile based on the Resource Procedure: <http://hl7.org/fhir/StructureDefinition/Procedure>

Resource Elements	support	Card	StructureDefinition
Procedure.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Procedure.meta		0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Procedure.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Procedure.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Procedure.extension:RequestedProcedureExtension	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedureExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedureExtension</a>
Procedure.extension:RequestedProcedureExtension.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Procedure.extension:RequestedProcedureExtension.extension:Unverified	√	0..*	
Procedure.extension:RequestedProcedureExtension.extension:Unverified.value[x]	√	1..1	
Procedure.extension:RequestedProcedureExtension.extension:DicomBased	√	0..*	
Procedure.extension:RequestedProcedureExtension.extension:DicomBased.value[x]	√	1..1	
Procedure.extension:RequestedProcedureExtension.extension:GuestStudy	√	0..*	
Procedure.extension:RequestedProcedureExtension.extension:GuestStudy.value[x]	√	1..1	
Procedure.extension:RequestedProcedureExtension.extension:InitialStoreDone	√	0..*	
Procedure.extension:RequestedProcedureExtension.extension:InitialStoreDone.value[x]	√	1..1	
Procedure.extension:RequestedProcedureExtension.extension:OriginalStudyUID	√	0..*	
Procedure.extension:RequestedProcedureExtension.extension:OriginalStudyUID.value[x]	√	1..1	
Procedure.extension:RequestedProcedureExtension.extension:ModalityType	√	0..*	
Procedure.extension:RequestedProcedureExtension.extension:ModalityType.value[x]	√	1..1	
Procedure.extension:RequestedProcedureExtension.extension:ModalityType.value[x].system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Procedure.extension:RequestedProcedureExtension.extension:ModalityType.value[x].code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Procedure.extension:RequestedProcedureExtension.extension:ModalityType.value[x].display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.extension:TransferAtAcquisition	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITransferLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITransferLinkExtension</a>

Resource Elements	support	Card	StructureDefinition
Procedure.extension:TransferAtAcquisition.value[x]	√	1..1	
Procedure.extension:TransferAtAcquisition.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.extension:TransferAtAcquisition.value[x].type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Procedure.extension:Attachment	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension</a>
Procedure.extension:Attachment.value[x]	√	1..1	
Procedure.extension:Pregnancy	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPregnancyLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIPregnancyLinkExtension</a>
Procedure.extension:Pregnancy.value[x]	√	1..1	
Procedure.identifier	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Procedure.identifier:RequestedProcedureID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Procedure.identifier:RequestedProcedureID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.identifier:DICOMStudyIUID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Procedure.identifier:DICOMStudyIUID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.identifier:ProcedureLevelAccessionNumber	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Procedure.identifier:ProcedureLevelAccessionNumber.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.basedOn	√	1..*	<a href="http://hl7.org/fhir/StructureDefinition/CarePlan">http://hl7.org/fhir/StructureDefinition/CarePlan</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest</a>
Procedure.status		0..*	
Procedure.category	√	0..*	
Procedure.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
Procedure.performed[x]	√	0..1	
Procedure.performer	√	0..*	
Procedure.performer.function	√	0..*	

Resource Elements	support	Card	StructureDefinition
Procedure.performer.actor	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/PractitionerRole">http://hl7.org/fhir/StructureDefinition/PractitionerRole</a> <a href="http://hl7.org/fhir/StructureDefinition/Patient">http://hl7.org/fhir/StructureDefinition/Patient</a> <a href="http://hl7.org/fhir/StructureDefinition/RelatedPerson">http://hl7.org/fhir/StructureDefinition/RelatedPerson</a> <a href="http://hl7.org/fhir/StructureDefinition/Device">http://hl7.org/fhir/StructureDefinition/Device</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment">http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment</a>
Procedure.bodySite	√	0..*	
Procedure.bodySite.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Procedure.bodySite.extension:PrimaryBodyPartExt	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPrimaryBodyPartExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIPrimaryBodyPartExtension</a>
Procedure.bodySite.extension:PrimaryBodyPartExt.value[x]	√	1..1	
Procedure.bodySite.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Procedure.bodySite.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Procedure.bodySite.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Procedure.bodySite.coding.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Procedure.report	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/DocumentReference">http://hl7.org/fhir/StructureDefinition/DocumentReference</a> <a href="http://hl7.org/fhir/StructureDefinition/Composition">http://hl7.org/fhir/StructureDefinition/Composition</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultAsDiagnosticReport">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportResultAsDiagnosticReport</a>

### 3.28 EIRequestedProcedureAsServiceRequest

Profile based on the Resource ServiceRequest:

<http://hl7.org/fhir/StructureDefinition/ServiceRequest>

Resource Elements	support	Card	StructureDefinition
ServiceRequest.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ServiceRequest.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ServiceRequest.meta.source.extension	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>

Resource Elements	support	Card	StructureDefinition
ServiceRequest.meta.source.extension:AE	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EISourceExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EISourceExtension</a>
ServiceRequest.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ServiceRequest.extension:OriginalPatient	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIOriginalPatientExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIOriginalPatientExtension</a>
ServiceRequest.extension:Pregnancy	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPregnancyLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIPregnancyLinkExtension</a>
ServiceRequest.extension:TaskInfo	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInfoExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInfoExtension</a>
ServiceRequest.identifier	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:DICOMStudyUID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:DICOMStudyUID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:ProcedureLevelAccessionNumber	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:ProcedureLevelAccessionNumber.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:RequestedProcedureID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:RequestedProcedureID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:ProcedurePlanCode	√	0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:ProcedurePlanCode.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.basedOn	√	1..*	<a href="http://hl7.org/fhir/StructureDefinition/CarePlan">http://hl7.org/fhir/StructureDefinition/CarePlan</a> <a href="http://hl7.org/fhir/StructureDefinition/MedicationRequest">http://hl7.org/fhir/StructureDefinition/MedicationRequest</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequest">http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequest</a>
ServiceRequest.status		0..*	
ServiceRequest.intent		0..*	
ServiceRequest.category	√	0..*	
ServiceRequest.priority	√	0..*	
ServiceRequest.priority.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ServiceRequest.priority.extension:EIPriority	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedurePriorityExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedurePriorityExtension</a>
ServiceRequest.priority.extension:EIPriority.value[x]	√	1..1	

Resource Elements	support	Card	StructureDefinition
ServiceRequest.code	√	0..1	http://hl7.org/fhir/StructureDefinition/code
ServiceRequest.code.coding.system	√	0..1	http://hl7.org/fhir/StructureDefinition/uri
ServiceRequest.code.coding.code	√	0..1	http://hl7.org/fhir/StructureDefinition/code
ServiceRequest.code.text	√	0..*	
ServiceRequest.subject	√	0..*	http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient
ServiceRequest.encounter	√	0..*	
ServiceRequest.occurrence[x]	√	0..1	
ServiceRequest.performer	√	0..*	http://hl7.org/fhir/StructureDefinition/PractitionerRole http://hl7.org/fhir/StructureDefinition/CareTeam http://hl7.org/fhir/StructureDefinition/HealthcareService http://hl7.org/fhir/StructureDefinition/Patient http://hl7.org/fhir/StructureDefinition/Device http://hl7.org/fhir/StructureDefinition/RelatedPerson http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment
ServiceRequest.bodySite		0..*	ServiceRequest.bodySite

### 3.29 EIRoom

Profile based on the Resource Location: <http://hl7.org/fhir/StructureDefinition/Location>

Resource Elements	support	Card	StructureDefinition
Location		0..*	
Location.id	√	0..1	http://hl7.org/fhir/StructureDefinition/id
Location.meta	√	0..1	http://hl7.org/fhir/StructureDefinition/Meta
Location.meta.versionId	√	0..1	http://hl7.org/fhir/StructureDefinition/id
Location.status	√	0..*	
Location.name	√	0..*	

### 3.30 EIScheduledProcedureStep

Profile based on the Resource ServiceRequest:  
<http://hl7.org/fhir/StructureDefinition/ServiceRequest>



Resource Elements	support	Card	StructureDefinition
ServiceRequest		0..*	
ServiceRequest.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ServiceRequest.meta		0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
ServiceRequest.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ServiceRequest.identifier	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:ProcedureStepID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:ProcedureStepID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.basedOn	√	1..*	<a href="http://hl7.org/fhir/StructureDefinition/CarPlan">http://hl7.org/fhir/StructureDefinition/CarPlan</a> <a href="http://hl7.org/fhir/StructureDefinition/MedicationRequest">http://hl7.org/fhir/StructureDefinition/MedicationRequest</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest</a>
ServiceRequest.basedOn.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.status	√	0..*	
ServiceRequest.status.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ServiceRequest.status.extension:EIStatus	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIScheduledProcedureStepStatusExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIScheduledProcedureStepStatusExtension</a>
ServiceRequest.status.extension:EIStatus.value[x]	√	1..1	
ServiceRequest.intent		0..*	
ServiceRequest.category	√	0..*	
ServiceRequest.subject	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
ServiceRequest.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.occurrence[x]	√	0..1	
ServiceRequest.occurrence[x].start	√	0..*	
ServiceRequest.occurrence[x].end	√	0..*	
ServiceRequest.locationReference	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRoom">http://www.agfa.com/ei/fhir/StructureDefinition/EIRoom</a>
ServiceRequest.locationReference.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

**3.31****EIServiceRequest**

Profile based on the Resource ServiceRequest:

<http://hl7.org/fhir/StructureDefinition/ServiceRequest>

Resource Elements	support	Card	StructureDefinition
ServiceRequest.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ServiceRequest.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
ServiceRequest.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ServiceRequest.meta.source.extension	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ServiceRequest.meta.source.extension:AE	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EISourceExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EISourceExtension</a>
ServiceRequest.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ServiceRequest.extension:Attachment	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachmentLinkExtension</a>
ServiceRequest.extension:Attachment.value[x]	√	1..1	
ServiceRequest.extension:Attachment.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.extension:Pregnancy	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPregnancyLinkExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIPregnancyLinkExtension</a>
ServiceRequest.extension:Pregnancy.value[x]	√	1..1	
ServiceRequest.extension:Pregnancy.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:FillerOrderNumber		1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:FillerOrderNumber.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:FillerOrderNumber.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:PlacerOrderNumber		0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:PlacerOrderNumber.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:PlacerOrderNumber.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:OrderLevelAccessionNumber		0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ServiceRequest.identifier:OrderLevelAccessionNumber.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.identifier:OrderLevelAccessionNumber.assigner.display		0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.status		0..*	
ServiceRequest.intent		0..*	

Resource Elements	support	Card	StructureDefinition
ServiceRequest.category	√	0..*	
ServiceRequest.subject	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
ServiceRequest.subject.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ServiceRequest.subject.extension:OriginalPatientID	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequestOriginalPatientIDExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequestOriginalPatientIDExtension</a>
ServiceRequest.subject.extension:OriginalPatientID.value[x]	√	1..1	
ServiceRequest.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.authoredOn	√	0..*	
ServiceRequest.requester	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/PractitionerRole">http://hl7.org/fhir/StructureDefinition/PractitionerRole</a> <a href="http://hl7.org/fhir/StructureDefinition/Organization">http://hl7.org/fhir/StructureDefinition/Organization</a> <a href="http://hl7.org/fhir/StructureDefinition/Patient">http://hl7.org/fhir/StructureDefinition/Patient</a> <a href="http://hl7.org/fhir/StructureDefinition/RelatedPerson">http://hl7.org/fhir/StructureDefinition/RelatedPerson</a> <a href="http://hl7.org/fhir/StructureDefinition/Device">http://hl7.org/fhir/StructureDefinition/Device</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
ServiceRequest.requester.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ServiceRequest.requester.extension:RequestingOrganization	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequestRequesterExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIServiceRequestRequesterExtension</a>
ServiceRequest.requester.extension:RequestingOrganization.extension:RequestingDepartment	√	0..*	
ServiceRequest.requester.extension:RequestingOrganization.extension:RequestingDepartment.value[x]	√	1..1	
ServiceRequest.requester.extension:RequestingOrganization.extension:RequestingDepartment.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.requester.extension:RequestingOrganization.extension:RequestingFacility	√	0..*	
ServiceRequest.requester.extension:RequestingOrganization.extension:RequestingFacility.value[x]	√	1..1	
ServiceRequest.requester.extension:RequestingOrganization.extension:RequestingFacility.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ServiceRequest.requester.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.32

### EIStructuredReport

Profile based on the Resource Observation: <http://hl7.org/fhir/StructureDefinition/Observation>

Resource Elements	support	Card	StructureDefinition
Observation		0..*	
Observation.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Observation.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Observation.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Observation.extension:priorStruturedReport	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/observation-sequelTo">http://hl7.org/fhir/StructureDefinition/observation-sequelTo</a>
Observation.extension:priorStruturedReport.value[x]		1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIStructuredReport">http://www.agfa.com/ei/fhir/StructureDefinition/EIStructuredReport</a>
Observation.extension:priorStruturedReport.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.status	√	0..*	
Observation.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.code.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Observation.code.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Observation.code.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Observation.code.coding.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Observation.value[x]	√	1..1	

### 3.33 EIStudy

Profile based on the Resource ImagingStudy:

<http://hl7.org/fhir/StructureDefinition/ImagingStudy>

Resource Elements	support	Card	StructureDefinition
ImagingStudy		0..*	
ImagingStudy.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ImagingStudy.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
ImagingStudy.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
ImagingStudy.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
ImagingStudy.extension:StudyExtension	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIStudyExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIStudyExtension</a>
ImagingStudy.extension:StudyExtension.extension:NumberOfImages	√	0..*	

Resource Elements	support	Card	StructureDefinition
ImagingStudy.extension:StudyExtension.extension:NumberOfImages.value[x]	√	1..1	
ImagingStudy.extension:StudyExtension.extension:Availability	√	0..*	
ImagingStudy.extension:StudyExtension.extension:Availability.value[x]	√	1..1	
ImagingStudy.extension:StudyExtension.extension:Availability.value[x].system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
ImagingStudy.extension:StudyExtension.extension:Availability.value[x].code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
ImagingStudy.extension:VolumeTypeExtension	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIVolumeTypeExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIVolumeTypeExtension</a>
ImagingStudy.extension:VolumeTypeExtension.value[x]	√	1..1	
ImagingStudy.identifier	√	1..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ImagingStudy.identifier:DICOMStudyUID	√	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
ImagingStudy.identifier:DICOMStudyUID.value		1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ImagingStudy.status	√	0..*	
ImagingStudy.subject	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
ImagingStudy.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ImagingStudy.basedOn	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/CarePlan">http://hl7.org/fhir/StructureDefinition/CarePlan</a> <a href="http://hl7.org/fhir/StructureDefinition/Appointment">http://hl7.org/fhir/StructureDefinition/Appointment</a> <a href="http://hl7.org/fhir/StructureDefinition/AppointmentResponse">http://hl7.org/fhir/StructureDefinition/AppointmentResponse</a> <a href="http://hl7.org/fhir/StructureDefinition/Task">http://hl7.org/fhir/StructureDefinition/Task</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest</a>
ImagingStudy.basedOn.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ImagingStudy.numberOfSeries	√	0..*	
ImagingStudy.numberOfInstances	√	0..*	
ImagingStudy.procedureReference	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure</a>
ImagingStudy.procedureReference.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ImagingStudy.description	√	0..*	
ImagingStudy.series	√	0..*	

Resource Elements	support	Card	StructureDefinition
ImagingStudy.series.uid	√	0..*	
ImagingStudy.series.modality	√	0..*	
ImagingStudy.series.modality.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
ImagingStudy.series.modality.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
ImagingStudy.series.modality.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
ImagingStudy.series.performer	√	0..*	
ImagingStudy.series.performer.actor	√	0..*	
ImagingStudy.series.performer.actor.display	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

### 3.34 EITaskInstance

Profile based on the Resource Task: <http://hl7.org/fhir/StructureDefinition/Task>

Resource Elements	support	Card	StructureDefinition
Task.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Task.meta.versionId		0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Task.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Task.extension:Responsibles	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstanceResponsiblesExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstanceResponsiblesExtension</a>
Task.extension:Responsibles.extension:IncludedResponsibles	√	0..*	
Task.extension:Responsibles.extension:IncludedResponsibles.value[x]	√	1..1	
Task.extension:Responsibles.extension:IncludedResponsibles.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Task.extension:Responsibles.extension:ExcludedResponsibles	√	0..*	
Task.extension:Responsibles.extension:ExcludedResponsibles.value[x]	√	1..1	
Task.extension:Responsibles.extension:ExcludedResponsibles.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Task.extension:additionRemovalPossibility		0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstanceAdditionRemovalExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstanceAdditionRemovalExtension</a>
Task.instantiatesCanonical		0..*	
Task.basedOn		0..*	
Task.status	√	0..*	
Task.priority	√	0..*	

Resource Elements	support	Card	StructureDefinition
Task.priority.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Task.priority.extension:eiTaskInstancePriority		0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstancePriorityExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstancePriorityExtension</a>
Task.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Task.code.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Task.code.extension:subtype		0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstanceSubtypeExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITaskInstanceSubtypeExtension</a>
Task.code.coding.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Task.code.coding.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Task.code.text	√	0..*	
Task.focus		0..*	
Task.for	√	0..*	
Task.executionPeriod.start		0..*	
Task.executionPeriod.end		0..*	
Task.authoredOn	√	0..*	
Task.lastModified	√	0..*	
Task.owner	√	0..*	
Task.note.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Task.note.extension:EIAnnotationCommentKey		0..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAnnotationCommentKeyExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIAnnotationCommentKeyExtension</a>
Task.restriction	√	0..*	
Task.restriction.period	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Period">http://hl7.org/fhir/StructureDefinition/Period</a>
Task.restriction.period.start	√	0..*	
Task.input	√	0..*	
Task.input:RequestedProcedures	√	0..*	
Task.input:RequestedProcedures.type		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Task.input:RequestedProcedures.type.coding		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Task.input:RequestedProcedures.type.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Task.input:RequestedProcedures.type.coding.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Task.input:RequestedProcedures.value[x]		1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsServiceRequest</a>

Resource Elements	support	Card	StructureDefinition
Task.input:variables	√	0..*	
Task.input:variables.type		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Task.input:variables.type.coding		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Task.input:variables.type.coding.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Task.input:variables.type.coding.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Task.input:variables.value[x]		1..1	
Task.input:data	√	0..*	
Task.input:data.type		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Task.input:data.type.coding		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Task.input:data.type.coding.system		0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Task.input:data.type.coding.code		0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Task.input:data.value[x]		1..1	

### 3.35 EITransfer

Profile based on the Resource Encounter: <http://hl7.org/fhir/StructureDefinition/Encounter>

Resource Elements	support	Card	StructureDefinition
Encounter		0..*	
Encounter.id	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Encounter.meta	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/Meta">http://hl7.org/fhir/StructureDefinition/Meta</a>
Encounter.meta.versionId	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/id">http://hl7.org/fhir/StructureDefinition/id</a>
Encounter.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Encounter.extension:LocationDetails		0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EILocationDetailsExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EILocationDetailsExtension</a>
Encounter.extension:LocationDetails.extension:Bed	√	0..*	
Encounter.extension:LocationDetails.extension:Bed.value[x]	√	1..1	
Encounter.extension:LocationDetails.extension:Room	√	0..*	
Encounter.extension:LocationDetails.extension:Room.value[x]	√	1..1	
Encounter.extension:LocationDetails.extension:Facility	√	0..*	
Encounter.extension:LocationDetails.extension:Facility.value[x]	√	1..1	



Resource Elements	support	Card	StructureDefinition
Encounter.extension:LocationDetails.extension:Facility.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.extension:LocationDetails.extension:Department	√	0..*	
Encounter.extension:LocationDetails.extension:Department.value[x]	√	1..1	
Encounter.extension:LocationDetails.extension:Department.value[x].reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.extension:ADTBased	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIADTBasedExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIADTBasedExtension</a>
Encounter.extension:ADTBased.value[x]	√	1..1	
Encounter.status	√	0..*	
Encounter.statusHistory		0..*	
Encounter.class	√	0..*	
Encounter.class.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Encounter.class.extension:PatientClass	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIClassExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EIClassExtension</a>
Encounter.class.extension:PatientClass.extension:Code	√	0..*	
Encounter.class.extension:PatientClass.extension:Code.value[x]	√	1..1	
Encounter.class.extension:PatientClass.extension:Description	√	0..*	
Encounter.class.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.class.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Encounter.classHistory		0..*	
Encounter.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.type.extension		0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Encounter.type.extension:Code	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EITypeCodeExtension">http://www.agfa.com/ei/fhir/StructureDefinition/EITypeCodeExtension</a>
Encounter.type.extension:Code.value[x]	√	1..1	
Encounter.type.text	√	0..*	
Encounter.subject	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Group">http://hl7.org/fhir/StructureDefinition/Group</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatient</a>
Encounter.subject.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.participant	√	0..*	
Encounter.participant:AttendingPhysician	√	0..*	

Resource Elements	support	Card	StructureDefinition
Encounter.participant:AttendingPhysician.type	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.participant:AttendingPhysician.type.coding	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Coding">http://hl7.org/fhir/StructureDefinition/Coding</a>
Encounter.participant:AttendingPhysician.type.coding.system	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Encounter.participant:AttendingPhysician.type.coding.code	√	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Encounter.participant:AttendingPhysician.individual	√	0..*	
Encounter.participant:AttendingPhysician.individual.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Encounter.period	√	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Period">http://hl7.org/fhir/StructureDefinition/Period</a>
Encounter.period.start	√	0..*	
Encounter.period.end	√	0..*	
Encounter.diagnosis		0..*	
Encounter.partOf	√	0..*	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientAdmission">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientAdmission</a>
Encounter.partOf.reference	√	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

## 4 EXTENSIONS

### 4.1 EIADTBasedExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

### 4.2 EIAnnotationCommentKeyExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:CommentKey	0..*	
Extension.extension:CommentKey.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:CommentKey.value[x]	1..1	
Extension.extension:CommentKey.value[x].coding.code	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Extension.extension:CommentKey.value[x].coding.display	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:CommentKey.value[x].text	0..*	

### 4.3 EIAttachmentContentExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Tag	0..*	
Extension.extension:Tag.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Tag.value[x]	1..1	
Extension.extension:Tag.value[x].coding.code	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Extension.extension:Tag.value[x].coding.display	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:Tag.value[x].text	0..*	
Extension.extension:FileFormat	0..*	
Extension.extension:FileFormat.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:FileFormat.value[x]	1..1	
Extension.extension:User	0..*	
Extension.extension:User.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:User.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
Extension.extension:User.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

Extension Elements	Card	StructureDefinition
Extension.extension:TagCatalog	0..*	
Extension.extension:TagCatalog.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:TagCatalog.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.4 EIAttachmentExtension

Extension Elements	Card	StructureDefinition
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Definition	0..*	
Extension.extension:Definition.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Definition.value[x]	1..1	
Extension.extension:Scope	0..*	
Extension.extension:Scope.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Scope.value[x]	1..1	
Extension.extension:Scope.value[x].system	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Extension.extension:Scope.value[x].code	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Extension.extension:Active	0..*	
Extension.extension:Active.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Active.value[x]	1..1	
Extension.extension:Level	0..*	
Extension.extension:Level.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Level.value[x]	1..1	
Extension.extension:Level.value[x].system	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Extension.extension:Level.value[x].code	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	Extension.value[x]

#### 4.5 EIAttachmentLinkExtension

Extension Elements	Card	StructureDefinition
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachment">http://www.agfa.com/ei/fhir/StructureDefinition/EIAttachment</a>
Extension.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

## 4.6 EIClassExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Code	0..*	
Extension.extension:Code.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Code.value[x]	1..1	
Extension.extension:Description	0..*	
Extension.extension:Description.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Description.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.7 EIConfigLabelExtension

Extension Elements	Card	StructureDefinition
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Code	0..*	
Extension.extension:Code.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Code.value[x]	1..1	
Extension.extension:Name	0..*	
Extension.extension:Name.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Name.value[x]	1..1	
Extension.extension:Active	0..*	
Extension.extension:Active.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Active.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.8 EICountersExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:TotalCount	0..*	
Extension.extension:TotalCount.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:TotalCount.value[x]	1..1	
Extension.extension:StatCount	0..*	

Extension Elements	Card	StructureDefinition
Extension.extension:StatCount.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:StatCount.value[x]	1..1	
Extension.extension:HasSuppressedResults	0..*	
Extension.extension:HasSuppressedResults.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:HasSuppressedResults.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.9 EIDataRetrievalStatusExtension

Extension Elements	Card	StructureDefinition
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.10 EIDigitalDictationExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:SoundFile	0..*	
Extension.extension:SoundFile.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:SoundFile.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile</a>
Extension.extension:SoundFile.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:Format	0..*	
Extension.extension:Format.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Format.value[x]	1..1	
Extension.extension:Purged	0..*	
Extension.extension:Purged.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Purged.value[x]	1..1	
Extension.extension:Duration	0..*	
Extension.extension:Duration.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Duration.value[x]	1..1	
Extension.extension:PrimaryKey	0..*	
Extension.extension:PrimaryKey.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:PrimaryKey.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>

Extension Elements	Card	StructureDefinition
Extension.value[x]	1..1	

#### 4.11 EIInternalIdExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.12 EILocationDetailsExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Bed	0..*	
Extension.extension:Bed.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Bed.value[x]	1..1	
Extension.extension:Room	0..*	
Extension.extension:Room.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Room.value[x]	1..1	
Extension.extension:Facility	0..*	
Extension.extension:Facility.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Facility.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIFacility">http://www.agfa.com/ei/fhir/StructureDefinition/EIFacility</a>
Extension.extension:Facility.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:Department	0..*	
Extension.extension:Department.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Department.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment">http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment</a>
Extension.extension:Department.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.13 EIMatchingIssuerExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	
Extension.value[x]	1..1	

#### 4.14 EIModifiedByExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:ModificationDate	0..*	
Extension.extension:ModificationDate.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ModificationDate.value[x]	1..1	
Extension.extension:ModifiedBy	0..*	
Extension.extension:ModifiedBy.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ModifiedBy.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
Extension.extension:ModifiedBy.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.15 EIMultiMediaReferenceContentExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:ModifiedTime	0..*	
Extension.extension:ModifiedTime.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ModifiedTime.value[x]	1..1	
Extension.extension:ConvertToDICOM	0..*	
Extension.extension:ConvertToDICOM.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ConvertToDICOM.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.16 EIOnlineRecognitionExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:WavFile	0..*	
Extension.extension:WavFile.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:WavFile.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile</a>



Extension Elements	Card	StructureDefinition
Extension.extension:WavFile.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:SrrFile	0..*	
Extension.extension:SrrFile.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:SrrFile.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile</a>
Extension.extension:SrrFile.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:SmdFile	0..*	
Extension.extension:SmdFile.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:SmdFile.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportSoundFile</a>
Extension.extension:SmdFile.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:Purged	0..*	
Extension.extension:Purged.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Purged.value[x]	1..1	
Extension.extension:ParagraphWhitespace	0..*	
Extension.extension:ParagraphWhitespace.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ParagraphWhitespace.value[x]	1..1	
Extension.extension:PrimaryKey	0..*	
Extension.extension:PrimaryKey.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:PrimaryKey.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.17 EIOriginalPatientExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	
Extension.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.value[x].display	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

#### 4.18 EIParticipationTypeExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Code	0..*	

Extension Elements	Card	StructureDefinition
Extension.extension:Code.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Code.value[x]	1..1	
Extension.extension:AddedOn	0..*	
Extension.extension:AddedOn.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:AddedOn.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.19 EIPatientExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:VIP	0..*	
Extension.extension:VIP.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:VIP.value[x]	1..1	
Extension.extension:SocialSecurityNumber	0..*	
Extension.extension:SocialSecurityNumber.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:SocialSecurityNumber.value[x]	1..1	
Extension.extension:Pregnancy	0..*	
Extension.extension:Pregnancy.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Pregnancy.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientPregnancy">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientPregnancy</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.20 EIPregnancyLinkExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientPregnancy">http://www.agfa.com/ei/fhir/StructureDefinition/EIPatientPregnancy</a>

#### 4.21 EIPrimaryBodyPartExtension

Extension Elements	Card	StructureDefinition
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.22 EIQuantityValueExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.23 EIReportResultExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:AwaitingContent	0..*	
Extension.extension:AwaitingContent.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:AwaitingContent.value[x]	1..1	
Extension.extension:MsdPrinted	0..*	
Extension.extension:MsdPrinted.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:MsdPrinted.value[x]	1..1	
Extension.extension:External	0..*	
Extension.extension:External.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:External.value[x]	1..1	
Extension.extension:CreationDate	0..*	
Extension.extension:CreationDate.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:CreationDate.value[x]	1..1	
Extension.extension:Intermediate	0..*	
Extension.extension:Intermediate.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Intermediate.value[x]	1..1	
Extension.extension:InConflictReason	0..*	
Extension.extension:InConflictReason.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:InConflictReason.value[x]	1..1	
Extension.extension:ReportBlob	0..*	
Extension.extension:ReportBlob.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ReportBlob.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIReportBlob">http://www.agfa.com/ei/fhir/StructureDefinition/EIReportBlob</a>
Extension.extension:ReportBlob.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.24 EIReportResultStatisticsExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:WordCount	0..*	
Extension.extension:WordCount.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:WordCount.value[x]	1..1	
Extension.extension:WordCountTyped	0..*	
Extension.extension:WordCountTyped.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:WordCountTyped.value[x]	1..1	
Extension.extension:LetterCount	0..*	
Extension.extension:LetterCount.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:LetterCount.value[x]	1..1	
Extension.extension:LetterCountTyped	0..*	
Extension.extension:LetterCountTyped.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:LetterCountTyped.value[x]	1..1	
Extension.extension:LetterCountNoWhitespace	0..*	
Extension.extension:LetterCountNoWhitespace.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:LetterCountNoWhitespace.value[x]	1..1	
Extension.extension:LetterCountTypedNoWhitespace	0..*	
Extension.extension:LetterCountTypedNoWhitespace.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:LetterCountTypedNoWhitespace.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.25 EIReportResultStatusExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.26 EIReportSectionExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>

Extension Elements	Card	StructureDefinition
Extension.extension:SpeechJobUID	0..*	
Extension.extension:SpeechJobUID.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:SpeechJobUID.value[x]	1..1	
Extension.extension:Mandatory	0..*	
Extension.extension:Mandatory.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Mandatory.value[x]	1..1	
Extension.extension:PriorReport	0..*	
Extension.extension:PriorReport.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:PriorReport.value[x]	1..1	
Extension.extension:RequestedProceduresCovered	0..*	
Extension.extension:RequestedProceduresCovered.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:RequestedProceduresCovered.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure">http://www.agfa.com/ei/fhir/StructureDefinition/EIRequestedProcedureAsProcedure</a>
Extension.extension:RequestedProceduresCovered.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:ReportSectionDefinitionCode	0..*	
Extension.extension:ReportSectionDefinitionCode.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ReportSectionDefinitionCode.value[x]	1..1	
Extension.extension:Sequence	0..*	
Extension.extension:Sequence.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Sequence.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 4.27 EIReportSeverityExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Code	0..*	
Extension.extension:Code.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Code.value[x]	1..1	
Extension.extension:UpdatedOn	0..*	
Extension.extension:UpdatedOn.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:UpdatedOn.value[x]	1..1	
Extension.extension:UpdatedBy	0..*	
Extension.extension:UpdatedBy.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>

Extension Elements	Card	StructureDefinition
Extension.extension:UpdatedBy.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
Extension.extension:UpdatedBy.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.28 EIRequestedProcedureAsProcedureExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:Unverified	0..*	
Extension.extension:Unverified.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Unverified.value[x]	1..1	
Extension.extension:DicomBased	0..*	
Extension.extension:DicomBased.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:DicomBased.value[x]	1..1	
Extension.extension:GuestStudy	0..*	
Extension.extension:GuestStudy.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:GuestStudy.value[x]	1..1	
Extension.extension:InitialStoreDone	0..*	
Extension.extension:InitialStoreDone.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:InitialStoreDone.value[x]	1..1	
Extension.extension:OriginalStudyUID	0..*	
Extension.extension:OriginalStudyUID.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:OriginalStudyUID.value[x]	1..1	
Extension.extension:ModalityType	0..*	
Extension.extension:ModalityType.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ModalityType.value[x]	1..1	
Extension.extension:ModalityType.value[x].code	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Extension.extension:ModalityType.value[x].display	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:HasReport	0..*	
Extension.extension:HasReport.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:HasReport.value[x]	1..1	
Extension.extension:HasValidatedReport	0..*	
Extension.extension:HasValidatedReport.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>

Extension Elements	Card	StructureDefinition
Extension.extension:HasValidatedReport.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.29 EIRequestedProcedurePriorityExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:PriorityLevel	0..*	
Extension.extension:PriorityLevel.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:PriorityLevel.value[x]	1..1	
Extension.extension:PriorityName	0..*	
Extension.extension:PriorityName.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:PriorityName.value[x]	1..1	
Extension.extension:PriorityCode	0..*	
Extension.extension:PriorityCode.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:PriorityCode.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.30 EIScheduledProcedureStepStatusExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.31 EISecUserExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:UserAccount	0..*	
Extension.extension:UserAccount.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:UserAccount.extension:LoginName	0..*	
Extension.extension:UserAccount.extension:LoginName.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>

Extension Elements	Card	StructureDefinition
Extension.extension:UserAccount.extension:LoginName.value[x]	1..1	
Extension.extension:UserAccount.extension:Domain	0..*	
Extension.extension:UserAccount.extension:Domain.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:UserAccount.extension:Domain.value[x]	1..1	
Extension.extension:UserAccount.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:UserAccount.value[x]	1..1	
Extension.extension:Guid	0..*	
Extension.extension:Guid.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Guid.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.32 EIServiceRequestOriginalPatientIDExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier">http://www.agfa.com/ei/fhir/StructureDefinition/EIIdentifier</a>
Extension.value[x].value	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.value[x].assigner.display	0..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

#### 4.33 EIServiceRequestRequesterExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:RequestingDepartment	0..*	
Extension.extension:RequestingDepartment.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:RequestingDepartment.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment">http://www.agfa.com/ei/fhir/StructureDefinition/EIDepartment</a>
Extension.extension:RequestingDepartment.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:RequestingFacility	0..*	
Extension.extension:RequestingFacility.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:RequestingFacility.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIFacility">http://www.agfa.com/ei/fhir/StructureDefinition/EIFacility</a>



Extension Elements	Card	StructureDefinition
Extension.extension:RequestingFacility.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.34 EIServiceTypeCodeExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.35 EISourceExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.36 EIStudyExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:NumberOfImages	0..*	
Extension.extension:NumberOfImages.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:NumberOfImages.value[x]	1..1	
Extension.extension:Availability	0..*	
Extension.extension:Availability.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:Availability.value[x]	1..1	
Extension.extension:Availability.value[x].system	0..1	<a href="http://hl7.org/fhir/StructureDefinition/uri">http://hl7.org/fhir/StructureDefinition/uri</a>
Extension.extension:Availability.value[x].code	0..1	<a href="http://hl7.org/fhir/StructureDefinition/code">http://hl7.org/fhir/StructureDefinition/code</a>
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.37 EITaskInfoExtension

Extension Elements	Card	StructureDefinition
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.38 EITaskInstanceAdditionRemovalExtension

Extension Elements	Card	StructureDefinition
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:RemoveRequestedProcedurePossible	0..*	
Extension.extension:RemoveRequestedProcedurePossible.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:RemoveRequestedProcedurePossible.value[x]	1..1	
Extension.extension:AddRequestedProcedurePossible	0..*	
Extension.extension:AddRequestedProcedurePossible.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:AddRequestedProcedurePossible.value[x]	1..1	
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.39 EITaskInstancePriorityExtension

Extension Elements	Card	StructureDefinition
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

#### 4.40 EITaskInstanceResponsiblesExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	<a href="http://hl7.org/fhir/StructureDefinition/Extension">http://hl7.org/fhir/StructureDefinition/Extension</a>
Extension.extension:IncludedResponsibles	0..*	
Extension.extension:IncludedResponsibles.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:IncludedResponsibles.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIASignmentGroup">http://www.agfa.com/ei/fhir/StructureDefinition/EIASignmentGroup</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a>
Extension.extension:IncludedResponsibles.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>
Extension.extension:ExcludedResponsibles	0..*	
Extension.extension:ExcludedResponsibles.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.extension:ExcludedResponsibles.value[x]	1..1	<a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional">http://www.agfa.com/ei/fhir/StructureDefinition/EIProfessional</a> <a href="http://www.agfa.com/ei/fhir/StructureDefinition/EIASignmentGroup">http://www.agfa.com/ei/fhir/StructureDefinition/EIASignmentGroup</a>
Extension.extension:ExcludedResponsibles.value[x].reference	1..1	<a href="http://hl7.org/fhir/StructureDefinition/string">http://hl7.org/fhir/StructureDefinition/string</a>

Extension Elements	Card	StructureDefinition
Extension.url	0..*	http://hl7.org/fhir/StructureDefinition/url
Extension.value[x]	1..1	

#### 4.41 EITaskInstanceSubtypeExtension

Extension Elements	Card	StructureDefinition
Extension.url	0..*	http://hl7.org/fhir/StructureDefinition/url
Extension.value[x]	1..1	
Extension.value[x].system	0..1	http://hl7.org/fhir/StructureDefinition/uri
Extension.value[x].code	0..1	http://hl7.org/fhir/StructureDefinition/code

#### 4.42 EITextSectionExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.extension	0..*	http://hl7.org/fhir/StructureDefinition/Extension
Extension.extension:TextFormat	0..*	
Extension.extension:TextFormat.url	0..*	http://hl7.org/fhir/StructureDefinition/url
Extension.extension:TextFormat.value[x]	1..1	
Extension.extension:Online	0..*	
Extension.extension:Online.url	0..*	http://hl7.org/fhir/StructureDefinition/url
Extension.extension:Online.value[x]	1..1	
Extension.url	0..*	http://hl7.org/fhir/StructureDefinition/url
Extension.value[x]	1..1	

#### 4.43 EITransferLinkExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	http://hl7.org/fhir/StructureDefinition/url
Extension.value[x]	1..1	http://www.agfa.com/ei/fhir/StructureDefinition/EITransfer
Extension.value[x].reference	1..1	http://hl7.org/fhir/StructureDefinition/string

#### 4.44 EITypeCodeExtension

Extension Elements	Card	StructureDefinition
Extension	0..*	
Extension.url	0..*	http://hl7.org/fhir/StructureDefinition/url
Extension.value[x]	1..1	

## 4.45 EIVolumeTypeExtension

Extension Elements	Card	StructureDefinition
Extension.url	0..*	<a href="http://hl7.org/fhir/StructureDefinition/url">http://hl7.org/fhir/StructureDefinition/url</a>
Extension.value[x]	1..1	

## 5 CODESYSTEMS & VALUESETS

### 5.1 EIAttachmentLevelCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/ei-cs-patient-type> defines the following codes:.

Code	Display
NORMAL	Normal
WARNING	Warning

#### 5.1.1 EIAttachmentLevelValueSet

ValueSet elements	Value
ValueSet.id[0]	EIAttachmentLevelValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIAttachmentLevel">http://www.agfa.com/ei/fhir/ValueSet/EIAttachmentLevel</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIAttachmentLevelValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentLevel">http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentLevel</a>

### 5.2 EIAttachmentScopeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentScope> defines the following codes:

Code	Display
PUBLIC	Public scope
PRIVATE	Private scope
TRUSTED	Trusted scope

#### 5.2.1 EIAttachmentScopeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIAttachmentScopeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIAttachmentScope">http://www.agfa.com/ei/fhir/ValueSet/EIAttachmentScope</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIAttachmentScopeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentScope">http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentScope</a>

### 5.3 EIAAttachmentTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentType> defines the following codes:

Code	Display
TEXT	Text attachment
KEYWORD	Keyword attachment
URL	URL attachment
IMAGE	Image attachment
DOCUMENT	Document attachment
VOICE_CLIP	Voiceclip attachment
PDF	PDF attachment
PDF_IMAGE	PDF Image attachment
USER	User attachment

#### 5.3.1 EIAAttachmentTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIAAttachmentTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIAttachmentType">http://www.agfa.com/ei/fhir/ValueSet/EIAttachmentType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIAAttachmentTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentType">http://www.agfa.com/ei/fhir/CodeSystem/EIAttachmentType</a>

### 5.4 EICommentLogCommentKeyCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EICommentLogCommentKey> defines the following codes:

Code	Display
FIXMERGE_MULTI_PROCEDURE_VERIFY_REPORT	FIXMERGE_MULTI_PROCEDURE_VERIFY_REPORT
FIXMERGE_REPORT_MOVED_VERIFY_REPORT_INTERNAL	FIXMERGE_REPORT_MOVED_VERIFY_REPORT_INTERNAL
FIXMERGE_VERIFY_REPORT_EXTERNAL	FIXMERGE_VERIFY_REPORT_EXTERNAL
FIXMERGE_VERIFY_REPORT_INTERNAL	FIXMERGE_VERIFY_REPORT_INTERNAL
SPLITSEGMENT_MULTI_PROCEDURE_VERIFY_REPORT	SPLITSEGMENT_MULTI_PROCEDURE_VERIFY_REPORT

Code	Display
VERIFY_REPORT_LATE_IMAGE_INTERNAL	VERIFY_REPORT_LATE_IMAGE_INTERNAL

### 5.4.1 EICommentLogCommentKeyValueSet

ValueSet elements	Value
ValueSet.id[0]	EICommentLogCommentKeyValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EICommentLogCommentKey">http://www.agfa.com/ei/fhir/ValueSet/EICommentLogCommentKey</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EICommentLogCommentKeyValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EICommentLogCo">http://www.agfa.com/ei/fhir/CodeSystem/EICommentLogCo</a> mmentKey

### 5.5 EIDataRetrievalStatusCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIDataRetrievalStatus> defines the following codes:

Code	Display
AVAILABLE	Data is available
NOT_ASKED	Data is not available, because it was not asked
TOO_MANY_RESULT	Data is not available, because the query would return too many results (slow query)
AVAILABLE_WITH_COUNT_EXCEEDS_MAX_RESULTS	Data is available even if the query returned too many results. The number of retrieved items is equal or less than the max results.
KEYS_ONLY	keys only

#### 5.5.1 EIDataRetrievalStatusValueSet

ValueSet elements	Value
ValueSet.id[0]	EIDataRetrievalStatusValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIDataRetrievalStatus">http://www.agfa.com/ei/fhir/ValueSet/EIDataRetrievalStatus</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIDataRetrievalStatusValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIDataRetrievalSta">http://www.agfa.com/ei/fhir/CodeSystem/EIDataRetrievalSta</a> tus

## 5.6 EIDocumentReferenceTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIDocumentReferenceType> defines the following codes:

Code	Display
ATTACHMENT	EI Attachment
MULTIMEDIA_REFERENCE	EI Multimedia Reference

### 5.6.1 EIDocumentReferenceTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIDocumentReferenceTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIDocumentReferenceType">http://www.agfa.com/ei/fhir/ValueSet/EIDocumentReferenceType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIDocumentReferenceTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIDocumentReferenceType">http://www.agfa.com/ei/fhir/CodeSystem/EIDocumentReferenceType</a>

## 5.7 EIDocumentTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIDocumentType> defines the following codes:

Code	Display
ATTACHMENT	native file for attachment
XDS	native file for XDS
NATIVE	other native files

### 5.7.1 EIDocumentTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIDocumentTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIDocumentType">http://www.agfa.com/ei/fhir/ValueSet/EIDocumentType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIDocumentTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIDocumentType">http://www.agfa.com/ei/fhir/CodeSystem/EIDocumentType</a>



## 5.8 EIdentifierTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIdentifierType> defines the following codes:

Code	Display
PPS_ID	Performed Procedure Step ID
SPS_ID	Scheduled Procedure Step ID
PPS_SOP_IUID	Performed Procedure Step SOP Instance UID
RP_ID	Requested Procedure ID
STUDY_UID	Study UID
RP_PLAN	Requested Procedure Plan Code
ADMISSION_NR	Admission Number
REPORT_UID	Report UID
REPORT_CODE	Report Code
MEASUREMENTS_GROUP	Measurement Container UID
IDENTITY_CODE	Identity code
QUERY_LIST_CODE	Query List Code
ACTIVITIES_OVERVIEW_CODE	Activities Overview Code

### 5.8.1 EIdentifierTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIdentifierTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIdentifierType">http://www.agfa.com/ei/fhir/ValueSet/EIdentifierType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIdentifierTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIdentifierType">http://www.agfa.com/ei/fhir/CodeSystem/EIdentifierType</a>

## 5.9 EInConflictReasonCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EInConflictReason> defines the following codes:

Code	Display
QC_FIX_MERGE	QC fix merge
QC_POST_PACS	Post pacs QC
PROCEDURES_UNLINKED	Procedures have been unlinked
UNKNOWN	Unknown

### 5.9.1 EInConflictReasonValueSet

ValueSet elements	Value
ValueSet.id[0]	EInConflictReasonValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EInConflictReason">http://www.agfa.com/ei/fhir/ValueSet/EInConflictReason</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EInConflictReasonValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EInConflictReason">http://www.agfa.com/ei/fhir/CodeSystem/EInConflictReason</a>

### 5.10 EILateralityCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EILaterality> defines the following codes:

Code	Display
R	Right
L	Left
U	Unpaired
BOTH	Left and right
ND	Not defined

#### 5.10.1 EILateralityValueSet

ValueSet elements	Value
ValueSet.id[0]	EILateralityValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EILaterality">http://www.agfa.com/ei/fhir/ValueSet/EILaterality</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EILateralityValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EILaterality">http://www.agfa.com/ei/fhir/CodeSystem/EILaterality</a>

### 5.11 EIListTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIListType> defines the following codes:

Code	Display
ACTIVITIES_OVERVIEW	Activities Overview

### 5.11.1 EListTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EListTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EListType">http://www.agfa.com/ei/fhir/ValueSet/EListType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EListTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EListType">http://www.agfa.com/ei/fhir/CodeSystem/EListType</a>

### 5.12 EIOrganizationTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIOrganizationType> defines the following codes:

Code	Display
FACILITY	Facility
DEPARTMENT	Department

### 5.12.1 EIOrganizationTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIOrganizationTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIOrganizationType">http://www.agfa.com/ei/fhir/ValueSet/EIOrganizationType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIOrganizationTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIOrganizationType">http://www.agfa.com/ei/fhir/CodeSystem/EIOrganizationType</a>

### 5.13 EIPatientClassCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIPatientClass> defines the following codes:

Code	Display
CUSTOM	Custom patient class
NA	Not applicable

### 5.13.1 EIPatientClassValueSet

ValueSet elements	Value
ValueSet.id[0]	EIPatientClassValueSet

ValueSet elements	Value
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIPatientClass">http://www.agfa.com/ei/fhir/ValueSet/EIPatientClass</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIPatientClassValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIPatientClass">http://www.agfa.com/ei/fhir/CodeSystem/EIPatientClass</a>

## 5.14 EIPerformerFunctionCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIPerformerFunction> defines the following codes:

Code	Display
PERFORMING_PHYSICIAN	Performing physician
PERFORMING_DEPARTMENT	Performing department
RESPONSIBLE_TECHNOLOGIST	Responsible technologist
INVOLVED_TECHNOLOGIST	Involved technologist

### 5.14.1 EIPerformerFunctionValueSet

ValueSet elements	Value
ValueSet.id[0]	EIPerformerFunctionValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIPerformerFunction">http://www.agfa.com/ei/fhir/ValueSet/EIPerformerFunction</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIPerformerFunctionValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIPerformerFunction">http://www.agfa.com/ei/fhir/CodeSystem/EIPerformerFunction</a>

## 5.15 EIPriorityLevelCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIPriorityLevel> defines the following codes:

Code	Display
Lowest	Lowest
Low	Low
Normal	Normal
High	High
Highest	Highest

### 5.15.1 EIPriorityLevelValueSet

ValueSet elements	Value
ValueSet.id[0]	EIPriorityLevelValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIPriorityLevel">http://www.agfa.com/ei/fhir/ValueSet/EIPriorityLevel</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIPriorityLevelValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIPriorityLevel">http://www.agfa.com/ei/fhir/CodeSystem/EIPriorityLevel</a>

### 5.16 EIProcedureTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIProcedureType> defines the following codes:

Code	Display
REQUESTED_PROCEDURE	Requested Procedure
PERFORMED_PROCEDURE_STEP	Performed Procedure Step

### 5.16.1 EIProcedureTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIProcedureTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIProcedureType">http://www.agfa.com/ei/fhir/ValueSet/EIProcedureType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIProcedureTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIProcedureType">http://www.agfa.com/ei/fhir/CodeSystem/EIProcedureType</a>

### 5.17 EIQueryListTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIQueryListType> defines the following codes:

Code	Display
TASKLIST	task List
STUDYLIST	study List
REPORTLIST	report list
LABELEDLIST	labeled list
DICOMQUERY	dicom query
CLIENT_SIDE_DICOM_QUERY	client side dicom query

Code	Display
CONFERENCE	conference list
ENTERPRISE_SEARCH_QUERY	enterprise search query

### 5.17.1 EIQueryListTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIQueryListTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIQueryListType">http://www.agfa.com/ei/fhir/ValueSet/EIQueryListType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIQueryListTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIQueryListType">http://www.agfa.com/ei/fhir/CodeSystem/EIQueryListType</a>

### 5.18 EIReportResultStatusCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIReportResultStatus> defines the following codes:

Code	Display
VALIDATED	Validated
PRELIMINARY	Preliminary
INCONFLICT	In conflict
PARTIALLY_VALIDATED	Partially validated

#### 5.18.1 EIReportResultStatusValueSet

ValueSet elements	Value
ValueSet.id[0]	EIReportResultStatusValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIReportResultStatus">http://www.agfa.com/ei/fhir/ValueSet/EIReportResultStatus</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIReportResultStatusValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIReportResultStatus">http://www.agfa.com/ei/fhir/CodeSystem/EIReportResultStat us</a>

### 5.19 EIReportSectionContentTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIReportSectionContentType> defines the following codes:

Code	Display
TEXT	Text section
MEASUREMENT	Measurement section

### 5.19.1 EIReportSectionContentTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIReportSectionContentTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIReportSectionContentType">http://www.agfa.com/ei/fhir/ValueSet/EIReportSectionContentType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIReportSectionContentTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIReportSectionContentType">http://www.agfa.com/ei/fhir/CodeSystem/EIReportSectionContentType</a>

### 5.20 EIScheduledProcedureStepStatusCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIScheduledProcedureStepStatus> defines the following codes:

Code	Display
IN_PROGRESS	In progress
DISCONTINUED	Discontinued
SCHEDULED	Scheduled
STARTED	Started
ARRIVED	Arrived
CANCELLED	Cancelled
COMPLETED	Completed
ORDERED	Ordered

### 5.20.1 EIScheduledProcedureStepStatusValueSet

ValueSet elements	Value
ValueSet.id[0]	EIScheduledProcedureStepStatusValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIScheduledProcedureStepStatus">http://www.agfa.com/ei/fhir/ValueSet/EIScheduledProcedureStepStatus</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIScheduledProcedureStepStatusValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False

ValueSet elements	Value
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIScheduledProcedureStepStatus">http://www.agfa.com/ei/fhir/CodeSystem/EIScheduledProcedureStepStatus</a>

## 5.21 EIServiceRequestTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIServiceRequestType> defines the following codes:

Code	Display
SERVICE_REQUEST	Service Request
REQUESTED_PROCEDURE	Requested Procedure
SCHEDULED_PROCEDURE_STEP	Scheduled Procedure Step

### 5.21.1 EIServiceRequestTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIServiceRequestTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIServiceRequestType">http://www.agfa.com/ei/fhir/ValueSet/EIServiceRequestType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIServiceRequestTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIServiceRequestType">http://www.agfa.com/ei/fhir/CodeSystem/EIServiceRequestType</a>

## 5.22 EIStudyAvailabilityCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIStudyAvailability> defines the following codes:

Code	Display
ONLINE	Online
NEARLINE	Nearline
OFFLINE	Offline
UNAVAILABLE	Unavailable
UNKNOWN	Unknown

### 5.22.1 EIStudyAvailabilityValueSet

ValueSet elements	Value
ValueSet.id[0]	EIStudyAvailabilityValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIStudyAvailability">http://www.agfa.com/ei/fhir/ValueSet/EIStudyAvailability</a>
ValueSet.version[0]	1.1



ValueSet elements	Value
ValueSet.name[0]	EIStudyAvailabilityValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIStudyAvailability">http://www.agfa.com/ei/fhir/CodeSystem/EIStudyAvailability</a>

## 5.23 EITaskInputDataCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputData> defines the following codes:

Code	Display
requestedProcedures	requestedProcedures
PARENT_REPORT	PARENT_REPORT
report	report
fixMergeContext	fixMergeContext
splitSegmentContext	splitSegmentContext
editorTool	editorTool
user	user
reportStructureDefinitionId	reportStructureDefinitionId
autoMerge	autoMerge
force_QC	force_QC
proceduresToQC	proceduresToQC
reportResidingProcedure	reportResidingProcedure
studyNoVerif	studyNoVerif
unverifiedReading	unverifiedReading
unsign	unsign
reportStatus	reportStatus
reportValidator	reportValidator
reportValidatorDate	reportValidatorDate
ignoreReadingWorkflow	ignoreReadingWorkflow
originatingTask	originatingTask
cancelledByUser	cancelledByUser
requestedProcedurePlanStep	requestedProcedurePlanStep
LoginUsers	LoginUsers
activeStudies	activeStudies
conference	conference
conferenceSession	conferenceSession

Code	Display
nrAuthors	nrAuthors
SOURCE_TASK_ID	SOURCE_TASK_ID
reopenCase	reopenCase
createCaseReview	createCaseReview
InternalStart	InternalStart
FORCE_WORKFLOW_CREATION	FORCE_WORKFLOW_CREATION
futureResponsibles	futureResponsibles
createNewCase	createNewCase
originalMasterStudies	originalMasterStudies

### 5.23.1 EITaskInputDataValueSet

ValueSet elements	Value
ValueSet.id[0]	EITaskInputDataValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EITaskInputData">http://www.agfa.com/ei/fhir/ValueSet/EITaskInputData</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EITaskInputDataValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputData">http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputData</a>

### 5.23.2 EITaskInputMapsValueSet

ValueSet elements	Value
ValueSet.id[0]	EITaskInputMapsValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EITaskInputMaps">http://www.agfa.com/ei/fhir/ValueSet/EITaskInputMaps</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EITaskInputMapsValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputVariables">http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputVariables</a>
ValueSet.compose[0].include[1].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputData">http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputData</a>

### 5.24 EITaskInputRequestedProceduresCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputRequestedProcedures> defines the following codes:

Code	Display
requestedProcedures	requestedProcedures
unlinkedRequestedProcedures	requestedProcedures

### 5.24.1 EITaskInputRequestedProceduresValueSet

ValueSet elements	Value
ValueSet.id[0]	EITaskInputRequestedProceduresValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EITaskInputRequestedProcedures">http://www.agfa.com/ei/fhir/ValueSet/EITaskInputRequestedProcedures</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EITaskInputRequestedProceduresValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputRequestedProcedures">http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputRequestedProcedures</a>

### 5.25 EITaskInputVariablesCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputVariables> defines the following codes:

Code	Display
activeStudies	activeStudies
activityContext	activityContext
autoMerge	autoMerge
cancelledByUser	cancelledByUser
conference	conference
conferenceSession	conferenceSession
editorTool	editorTool
followUp	followUp
force_QC	force_QC
INTERLOCK_TASK	INTERLOCK_TASK
MESSAGE	MESSAGE
nrAuthors	nrAuthors
originalMasterStudies	originalMasterStudies
originatingTask	originatingTask
PARENT_REPORT	PARENT_REPORT
report	report
ReportID	ReportID

Code	Display
reportStatus	reportStatus
reportValidator	reportValidator
reportValidatorDate	reportValidatorDate
requestedProcedurePlanStep	requestedProcedurePlanStep
requestedProcedures	requestedProcedures
skipInterlock	skipInterlock
skipInterlockSplitMerge	skipInterlockSplitMerge
SKIP_SPS_UPDATE	SKIP_SPS_UPDATE
studyNoVerif	studyNoVerif
StudyUID	StudyUID
targetRequestedProcedure	targetRequestedProcedure
TASK_CREATION_SCRID	TASK_CREATION_SCRID
unlinking	unlinking
unsign	unsign
unverifiedReading	unverifiedReading
WORKFLOW_NAME	WORKFLOW_NAME
workflowOverrides	workflowOverrides

### 5.25.1 EITaskInputVariablesValueSet

ValueSet elements	Value
ValueSet.id[0]	EITaskInputVariablesValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EITaskInputVariables">http://www.agfa.com/ei/fhir/ValueSet/EITaskInputVariables</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EITaskInputVariablesValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputVariables">http://www.agfa.com/ei/fhir/CodeSystem/EITaskInputVariables</a>

### 5.26 EITaskInstanceSubtypeCodes

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EITaskInstanceSubtypeCodes> defines the following codes:

Code	Display
Feedback	Transcription/Correction after feedback
Verify report	Verify report
Verify review	Review feedback

Code	Display
FixMerge	FixMerge
Split	Split
Segment	Segment
Delete	Delete
Feedback_new_case	New case after feedback
Feedback_case_review	Case review after feedback
Feedback_case_follow_up	Case follow-up after feedback

### 5.26.1 EITaskInstanceSubtypeCodesValueSet

ValueSet elements	Value
ValueSet.id[0]	EITaskInstanceSubtypeCodesValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EITaskInstanceSubtypeCodes">http://www.agfa.com/ei/fhir/ValueSet/EITaskInstanceSubtypeCodes</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EITaskInstanceSubtypeCodesValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EITaskInstanceSubtypeCodes">http://www.agfa.com/ei/fhir/CodeSystem/EITaskInstanceSubtypeCodes</a>

### 5.27 EITaskInstanceTypeCodes

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EITaskInstanceTypeCodes> defines the following codes:

Code	Display
transcribe dictation	
report approval	
create service request	
reading	
correction	
addendum reading	
review	
qc	
qc-segment	
order entry	
preparation	
acquisition	

Code	Display
post-processing	
order review	
print and export	
discussion	
retake	
new case	
case review	
case follow-up	
case feedback	

## 5.28 EIUVerifiedTypeCodeSystem

This code system <http://www.agfa.com/ei/fhir/CodeSystem/EIUVerifiedType> defines the following codes:

Code	Display
VERIFIED	Verified
NO_VERIFICATION	No Verification
FAILED_VERIFICATION	Failed Verification

### 5.28.1 EIUVerifiedTypeValueSet

ValueSet elements	Value
ValueSet.id[0]	EIUVerifiedTypeValueSet
ValueSet.url[0]	<a href="http://www.agfa.com/ei/fhir/ValueSet/EIUVerifiedType">http://www.agfa.com/ei/fhir/ValueSet/EIUVerifiedType</a>
ValueSet.version[0]	1.1
ValueSet.name[0]	EIUVerifiedTypeValueSet
ValueSet.status[0]	draft
ValueSet.experimental[0]	False
ValueSet.compose[0].include[0].system[0]	<a href="http://www.agfa.com/ei/fhir/CodeSystem/EIUVerifiedType">http://www.agfa.com/ei/fhir/CodeSystem/EIUVerifiedType</a>