
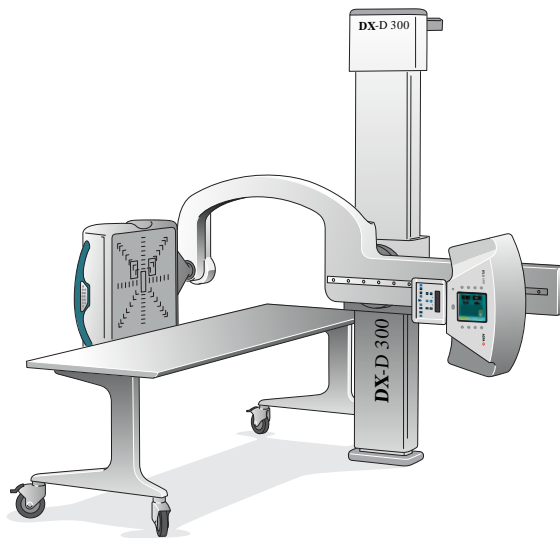


DX-D 300

8207/050

User Manual

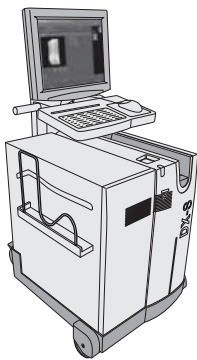

Contents

Legal Notice.....	5
Introduction to this Manual.....	5
Scope.....	6
About the safety notices in this document.....	7
Disclaimer.....	8
Introduction.....	8
Intended Use.....	9
Intended User.....	10
Configuration.....	11
Operation Controls.....	12
MUSICA Acquisition Workstation (NX).....	13
Software Console on the NX Workstation.....	14
DR Detector Switch.....	15
X-Ray generator control in the operator room.....	16
U-arm control panel.....	17
Control panel of the collimator.....	18
DR Detector handle control buttons.....	19
U-arm remote control.....	20
Portable DR detector.....	21
Emergency stop button.....	22
Emergency shutdown power switch.....	23
System Documentation.....	24
The DX-D 300 User Documentation.....	25
The Getting Started material.....	25
Options and Accessories.....	26
Anti-scatter grids.....	26
Product Complaints.....	27
Compliance.....	28
General.....	29
Safety.....	29
Electromagnetic Compatibility.....	29
X-Ray Safety.....	29
Classification.....	30
Connectivity.....	31
X-ray Exposure Parameters.....	31
Labels.....	32
Type label.....	32
Messages.....	34
Messages on the software console.....	34
Installation.....	35
Labeling the DR detectors.....	35
Cleaning and Disinfecting.....	36
Cleaning.....	37
Disinfecting.....	38
Approved disinfectants.....	39

Patient data security.....	40
Requirements on the operating environment.....	40
Maintenance.....	42
Environmental protection.....	43
Safety Directions.....	44
Operation.....	45
Starting DX-D 300.....	46
Performing an exposure using the DR Detector.....	47
Step 1: retrieve the patient info.....	48
Step 2: Select the exposure.....	49
Step 3: Prepare the exposure.....	50
Step 4: Check the exposure settings.....	51
Step 5: execute the exposure.....	52
Step 6: perform a quality control.....	52
Performing a Full Leg Full Spine examination.....	53
Performing an exposure using a CR cassette.....	54
Step 1: retrieve the patient info.....	55
Step 2: select the exposure.....	56
Step 3: Prepare the exposure.....	57
Step 4: Check the exposure settings.....	57
Step 5: Execute the exposure.....	58
Step 6: repeat steps 2 to 5 for the next subexposures.....	58
Step 7: digitize the image.....	58
Step 8: perform a quality control.....	58
Stopping the System.....	59
Operating the DX-D Software Console.....	59
Device Status Frame.....	60
Modality Position.....	61
Filter Status.....	62
Anti-scatter grid status.....	63
Position Status.....	64
Ready For Exposure.....	65
Preparation.....	66
X-Ray On.....	67
Heat Units.....	68
DAP Display Frame.....	69
Error Status Frame.....	70
Generator Controls Window.....	71
Radiographic Parameters.....	72
Focal Spot Indicator.....	73
X-Ray Tube Load.....	74
Automatic Exposure Control (AEC).....	75
Radiographic Working Modes.....	78
X-Ray Modality Controls Window.....	80
Positioning Parameters.....	81
Collimator Parameters.....	82
Problem solving.....	82
DR Detector is Exceeding the Maximum Working Temperature.....	83
DR detector must be recalibrated.....	83
DR Detector Problem.....	83

Radiographic Parameter Limits.....	84
Self-diagnosis Indicators.....	86


Technical Data..... 87

DX-D 300 technical data.....	88
Portable DR detector technical data (mounted fixed in the bucky).....	89
Fixed DR detector technical data.....	91
Fixed DR detector technical data.....	93
Fixed DR detector environmental conditions.....	94
U-Arm technical data.....	95
Portable DR detector technical data.....	96
Dose Area Product Meter (IBA DAP) Technical Data.....	97

Remarks for HF-emission and immunity..... 97

Immunity to RF wireless communication equipment.....	100
--	-----

Legal Notice

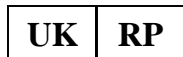

Agfa HealthCare UK Limited, 6-9 The Square, Stockley Park, Uxbridge, Middlesex UB11 1FW, UK



Agfa NV, Septestraat 27, 2640 Mortsel - Belgium

For more information on Agfa products, please visit medimg.agfa.com.

Agfa and the Agfa rhombus are trademarks of Agfa-Gevaert N.V., Belgium or its affiliates. NX and DX-D 300 are trademarks of Agfa NV, Belgium or one of its affiliates. All other trademarks are held by their respective owners and are used in an editorial fashion with no intention of infringement.

Agfa NV makes no warranties or representation, expressed or implied, with respect to the accuracy, completeness or usefulness of the information contained in this document and specifically disclaims warranties of suitability for any particular purpose. Products and services may not be available for your local area. Please contact your local sales representative for availability information. Agfa NV diligently strives to provide as accurate information as possible, but shall not be responsible for any typographical error. Agfa NV shall under no circumstances be liable for any damage arising from the use or inability to use any information, apparatus, method or process disclosed in this document. Agfa NV reserves the right to make changes to this document without prior notice. The original version of this document is in English.

Copyright 2023 Agfa NV

All rights reserved.

Published by Agfa NV

2640 Mortsel - Belgium.

No part of this document may be reproduced, copied, adapted or transmitted in any form or by any means without the written permission of Agfa NV

Introduction to this Manual

- [Scope](#) on page 6
- [About the safety notices in this document](#) on page 7
- [Disclaimer](#) on page 8

Scope

This User Manual describes the features of the DX-D 300 System, an integrated Digital Radiography X-Ray System to be used as medical diagnostic aid in General Radiography and emergency departments. It explains how the different components of the DX-D 300 System work together.

About the safety notices in this document

The following samples show how warnings, cautions, instructions and notes appear in this document. The text explains their intended use.



DANGER: A danger safety notice indicates a hazardous situation of direct, immediate danger for a potential serious injury to a user, service engineer, patient or any other person.



Warning: A warning safety notice indicates a hazardous situation which can lead to a potential serious injury to a user, service engineer, patient or any other person.



Caution: A caution safety notice indicates a hazardous situation which can lead to a potential minor injury to a user, service engineer, patient or any other person.



An instruction is a direction which, if it is not followed, can cause damage to the equipment described in this manual or any other equipment or goods and can cause environmental pollution.



A prohibition is a direction which, if it is not followed, can cause damage to the equipment described in this manual or any other equipment or goods and can cause environmental pollution.



Note Notes provide advice and highlight unusual points. A note is not intended as an instruction.

Disclaimer

Agfa assumes no liability for use of this document if any unauthorized changes to the content or format have been made.

Every care has been taken to ensure the accuracy of the information in this document. However, Agfa assumes no responsibility or liability for errors, inaccuracies or omissions that may appear in this document. To improve reliability, function or design Agfa reserves the right to change the product without further notice. This manual is provided without warranty of any kind, implied or expressed, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.



Note In the United States, Federal law restricts this device on order of a physician for prescription use only.

Introduction

- [Intended Use](#) on page 9
- [Intended User](#) on page 10
- [Configuration](#) on page 11
- [Operation Controls](#) on page 12
- [System Documentation](#) on page 24
- [Options and Accessories](#) on page 26
- [Product Complaints](#) on page 27
- [Compliance](#) on page 28
- [Connectivity](#) on page 31
- [Labels](#) on page 32
- [Messages](#) on page 34
- [Installation](#) on page 35
- [Cleaning and Disinfecting](#) on page 36
- [Patient data security](#) on page 40
- [Maintenance](#) on page 42
- [Environmental protection](#) on page 43
- [Safety Directions](#) on page 44

Intended Use

The DX-D 300 system is an integrated X-ray imaging system used in hospitals, clinics and medical practices by physicists, radiographers and radiologists to make, process and view static X-ray radiographic images of the skeleton (including skull, spinal column and extremities), chest, abdomen and other body parts on adult, pediatric or neonatal patients.

Applications can be performed with the patient in the sitting, standing or lying position.

This device is not intended for mammography applications.

Intended User

This manual has been written for trained users of Agfa products and trained diagnostic X-Ray clinical personnel who have received proper training.

Users are those persons who actually handle the equipment and those who have authority over the equipment.

Before attempting to work with this equipment, the user must read, understand, note and strictly observe all warnings, cautions and safety markings on the equipment.

Configuration

DX-D 300 is a DR X-ray system (Direct Radiography X-ray system) that can be combined with components from a CR X-ray system.

The complete DX-D 300 consists of the following components:

- Vertical column with U-arm
- X-Ray tube with automatic collimator with DAP (dose area product meter)
- Integrated 4343R DR Detector or DR Detector Bucky
- Portable DR Detector
- Mobile table
- X-Ray generator
- NX workstation

The DR Detector bucky has two variants. Depending on the configuration, the use of the DR Detector bucky is restricted to

- DR Detectors with a size equivalent to 14x17 inch (43x35 cm) and 43x35 CR Cassettes
- DR Detectors with a size equivalent to 17x17 inch (43x43 cm)

The DX-D 300 configuration with integrated DR Detector automatically detects the grid status. The DX-D 300 configuration with DR Detector Bucky does not detect the grid status.

DX-D 300 can be used in combination with:

- CR digitizer


DX-D 300 supports the Full Leg Full Spine application using the integrated 4343R DR Detector or a portable DR Detector in the DR Detector bucky.



1. NX workstation
2. X-Ray Generator Control
3. Mobile table
4. DR Detector
5. U-arm
6. Vertical column
7. X-Ray tube

Figure 1: DX-D 300 configuration with integrated DR Detector

Operation Controls

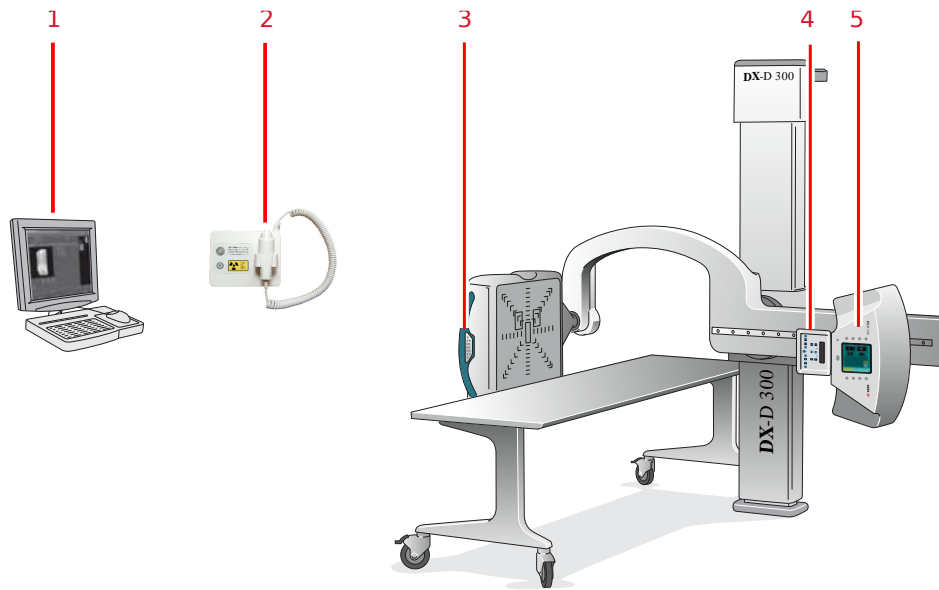

1. NX application and software console
2. X-Ray generator control box
3. DR Detector handle control buttons
4. Collimator control panel
5. U-arm control panel

Figure 2: DX-D 300 operation controls

- [MUSICA Acquisition Workstation \(NX\)](#) on page 13
- [Software Console on the NX Workstation](#) on page 14
- [DR Detector Switch](#) on page 15
- [X-Ray generator control in the operator room](#) on page 16
- [U-arm control panel](#) on page 17
- [Control panel of the collimator](#) on page 18
- [DR Detector handle control buttons](#) on page 19
- [U-arm remote control](#) on page 20
- [Portable DR detector](#) on page 21
- [Emergency stop button](#) on page 22
- [Emergency shutdown power switch](#) on page 23

MUSICA Acquisition Workstation (NX)

The MUSICA Acquisition workstation is used to define patient information, select exposures and process images.

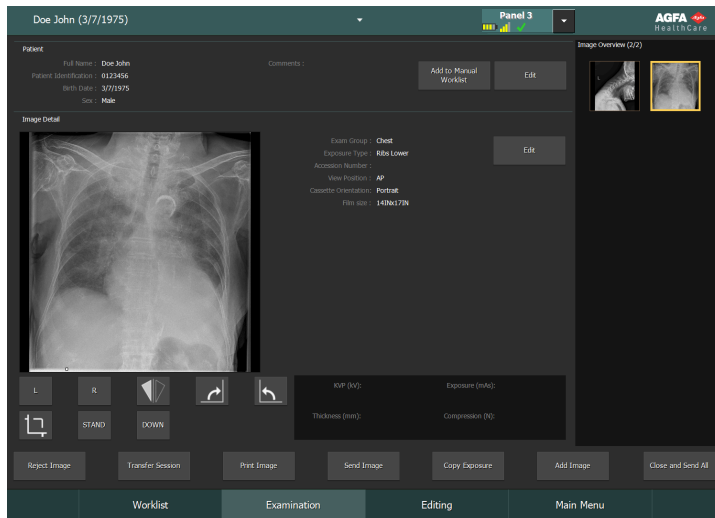



Figure 3: MUSICA Acquisition workstation software

The operation of the workstation application is described in the MUSICA Acquisition Workstation User Manual, document 4420.

The software is further referred to as "NX" and the PC on which it runs the "NX workstation".

Software Console on the NX Workstation

The Software Console is used to control X-Ray generator settings and X-Ray system position.

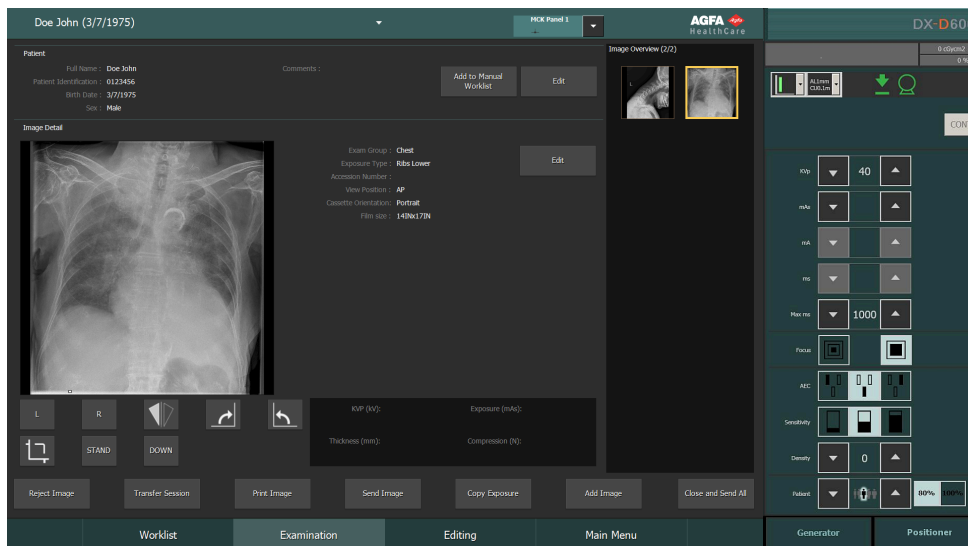

The software console has two screens:

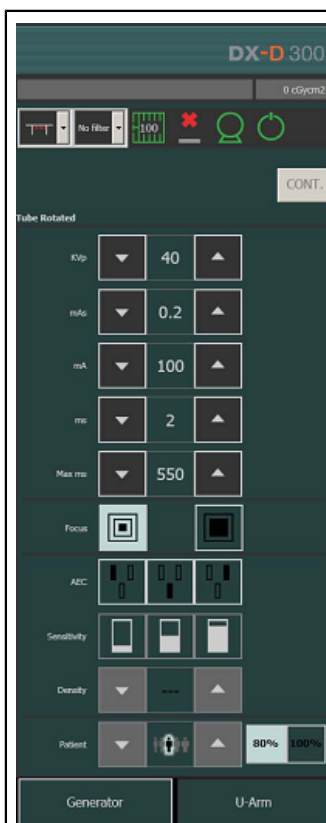


Figure 4: X-Ray generator controls



Figure 5: X-Ray system automatic positioning controls

DR Detector Switch

The **DR Detector Switch** is available in the title bar of the MUSICA Acquisition Workstation. The **DR Detector Switch** shows which DR detector is active and shows its status. The **DR Detector Switch** can be used to activate another DR detector.



Figure 6: DR Detector Switch

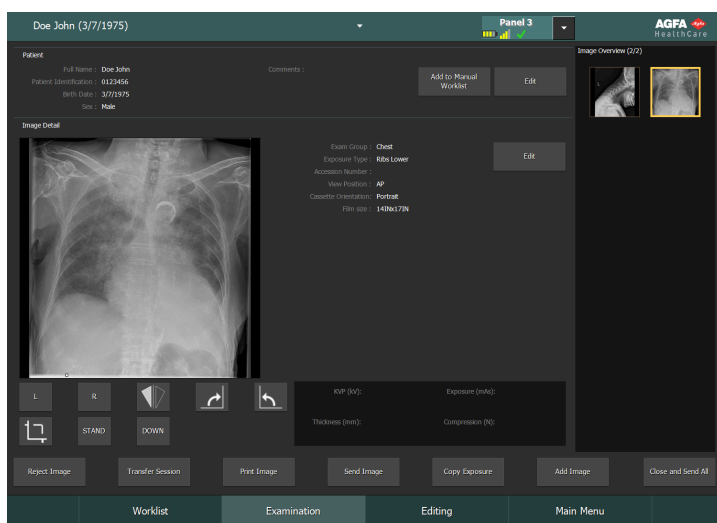


Figure 7: Title bar with DR Detector Switch

Battery status icon					(empty)
Meaning	Full	Medium	Low	Empty	Wired DR Detector Wireless DR detector is off or disconnected

Connection status icon (wifi/wired)					(empty)
Meaning	Good	Low	Bad	Wired DR detector	DR detector is off or disconnected

DR Detector status icon				(empty)
Meaning	DR detector is ready for exposure	DR detector is initializing for exposure	DR detector is off or disconnected or in error	DR detector is inactive (no thumbnail selected)

X-Ray generator control in the operator room

The X-Ray generator control box contains buttons to switch on and off the X-Ray generator and a handswitch to make exposures.



Figure 8: the X-Ray generator control box

Following warning is printed on the X-ray generator control box in English:



Warning: This x-ray unit may be dangerous to patient and operator unless safe exposure factors, operating instructions and maintenance schedules are observed.



This label is on the X-ray generator control box. If the system has just been stopped, wait at least 10 seconds before starting it again.

U-arm control panel

On the U-arm, the control panel with touch screen console and control buttons to control X-Ray generator settings and U-arm position.

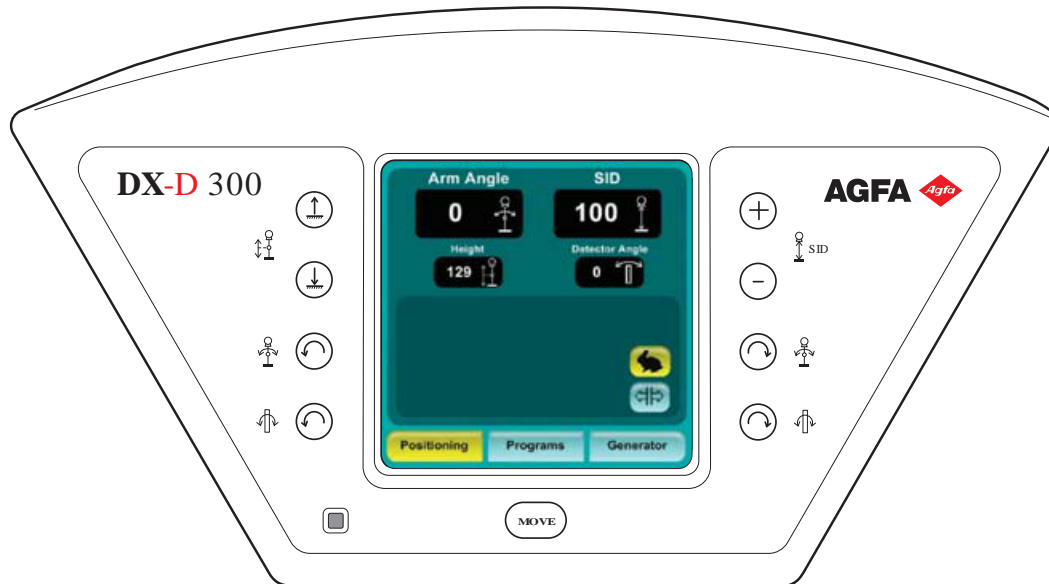


Figure 9: U-arm control panel

Control panel of the collimator

The control panel of the automatic collimator:

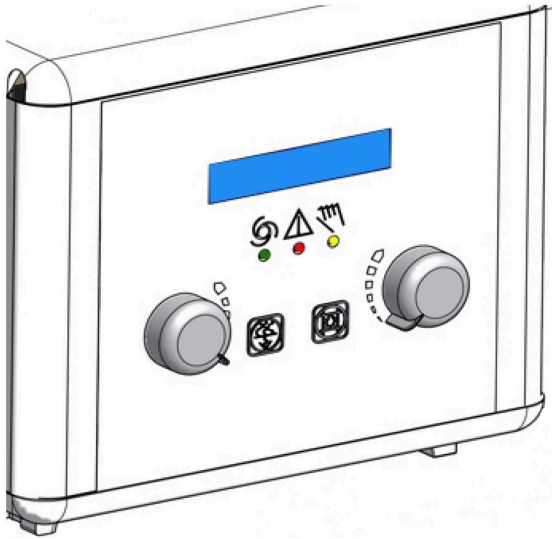


Figure 10: the control panel of the automatic collimator

The display shows the dimensions of the collimated area and of the source image distance (SID) in centimeters or in inches. Values in centimeter have no decimal point. Values in inches have one digit after the decimal point.

DR Detector handle control buttons

The DR Detector handle control buttons to control the U-arm position

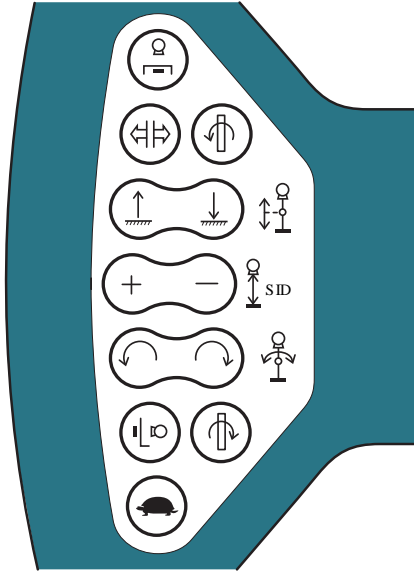


Figure 11: DR Detector handle control buttons

U-arm remote control

The remote control to control U-arm position

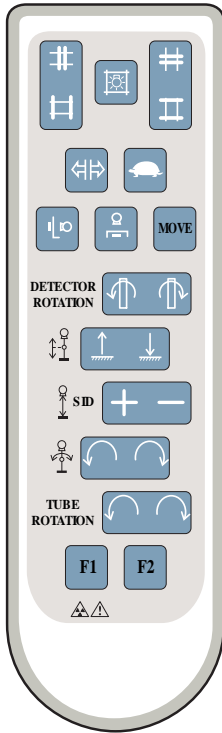




Figure 12: U-arm remote control

Portable DR detector

When performing an exposure, keep in mind the following detector orientation aids:

Table 1: Orientation aids

	Tube side icon, indicating the side that faces the X-ray tube
	Patient orientation marker, filled rectangle printed at the corner of the detector, for consistent orientation relative to the patient

For an overview of the operation controls of the DR detector, refer to the user manual of the DR detector.

The DR detector may come in contact with the patient.



Note DR detectors that operate wireless contain an RF transmitter. For detailed information, refer to the DR detector user manual.

Emergency stop button

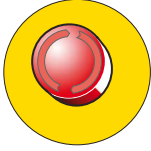


Figure 13: Emergency stop button

If a system malfunction causes an emergency situation involving the patient, operating personnel or any system component, activate the emergency stop. All motor driven system movements will be stopped.

For detailed information about the emergency button/switch, refer to the DX-D 300 U-arm User Manual (document 0174).

Emergency shutdown power switch

Use the emergency shutdown power switch, if a dangerous situation cannot be eliminated by pressing the emergency stop button.



Warning: Use the emergency shutdown power switch in case of danger to patients, operators, third parties, or one of the units. The entire system will be shut down and the power supply will be disconnected.

The emergency shutdown power switch for the room is typically located on the wall and easy to access, often close to the power off switch of the X-ray system. It is installed and labeled by customer.



Warning: It must be ensured that the emergency switches are always freely accessible.

System Documentation

The user documentation consists of:

- DX-D 300 User Documentation CD (digital media).
- NX User Documentation CD (digital media).
- User documentation for the supported DR Detectors
- Digitizer User Documentation CD (digital media).
- DX-D 300 Owner's Manual (paper binder).
- Getting Started material.

The documentation shall be kept with the system for easy reference.

The most extensive configuration is described within this manual, including the maximum number of options and accessories. Not every function, option or accessory described may have been purchased or licensed on a particular piece of equipment.

Technical documentation is available in the product service documentation which is available from your local support organization.

The most recent version of this document is available on <http://www.agfahealthcare.com/global/en/library/index.jsp>

- [The DX-D 300 User Documentation](#) on page 25
- [The Getting Started material](#) on page 25

The DX-D 300 User Documentation

- DX-D 300 User Manual (this document), document 0172.
- DX-D 300 U-arm User Manual, document 0174.
- DX-D Full Leg Full Spine User Manual, document 0179.
- DX-D DR Detector Calibration Key User Manual, document 0134.

The Getting Started material

- Getting Started with NX, document 4417.
- Getting started with DX-D 300, document 0170.

Options and Accessories

- DX-D Full Leg Full Spine Stand (for the DX-D Full Leg Full Spine application)
- CR FLFS Cassette Holder (for the CR Full Leg Full Spine application)

For options and accessories information, refer to the DX-D 300 U-arm User Manual, document 0174.

- [Anti-scatter grids](#) on page 26

Anti-scatter grids

Anti-scatter grids are used to reduce scattered radiation and improve image quality. Grids are available as an option.

Refer to the Agfa website for specifications on the anti-scatter grids that have been found compatible with the system and the DR Detectors.

<http://www.agfahealthcare.com/global/en/library/overview.jsp?ID=54332498>

Product Complaints

Any health care professional (for example a customer or a user) who has any complaints or has experienced any dissatisfaction with the quality, durability, reliability, safety, effectiveness, or performance of this product must notify Agfa.

For a patient/user/third party in the European Union and in countries with identical regulatory regimes (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Contact address:

Agfa Service Support - local support addresses and phone numbers are listed on www.agfa.com

Agfa - Septestraat 27, 2640 Mortsel, Belgium

Agfa - Fax +32 3 444 7094

Compliance

- [General](#) on page 29
- [Safety](#) on page 29
- [Electromagnetic Compatibility](#) on page 29
- [X-Ray Safety](#) on page 29
- [Classification](#) on page 30

General

- The product has been designed in accordance with Regulation (EU) 2017/745 on medical devices (MDR)

Safety

- IEC 60601-1

Electromagnetic Compatibility

- IEC 60601-1-2


X-Ray Safety

- IEC 60601-1-3
- IEC 60601-2-54
- IEC 60601-2-7

For USA

The system conforms to DHHS radiation Standards of 21CFR subchapter J as of the date of manufacture.

Classification

Type of protection against electric shock	Class 1 Equipment
Degree of protection against electric shock	Type B Applied Part 
Degree of protection against ingress of liquids	IPX0 as defined in IEC60529. Ordinary equipment (enclosed equipment without protection against ingress of liquids).
Methods of disinfection recommended by the manufacturer	Disinfectable equipment (or elements)
Degree of safety of application in the presence of flammable anesthetic mixture with air or with oxygen or with nitrous oxide	Equipment for use in environments where no flammable gases or vapors are present
Mode of operation	Suitable for continuous operation
Labelling	<ul style="list-style-type: none"> • CE label: Regulation 2017/745 (for European Union), EN 60601-1 • CUL label: CSA 22.2 No 601.1 (Canada)
Remarks for HF-emission and immunity	This equipment generates, uses and can radiate radio frequency (RF) energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. In any circumstance; however, there is no guarantee that interference will not occur in a particular installation.

Connectivity

The NX workstation is connected to the X-ray system to exchange X-ray exposure parameters.

The NX workstation requires a 100 Mbit ethernet network to exchange information with a number of other devices.

The NX workstation communicates with other devices in the hospital network using one of the following protocols:

- DICOM
- IHE

The NX workstation can be connected to a RIS system (input scheduling), a PACS system (output image/data management) and to a hardcopy device (output image).



Note The data connections between the components of the system are separate from the hospital network and should not be disconnected or modified.

- [X-ray Exposure Parameters](#) on page 31

X-ray Exposure Parameters

The X-Ray Exposure Parameters and DAP value can be configured to be

- displayed in the NX Image Detail pane,
- printed in the film text box,
- transmitted to the Archive,
- transmitted to the RIS via MPPS (Modality Performed Procedure Step).









The NX Image Detail pane displays the X-Ray Exposure Parameters and DAP value for the individual sub-exposures.

Only the cumulative DAP value is transmitted to the Archive.




Caution: Incomplete exposure parameters (kV, mAs) are transmitted to Archive for multiple sub-exposures on one cassette. Only the exposure parameters for one sub-exposure are transmitted. Don't use multiple sub-exposures when the exposure parameters are interpreted by the Archive.


Labels

Lable	Meaning
	This mark shows compliance of the equipment with Regulation 2017/745 (for European Union).
	Type B applied part
	Serial number
	Manufacturer
	Date of manufacture
	Dangerous voltage
	Ionizing radiation
	The INMETRO label is positioned close to the type label.

- [Type label](#) on page 32

Type label

Mark	Meaning
 <p>(Sample of subtype 8207/050)</p>	<p>The type label is located near to the top of the vertical column.</p> <p>The type label information for each combination of X-ray tube and X-ray generator is available in the technical data.</p>

Mark	Meaning
 Agfa NV Sepstersraat 27, 2640 Mortsel, Belgium This product complies with the DHHS requirements of 21 CFR subchapter J as of the date of manufacture Made in Belgium Mortsel ☞ Month - YYYY	The 21 CFR Subchapter J label is positioned close to the type label.

Messages

Messages are displayed on the NX workstation monitor, on the touch screen console in the operator room or on the control panel.

Special messages are displayed in a dialog box in the middle of the screen or in a fixed part of the screen. This message will tell that either a problem has occurred or that a requested action cannot be performed.

The user must read these messages carefully. They will provide information on what to do from then on. This will be either performing an action to resolve the problem or to contact the Agfa service organization.

Details on the contents of messages can be found in the service documentation which is available to Agfa service personnel.

- [Messages on the software console](#) on page 34

Messages on the software console

Error codes indicate the potential cause of a system failure. Error codes are shown on the software console. Correct the cause of the error and press the “CONT.” button until its indication disappears.

All these error codes will enable the operator to indirectly convey the possible source of error to service personnel. This may prevent the need for a service call or enable service personnel to anticipate corrective actions prior to arriving on site.



Figure 14: Example of error code

Installation

The NX Workstation complies with the IEC 60950 and IEC 62368-1 standards for Information Technology. This means that, although it is absolutely safe, patients may not come into direct contact with the equipment. Therefore, the workstation must be placed outside a radius of 1.5 m (IEC/EN 60601-1) or 1.83 m (ANSI/AAMI ES60601-1) around the patient (according to the local valid regulation).

The other components of the DX-D 300 are suitable for use within the patient environment.

- [Labeling the DR detectors](#) on page 35

Labeling the DR detectors



Caution: Selecting the wrong DR detector can cause the need to retake the image.

On a configuration with multiple wireless DR detectors of the same type, it is required to apply labeling to the DR detector containing a unique nickname for each DR detector. The nicknames must be configured on the MUSICA Acquisition Workstation. The **DR detector switch** shows which DR detector is active and shows its status, by means of the nickname of the DR detector.

Cleaning and Disinfecting

All appropriate policies and procedures should be followed to avoid contamination of the staff, patients and equipment. All existing universal precautions should be extended to avoid potential contaminations and to avoid patients coming into (close) contact with the device. The user is responsible for selecting a disinfection procedure.

- [Cleaning](#) on page 37
- [Disinfecting](#) on page 38
- [Approved disinfectants](#) on page 39

Cleaning

To clean the exterior of the equipment:

1. Stop the system.



Caution: Wet cleaning of the equipment while it is connected to the electric circuit includes the risk of electric shock and of short circuit.

2. Wipe the exterior of the device with a clean, soft, damp cloth. Use a mild soap or detergent if required. Do not use any corrosive, dissolving or abrasive cleaning or polishing agents. Make sure no liquid gets in the device.



Caution: Clean the equipment with only a little moisture.



Note Do not open the equipment for cleaning. No components inside the device require cleaning by the user.

Using unsuitable cleaning agents or methods can damage the property when surface becomes dull and brittle (e.g. alcohol-containing agents).

3. Start up the system.

Disinfecting



Warning: To disinfect the device, use only disinfectants and disinfection methods that are approved by Agfa and that correspond to the national regulation and guidelines as well as explosion protection.

If you plan to use other disinfectants, approval of Agfa is needed before use, as most disinfectants can damage the device. UV disinfection is also not allowed.

Perform the procedure following the instructions for use, the disposal instructions and the safety instructions of the selected disinfectants and tools and of the hospital.

Items contaminated with blood or body fluids, which may contain blood-borne pathogens, should be cleaned and then receive intermediate level disinfection with a product having an EPA-registered claim for activity against hepatitis B.

Approved disinfectants

Refer to the Agfa website for specifications on the disinfectants that have been found compatible with the cover material of the device and can be used on the outer surface of the device.

<http://www.agfahealthcare.com/global/en/library/overview.jsp?ID=41651138>

Patient data security

The user must ensure that the patients' legal requirements are met and that the security of the patient data is guarded.

The user must define who can access patient data in which situations.

The user must have a strategy available on what to do with patient data in case of a disaster.

- [Requirements on the operating environment](#) on page 40

Requirements on the operating environment

These operating environment requirements for information security and privacy (ISP), set in compliance with point 17(4) and 18(8) of Annex I of the EU Medical Device Regulation 2017/745, must be implemented and used in connection with the use of the Agfa medical device by the Customer (User). These are minimum requirements and designed to protect against unauthorised access that could hamper the device from functioning as intended.

Although Agfa has defined these ISP Operating Environment Requirements for implementation by the Customer, Agfa makes no warranties, expressed or implied regarding those ISP Operating Environment Requirements.

Agfa disclaims all liability if a security incident would occur despite the implementation of these ISP Operating Environment Requirements by the Customer.

Agfa reserves the right to revise these ISP Operating Environment Requirements and to make changes to them at any time. Possible revisions of the ISP Operating Environment Requirements will only be available in an electronic form, on request, via our website, by using the user documentation request form <http://www.agfahealthcare.com/global/en/library/index.jsp>.

The information presented herein is sensitive and is company confidential. Without written authority from Agfa, further distribution outside the company is not allowed.

- Perimeter firewalls shall be in place and appropriately configured in order to ensure that communications between medical devices and external resources are either denied or restricted to just the communications that are essential for the medical devices to properly function.
- Network Intrusion Detection/Prevention Systems (NIDS/NIPS) shall be in place at the perimeter and appropriately configured, in order to provide early warning of an attack attempt or successful compromise of a medical device as well as to attempt to prevent compromise of medical devices.
- A Network Time Protocol Server shall be configured in the medical devices in order to synchronize the time in the audit logs with the time on the NTP server.
- Medical devices shall be on an isolated network segment that restricts communication of the medical devices to the systems that are required for the device to function.
- Internal firewalls shall be put in place to improve upon network segmentation and to further restrict communications of medical devices to the systems (internal and external) that they need to interact with.
- Medical device configurations shall be backed up in a secure separate device.
- Security controls shall be put in place to ensure that physical access to medical devices is limited only to authorized individuals and that physical theft of the device is prohibited.
- An incident response plan detailing responsibilities and how to react and recover from incidents, shall be in place. Staff involved in the incident response plan shall be trained to respond appropriately and effectively.
- A formal user provisioning and de-provisioning process shall be implemented to enable the appropriate management of access rights to medical devices.
- Users shall be assigned unique accounts to medical devices.

- User access rights to medical devices shall be reviewed for appropriateness and corrected as needed, at regular intervals not exceeding once a year.

Maintenance

Maintenance procedures are described in the DX-D 300 Owner's Manual.

Environmental protection



Figure 15: WEEE symbol

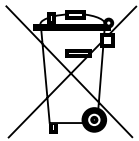


Figure 16: Battery symbol

WEEE end user notice

The directive on Waste Electrical and Electronic Equipment (WEEE) aims to prevent the generation of electric and electronic waste and to promote the reuse, recycling and other forms of recovery. It therefore requires the collection of WEEE, recovery and reuse or recycling.

Due to the implementation into national law, specific requirements can be different within the European Member States. The WEEE symbol on the products, and/or accompanying documents means that used electrical and electronic products should not be treated as, or mixed with general household waste. For more detailed information about take-back and recycling of this product please contact your local service organization and/or dealer. The recycling of materials will help to conserve natural resources.




















Caution: By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

Battery notice

The battery symbol on the products, and/or accompanying documents means that the used batteries should not be treated as, or mixed with general household waste. The battery symbol on batteries or its packaging may be used in combination with a chemical symbol. In cases where a chemical symbol is available it indicates the presence of respective chemical substances. If your equipment or replaced spare parts contain batteries or accumulators please dispose of them separately according to local regulations.

For battery replacements please contact your local sales organization.

Safety Directions

-  **Warning:** Only qualified and authorized personnel shall operate this system. In this context 'qualified' means those persons legally permitted to operate this equipment in the jurisdiction in which the equipment is being used, and 'authorized' means those persons authorized by the authority controlling the use of the equipment. Full use must be made of all radiation protection features, devices, systems, procedures and accessories.
-  **Warning:** Improper changes, additions, maintenance or repair of the equipment or the software can lead to personal injury, electrical shock and damage to the equipment. Safety is only guaranteed when changes, additions, maintenance or repairs are carried out by an Agfa certified field service engineer. A non certified engineer performing a modification or service intervention on a medical device, acts on his own responsibility and makes the warranty void.
-  **Warning:** Do not connect the equipment with anything other than specified. Doing so may result in fire or electric shock.
-  **Warning:** Do not connect additional extensions cords or multiple power socket outlets to the system.
-  **Warning:** Safety is only guaranteed when an Agfa certified field service engineer has installed the product.
-  **Warning:** There is a risk of collision or crushing for patients, operating staff, unit and objects, caused by unit movements which could be released by inadmissible actuation of operating elements by patients.
-  **Warning:** Ionizing radiation can lead to radiation injuries if handled incorrectly. When radiation is applied, the required protective measures must be complied with.
-  **Warning:** The operator must take precautions to protect himself against dangerous X-ray exposure when using the DR detector in the X-ray beam path of an X-ray source.
-  **Caution:** The DR Detector or CR cassette are not intended to be used as a primary barrier to X-rays. The user is responsible for ensuring the safety of the operator, bystanders, and the subjects being radiographed.
-  **Caution:** Strictly observe all warnings, cautions, notes and safety markings within this document and on the product.
-  **Caution:** All Agfa medical products must be used by trained and qualified personnel.
-  **Caution:** Avoid unnecessary dose by checking before exposure if the DR Detector Switch displays the name of the DR Detector that is being used and if the status of the DR Detector is ready for exposure.
-  **Caution:** System unavailability due to hardware or software failure. If the product is used in critical clinical workflows, a backup system has to be foreseen.
-  **Caution:** Before moving the U-arm out of horizontal position, check that no objects are lying on the bucky that can fall off.
-  **Caution:** Damaged grid. Reduced image quality. Please handle the grids with special care.
-  **Caution:** When inserting the anti-scatter grids, it is essential that the grid corresponds to the intended source-image-distance (SID) to which the grid is focussed. Because of the focussing of the grids, the tube head unit must be centered onto the bucky.
-  **Caution:** Excessive ambient temperature may impact performance of DR Detectors and cause permanent damage to the equipment. Refer to the related user manual for environ-

mental conditions for the DR detector. If ambient temperature and humidity is outside the specified range, do not operate the system or use air conditioning. Frost due to low temperatures can damage internal circuits. Warranty will be void if it is obvious that operating conditions are not met.



Caution: Power failure can cause an image being lost.

Connect the workstation and the digitizer to an uninterrupted power supply (UPS) or an institutional standby generator.

Operation

- [Starting DX-D 300](#) on page 46
- [Performing an exposure using the DR Detector](#) on page 47
- [Performing a Full Leg Full Spine examination](#) on page 53
- [Performing an exposure using a CR cassette](#) on page 54
- [Stopping the System](#) on page 59

Starting DX-D 300



Note Allow the DR Detector to warm up before the DX-D 300 is used for clinical purposes. The warming-up time starts as soon as the DR Detector has been powered on and the NX workstation is running. To check if a warming-up time is required, refer to the DR Detector User Manual.

1. Switch on the X-Ray generator control in the operator room.

The X-Ray generator and the wired DR Detector are powered on.

2. Switch on the U-arm unit using the button on the U-arm control box in the examination room.

The U-arm unit and the touch screen console are powered on.

3. Start NX.

The NX application and the software console are available on the NX workstation.

For detailed information about starting up NX, refer to the NX User Manual, document 4420.

4. In a configuration with a wireless DR Detector, power on the DR Detector:

- attach a fully charged battery pack to the DR Detector.
- turn on the DR Detector.
- if needed, register the DR Detector to the NX workstation.

For detailed information about starting up the DR Detector, refer to the DR Detector User Manual.

Performing an exposure using the DR Detector

- [Step 1: retrieve the patient info](#) on page 48
- [Step 2: Select the exposure](#) on page 49
- [Step 3: Prepare the exposure](#) on page 50
- [Step 4: Check the exposure settings](#) on page 51
- [Step 5: execute the exposure](#) on page 52
- [Step 6: perform a quality control](#) on page 52

Step 1: retrieve the patient info

At the MUSICA Acquisition Workstation:


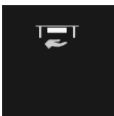
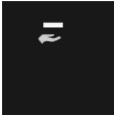
1. When a new patient comes in, define the patient info for the exam.
2. Start the exam.

If the workstation is connected to a second monitor that is positioned outside the operator room, make sure that the patient data is not exposed to unauthorized persons.

Step 2: Select the exposure

In the operator room at the NX workstation:

Select the thumbnail for the exposure in the Image Overview pane of the Examination window.

 <p>Lower Extremiti... Ankle Stress Lat</p>	U-arm with integrated DR Detector
	Portable DR Detector in the DR bucky
	Free exposure using a portable DR Detector

The selected DR Detector is activated. The DR Detector Switch shows which DR Detector is active and shows its status.

The default X-Ray exposure parameters for the selected exposure are sent to the modality and displayed on the software console.

The default U-arm position for the selected exposure is sent to the modality and displayed on the software console, for automatic positioning of the U-arm.

Step 3: Prepare the exposure

1. In the examination room, position the U-arm:
Press and hold the MOVE button on the U-arm control panel or remote control.
The U-arm moves to the default position for the selected exposure.
2. When using a portable DR Detector, position the DR Detector for the exposure.
The DR Detector Switch shows which DR Detector is active and shows its status.
3. Position the patient:
 - a) Position the patient.
 - b) Check if the X-Ray system position is suitable for the exposure.
 - c) Make final adjustments to the position of the U-arm using the control buttons on the control panel or remote control.
 - d) Switch on the light localizer on the collimator. Adapt collimation if required.
 - e) Apply radiation protective measures for the patient if needed.



Warning: Monitor the patient position (hands, feet, fingers, etc.) with special care to avoid injury to the patient caused by unit movements. Patient hands must be kept away from mobile components of the unit. Intravenous tubing, catheters and other patient connected lines should be routed away from moving equipment.



Warning: Liquids ingressing the DR Detector may cause malfunction and contamination.

If there is a chance that the detector comes in contact with liquids (bodily fluids, disinfectants,...), the DR Detector must be wrapped in a protective plastic bag while performing the examination.



Caution: Use the smallest X-ray field collimation. Make sure that the area of interest will be completely exposed and the X-ray field does not exceed the area of interest. The collimator automatically limits the collimated area to the size of the detector, unless it is unlocked to manual mode using the key on the back.

Step 4: Check the exposure settings

Related information

[DR Detector Switch](#) on page 15

On the NX application:

1. Check if the DR Detector Switch displays the name of the DR Detector that's being used
2. If a wrong DR Detector is displayed, select the right DR Detector by clicking the drop down arrow on the DR Detector Switch.
3. Check if the status of the DR Detector is ready for exposure.

On a DR Detector that has a status indicator:

Check if the status of the DR Detector is ready for exposure. If the status is not ready for exposure, the DR Detector cannot be used for making an exposure.

In the operator room at the X-ray generator console:

1. Check if the exposure settings displayed on the console are suitable for the exposure.
2. If other exposure values are required than those defined in the NX exam, use the console to overwrite the default defined exposure settings.

Step 5: execute the exposure

In the operator room:

Press the exposure button to execute the exposure.



Make sure the generator is ready for exposure before you press the exposure button.



Warning: The radiation indicator on the control console lights up during exposure release.



Warning: Do not select another thumbnail until the preview image is visible in the active thumbnail.

In the operator room at the NX workstation:

- The image is acquired from the DR detector and displayed in the thumbnail.
- The actual X-Ray exposure parameters are sent back from the generator to the NX workstation and are shown in the Image Detail pane.
- If collimation is applied, the image is automatically cropped at the collimation borders.

Step 6: perform a quality control

At the MUSICA Acquisition Workstation:

1. Select the image on which quality control is to be performed.
2. Prepare the image for diagnosis by using e.g. L/R markers or annotations.
3. If the image is OK, send the image to a hardcopy printer and/or PACS (Picture Archiving and Communication System).

Performing a Full Leg Full Spine examination

Refer to the DR Full Leg Full Spine User Manual (document 0179).

The availability of DR Full Leg Full Spine depends on the configuration of the system.

Refer to the CR Full Leg Full Spine User Manual (document 4408, part of the NX User Documentation).

Performing an exposure using a CR cassette



Note Using an ID Tablet to identify cassettes before the exposure will break the communication of X-ray parameters between the NX workstation and the X-ray generator console. It is advised to identify cassettes after the exposure, as described in this workflow.

- [Step 1: retrieve the patient info](#) on page 48
- [Step 2: select the exposure](#) on page 56
- [Step 3: Prepare the exposure](#) on page 57
- [Step 4: Check the exposure settings](#) on page 57
- [Step 5: Execute the exposure](#) on page 58
- [Step 6: repeat steps 2 to 5 for the next subexposures](#) on page 58
- [Step 7: digitize the image](#) on page 58
- [Step 8: perform a quality control](#) on page 58

Step 1: retrieve the patient info

At the MUSICA Acquisition Workstation:


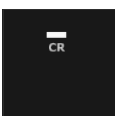
1. When a new patient comes in, define the patient info for the exam.
2. Start the exam.


If the workstation is connected to a second monitor that is positioned outside the operator room, make sure that the patient data is not exposed to unauthorized persons.

Step 2: select the exposure

In the operator room at the NX workstation:

1. Select the thumbnail for the exposure in the Image Overview pane of the Examination window.


	Cassette in the DR bucky
	Free exposure using a cassette

 **Note** For a bucky exposure, only cassette size 43x35 is supported.


The default X-Ray exposure parameters for the selected exposure are sent to the modality and displayed on the software console.

The default U-arm position for the selected exposure is sent to the modality and displayed on the software console, for automatic positioning of the U-arm.

2. Select the subexposure if more than one image is required for the same cassette.
If an image thumbnail is configured for multiple exposures on a single cassette, another set of thumbnails is shown in the image detail pane. Now you have to select one of these thumbnails to send the proper default X-Ray exposure parameters to the modality for each exposure.

 **Note** When working in a PACS environment, the preferred workflow is to have only one image per cassette. This is needed for optimal use of hanging protocols. However, in particular cases (e.g. printing sites) it is supported to make more than one exposure per cassette.

Step 3: Prepare the exposure

1. In the examination room, position the U-arm:
 - Press and hold the MOVE button on the U-arm control panel or remote control.
 - The U-arm moves to the default position for the selected exposure.
2. Position the patient:
 - a) Position the cassette.
 -  **Note** For a free exposure, partial lead covering of the cassette may be required if multiple images are taken on one cassette.
 - b) Position the patient.
 - c) Check if the X-Ray system position is suitable for the exposure.
 - d) Make final adjustments to the position of the U-arm using the control buttons on the control panel or remote control.
 - e) Set the correct distance between cassette and X-Ray tube.
 - f) Switch on the light localizer on the collimator. Adapt collimation if required.
 - g) Apply radiation protective measures for the patient if needed.



Warning: Monitor the patient position (hands, feet, fingers, etc.) with special care to avoid injury to the patient caused by unit movements. Patient hands must be kept away from mobile components of the unit. Intravenous tubing, catheters and other patient connected lines should be routed away from moving equipment.



Caution: Use the smallest X-ray field collimation. Make sure that the area of interest will be completely exposed and the X-ray field does not exceed the area of interest. The collimator automatically limits the collimated area to the size of the detector, unless it is unlocked to manual mode using the key on the back.

Step 4: Check the exposure settings

In the operator room at the software console, or in the examination room at the touch screen console:

1. Check if the exposure settings displayed on the console are suitable for the exposure.
2. If other exposure values are required than those defined in the NX exam, use the console to overwrite the default defined exposure settings.

Step 5: Execute the exposure

In the operator room:

Press the exposure button to execute the exposure.



Warning: The radiation indicator on the control console lights up during exposure release.

In the operator room at the NX workstation:

- The actual X-Ray exposure parameters are sent back from the console to the NX workstation and are shown in the Image Detail pane.
- The actual X-Ray exposure parameters and the Exposure Index (EI) value on the NX workstation can be used to monitor the performance of the Automatic Exposure Control of the X-Ray system.
- A green OK mark appears on all thumbnails for which the exposures are made and for which exposure settings are sent back to the NX workstation.

Step 6: repeat steps 2 to 5 for the next subexposures

Step 7: digitize the image

In the examination room:

Take the exposed cassette.

In the operator room:

1. Insert the cassette in the digitizer.
2. Click ID in the examination window of NX.



Note You can also use an ID Tablet to identify the cassette and digitize it using any digitizer.

The image will appear in the image overview pane of the examination window.

Step 8: perform a quality control

In the operator room at the NX workstation:

1. Select the image on which quality control is to be performed.
2. Prepare the image for diagnosis by using e.g. L/R markers or annotations.
3. If the image is OK, send the image to a hardcopy printer and/or PACS (Picture Archiving and Communication System).

Stopping the System

To stop the system:

1. Stop the MUSICA Acquisition workstation.

The MUSICA Acquisition workstation can be stopped in two ways, either by logging out of Windows or without logging out of Windows.

For detailed information, refer to the MUSICA Acquisition workstation User Manual, document 4420.



Note Stopping the MUSICA Acquisition workstation does not stop the DR detector. If the power of the DR Detector remains on, no warming-up time will be needed after starting the MUSICA Acquisition workstation.

2. Switch off the U-arm unit using the button on the U-arm control box in the examination room. The green light goes out.
3. Press the Power OFF button on the X-ray generator control box to switch off the generator. The green light goes out.
4. In a configuration with a wireless DR detector, power off the DR detector:
 - turn off the DR detector.
 - remove the battery pack.



Note If the DR detector is powered down, a warming-up may be required on the next start-up.



Warning: If the system has just been stopped, wait at least 10 seconds before starting it again.

Operating the DX-D Software Console

- [Device Status Frame](#) on page 60
- [DAP Display Frame](#) on page 69
- [Error Status Frame](#) on page 70
- [Generator Controls Window](#) on page 71
- [X-Ray Modality Controls Window](#) on page 80

Device Status Frame



Figure 17: Device status frame







- [Modality Position](#) on page 61
- [Filter Status](#) on page 62
- [Anti-scatter grid status](#) on page 63
- [Position Status](#) on page 64
- [Ready For Exposure](#) on page 65
- [Preparation](#) on page 66
- [X-Ray On](#) on page 67
- [Heat Units](#) on page 68

Modality Position

The modality position is automatically selected, based on the selected exposure.

To modify the position on the modality where the exposure will be made, click the drop-down arrow and select the modality position from the list.

Table 2: Modality Position

Icon	Description
	The image is planned for the radiographic table using the catapult bucky.
	The image is planned as a free exposure using CR.
	The image is planned for the radiographic table using the fixed DR detector.
	The image is planned for the Portable DR detector inserted in the radiographic table bucky.
	The image is planned as a free exposure using the Portable DR detector.
	The image is planned as a free exposure.




The type and configuration of the X-ray system defines which modality positions are available.



Note The modality position of a U-Arm X-ray system is represented by the icon for a radiographic table.

The modality position icon indicates the status of the DR detector.

Table 3: DR detector status

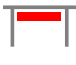Icon	Status description
	Grey: the image is planned and the DR detector is in sleep mode.
	Green: the DR detector is ready to acquire the exposure on the selected acquisition system. Green flashing: the exposure has been performed and the acquisition is ongoing.
	Red: the DR detector is out of order. Red flashing: the selected acquisition system is starting up.

The available workstations depend on the modality type and configuration.

Filter Status


On systems with automatic filtering, the filter is automatically set, based on the selected exposure. The filter setting can be modified on the software console or on the collimator.

- on the software console, click the filter status drop-down arrow and select the filter from the list.
- on the collimator, use the filter button

Table 4: Collimator with automatic filter

(no icon)	No filter is used.
0.1 mm Cu 1 mm Al	A filter is used. Material and thickness of the filter are specified.




Table 5: Collimator with manual filter

(no icon)	No filter is required.
	A filter is required. Insert the filter manually.

Anti-scatter grid status


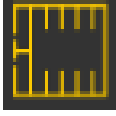
- The grid status is automatically detected.

Table 6: Grid status - automatically detected

	Empty: no grid is required.
	Green: the correct grid is inserted. The grid type is specified.
	Red: a grid is required. The correct grid is not inserted or inserted wrongly.

- The grid status is not automatically detected.




Table 7: Grid status - not automatically detected

	Empty: no grid is required.
	Orange: a grid is required.

Position Status

The position status is available on systems with automatic positioning.



Table 8: Position status

Icon	Description
	On target. The position of the X-ray system matches the target position.
	Not on target. The position of the X-ray system does not match the target position.
	Moving. The X-ray system is moving towards the target position.

Ready For Exposure

An icon indicates whether the X-ray system is ready for taking the exposure.

Table 9: Exposure ready

Icon	Description
 A green outline icon of an X-ray tube head, indicating that the system is ready for exposure.	Exposure ready. Indicates that the selected technique is properly set and there are no interlock failures or system faults.
 A red outline icon of an X-ray tube head with a white asterisk in the bottom right corner, indicating that the system is not ready for exposure.	Exposure not ready.

Preparation



Figure 18: Prep

Press the handswitch halfway (“Prep” position) to prepare the X-ray tube for exposure. The indicator will light up when the X-ray tube is prepared and there are no interlock failures or system faults.

After pressing this push-button, the following functions are activated:

- Anode rotation.
- Filament current switches from stand-by to the selected mA.

X-Ray On



Figure 19: X-ray on

After pressing the handswitch completely, the X-ray exposure is made. The indicator on the console will light up.

Heat Units

The status of the heat units is displayed below the X-ray icon.

During exposures, the heat units are calculated and totalled. The heat units display shows the percentage of the thermal capacity of the X-ray tube that is used. For example, a display of “0%” would indicate that all the heat units capacity of the X-ray tube remains.

DAP Display Frame

The Dose Display shows the radiation value of the last exposure. The radiation measure is read as DAP value (Dose Area Product) in $\text{cGy}\cdot\text{cm}^2$ (for example: 12.22 $\text{cGy}\cdot\text{cm}^2$).

Selecting a new exposure resets the DAP value.



Figure 20: DAP display frame

Error Status Frame



1. Button to confirm errors


The Error Status frame displays warnings, error numbers and error descriptions. If the error has been resolved, the “CONT.” button becomes active. Click the active button to reset the error condition.

A click on the text in the Error Status frame produces a pop-up message that includes the whole text.



Figure 21: Error status frame and pop-up message

Generator Controls Window

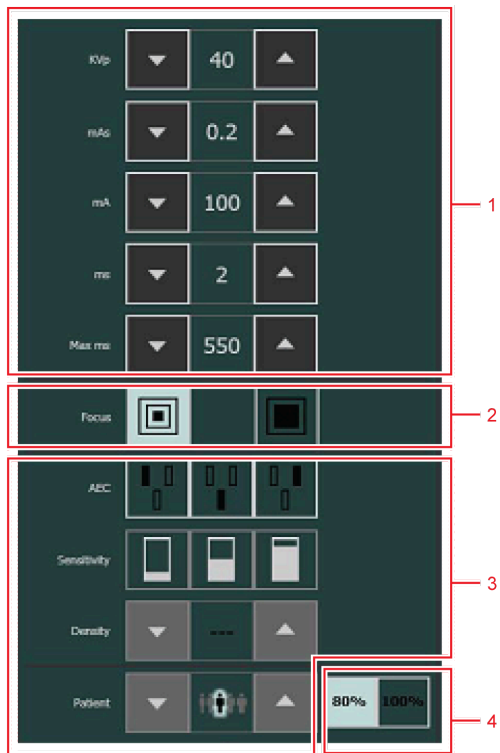

1. Radiographic parameters
2. Focal spot indicator
3. AEC buttons
4. X-ray tube load

Figure 22: Generator controls

- [Radiographic Parameters](#) on page 72
- [Focal Spot Indicator](#) on page 73
- [X-Ray Tube Load](#) on page 74
- [Automatic Exposure Control \(AEC\)](#) on page 75
- [Radiographic Working Modes](#) on page 78

Radiographic Parameters

You can set up following radiographic parameters:

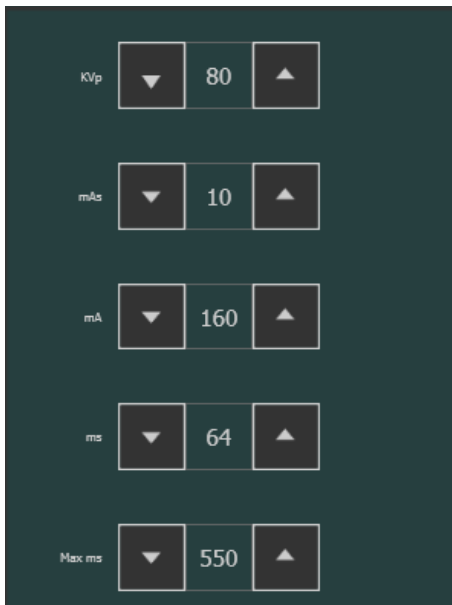


Figure 23: Radiographic parameters

To increase the radiographic parameters step by step, use the UP and DOWN arrows. The values increase or decrease step by step each time the corresponding button is touched.

- **kVp**: shows the radiographic kVp value (X-ray tube voltage) selected for the parameters.
- **mAs** can show:
 - The radiographic mAs value selected for the parameter.
 - The max mAs for an exposure using AEC. Based upon the max mAs and the mA setting, the generator must limit its maximum exposure time. X-rays outside these limits are NOT allowed. The highest allowed setting for max mAs depends on the mA.
 - When an exposure is made, it shows the actual mAs at the end of the exposure.
- **mA**: shows the radiographic mA value (current) selected for the parameter. The focal spot selection is changed automatically if the mA value exceeds the applicable range.
- **ms** can show:
 - The time value (in milliseconds) selected for the parameter.
 - When an exposure is made, it shows the actual time at the end of the exposure.
- **Max ms** shows: the maximum exposure time allowed with DR detector operation (550 ms or 1000 ms). Based upon this, the generator must limit its maximum exposure time. X-rays outside the integration time slot of the DR detector are NOT allowed. This makes that with AEC, the exposure is terminated even if the target dose is not reached. Not available in Free Exposure mode using DR or Free Exposure mode using CR.

When using AEC, the exposure is terminated by the max ms or max mAs settings, even if the target dose is not reached.

Focal Spot Indicator

A focal spot indicator shows the selected focal spot of the X-ray tube: “Small” or “Large”.

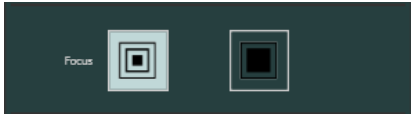


Figure 24: Focal spot indicator

You can change the focal spot by touching this indicator. It keeps kVp and constant mAs, whenever it is possible. The mA value available is set according to maximum power, instantaneous power, space charge, etc.

When a focal spot is selected, it sets the highest mA value available for the selected focal spot and the respective exposure time in order to keep constant mAs, whenever the mA value does not exceed the maximum tube power and the exposure time value does not exceed the maximum integration time of the DR detector (Max ms).



Note You can do the focal spot change whenever the present conditions of the X-ray tube allow it. The mA station set for the focal spot change is configured by the field engineer during the installation.

X-Ray Tube Load

As a way to increase the tube life cycle, the power percentage of the tube is reduced to a 80% by default. If a specific technique requires 100% of the X-ray tube power, touch the 100% button.

Depending on the status of the heat units, switching to 100% may not be possible.

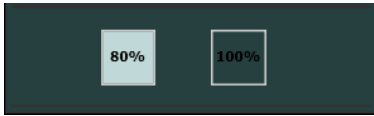


Figure 25: Tube power

Automatic Exposure Control (AEC)

Automatic Exposure Control (AEC) produces consistent density with excellent contrast regardless of the radiographic technique selected. The AEC module comprises the controls for the selection of the exposure detector fields (ion chamber), sensitivity and density compensation.

To activate AEC mode, touch any of the three AEC field buttons.

To deactivate AEC mode, touch all the selected AEC field buttons until none of them is selected.

If the message “Wrong AEC Selection” is shown on the software console before the exposure, it means that the selected kVp value, AEC density and/or sensitivity set a technique that is out of the operative range with AEC and the next exposure will be inhibited. Change any parameter (kVp value, AEC density or sensitivity) in order to obtain a technique enabled for AEC.

Field Selection

Each button indicates its related physical location of the selected field in the AEC exposure detector, and you may select or deselect it by touching it.

Any combination of fields can be selected and the color of the buttons changes (highlighted) when active.



Figure 26: Field selection

Sensitivity

Each of these buttons allows adjustment of the AEC cut-off dose (low dose, middle dose and high dose: depending on configuration at installation time). Each time a button is selected (highlighted), the others are automatically deselected.



Figure 27: Sensitivity

Density

These buttons are used to adjust the AEC cut-off dose (and patient entrance dose accordingly).



Figure 28: Density

Density can be increased and decreased in a range of -4 to +4. Each step gives a difference of 25% in dose (25% is the default setting). When disabled, the density range number appears in black.


Table 10: Density scale variation over reference dose (0)

	Density
-4	rd x 0.41
-3	rd x 0.51
-2	rd x 0.64
-1	rd x 0.80
0	Reference dose (rd)
+1	rd x 1.25
+2	rd x 1.56
+3	rd x 1.95
+4	rd x 2.44


Patient Size

The size of the patient is classified in five categories: Extra Small, Small, Medium, Large and Extra Large.

Touch the UP or DOWN arrows to select the desired patient size.

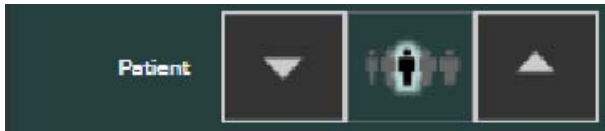**Figure 29: Patient size****AEC Backup Time**

If the exposure is aborted by the AEC back-up timer, the “CONT.” button blinks and the message “Not Enough Dose” is shown on the software console. Next exposure is inhibited until the AEC function is reset by touching the “CONT.” button. When the generator is in “Prep” mode, the AEC function can not be reset.

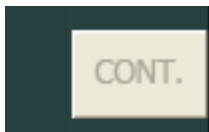**Figure 30: AEC Backup time****Rapid Termination**

The rapid termination safety device terminates the X-ray exposure when no radiation is detected in the ion chamber or when the selected parameters (short backup time/mAs) are not appropriate for an exposure with AEC.

AEC rapid termination compares the AEC ramp with a 25% of the final value at 30% of the AEC backup time. It is activated after 30% of the AEC backup time and after 10 ms of exposure, both

conditions have to be fulfilled. Whenever the AEC is active, it is recommended to select an exposure back-up time higher than 100 ms for improved operation of rapid termination.

The error code related to the rapid termination is E95.

Radiographic Working Modes

You can select following radiographic working modes according to the parameters to be controlled and the degree of automation:

- One Point Mode (1P), by selecting kVp with AEC operations.
- Two Point Mode (2P), by selecting kVp and mAs.
- Three Point Mode (3P), by selecting kV, mA and exposure time independently.

One Point Mode (1P)

By selecting one of the AEC field buttons, the one point mode is activated.

The value of kVp, mAs, mA, max ms, the setting of focal spot, density, sensitivity, patient size and the selected AEC fields can be adjusted.

The value of mAs is the max mAs. The default value for max mAs is the highest supported value.

The value for ms is not available.

Disabling all AEC fields will switch to two point mode.

After exposure all values reflect the settings actually used by the generator.

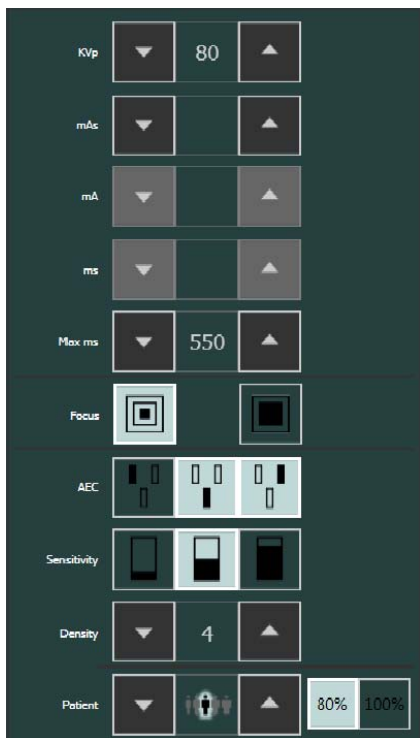


Figure 31: 1P working mode

Two Point Mode (2P)

The value of kVp, mAs, max ms, the setting of focal spot and X-ray tube load can be adjusted.

The value of mA and ms are adjusted automatically to keep the mAs value constant, within the boundaries of generator or X-ray tube limitations.

The default value for mAs is 20 mAs.

The setting of density, sensitivity and patient size is not available.

By selecting one of the AEC field buttons, the one point mode is activated.

By adjusting the value of mA or ms, the three point mode is activated.

After exposure all values reflect the settings actually used by the generator.



Figure 32: 2P working mode

Three Point Mode (3P)

The value of kVp, mA and ms can be adjusted. The other values are adjusted automatically to keep the mAs value constant.

X-Ray Modality Controls Window

- [Positioning Parameters](#) on page 81
- [Collimator Parameters](#) on page 82

Positioning Parameters

On systems with automatic positioning, the target position is automatically set, based on the selected exposure.

To modify the target position, you can:

- scroll through the preconfigured set of target positions by clicking the automatic position selection buttons. Each target position has a reference number. A reference image is displayed. The target positioning parameters are set.
- increase or decrease the positioning parameters step by step: use the UP and DOWN arrows. The values increase or decrease step by step each time the corresponding button is touched.

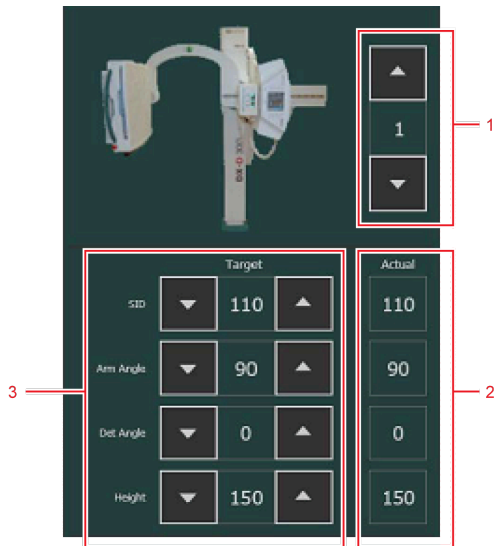


Figure 33: Positioning parameters

1. Automatic position selection buttons
2. Actual positioning parameters
3. Target positioning parameters

The selected modality position defines which positioning parameters are available.



Note Depending on the system configuration, the target positioning parameters may not be editable.

To move the X-ray system to the target position, push and hold the MOVE button on the X-ray system or on the remote control. When the X-ray system reaches the target position, the values for the actual position parameters match the values for the target position parameters and the “On target” status is displayed in the device status frame.



Figure 34: “On target” position status

To make final adjustments to the position, use the position controls on the X-ray system.

The type of X-ray system defines which automatic positions and positioning parameters are available. Typical parameters are table height and bucky position for an RAD Table, bucky height for an RAD Wall Stand and height, arm angle and detector angle for a U-Arm X-ray system, as well as general parameters like Source Image Distance (SID).

Collimator Parameters

On systems with an automatic collimator, the collimation is automatically set, based on the selected exposure.

To modify the automatic collimation, increase or decrease the collimator parameters step by step, using the UP and DOWN arrows. The values increase or decrease step by step each time the corresponding button is touched.

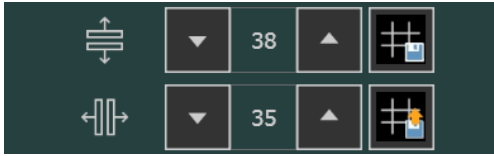


Figure 35: Collimator parameters

To use the same collimation setting on different exposures, push the store button on the first exposure and push the restore button on all later exposures that require the same collimation setting.

Problem solving

- [DR Detector is Exceeding the Maximum Working Temperature](#) on page 83
- [DR detector must be recalibrated](#) on page 83
- [DR Detector Problem](#) on page 83
- [Radiographic Parameter Limits](#) on page 84
- [Self-diagnosis Indicators](#) on page 86

DR Detector is Exceeding the Maximum Working Temperature

Details	A message is displayed on NX indicating that the DR Detector is exceeding the maximum working temperature.
Cause	Due to ambient temperature conditions and the number of acquired images, the DR Detector's internal temperature may become too high.
Brief Solution	<ol style="list-style-type: none"> 1. Power off the DR Detector. 2. Leave the DR Detector unpowered for at least one hour. 3. Stop the NX workstation. 4. Power on the DR Detector. 5. Start the NX workstation.

DR detector must be recalibrated

Details	A message is displayed indicating that the DR detector must be recalibrated.
Cause	A DR detector must be recalibrated at regular interval.
Brief Solution	<p>Follow the instructions in the user manual to calibrate the DR detector:</p> <ul style="list-style-type: none"> • DR Detector Calibration Key User Manual, document 0134

DR Detector Problem

Details	An error message is displayed on NX indicating a problem related to the DR detector.
Cause	-
Brief Solution	<ol style="list-style-type: none"> 1. Power off the DR detector. 2. Stop the NX workstation. 3. Power on the DR detector. 4. Start the NX workstation.

Radiographic Parameter Limits

If the value of a radiographic parameter cannot be increased or decreased, one of these limits may be applied:

- **Radiographic Parameters Limit.** A maximum or minimum radiographic parameter limit is reached. An information message is displayed.
- **Generator Power Limit.** The generator power limit (kVp x mA) is reached. An information message is displayed. If the generator power limit is reached by increasing the mA up to a maximum possible value, you could increase the kVp up to its maximum value. The mA value will then automatically decrease as long as the mAs value is kept constant.
- **Space Charge.** The space charge limit in the selected X-ray tube is reached by changing the kVp or mA values. An information message is displayed.
- **Maximum Energy (60kJ).** Only in AEC mode, the maximum energy (60kJ) is exceeded. An information message is displayed.
- **Instantaneous Power.** The instantaneous power limit of the X-ray tube (ratings limit or the X-ray tube is momentarily overheated) is reached by selecting some technique. An information message is displayed.

The following table indicates the different information messages that may appear on the console when increasing or decreasing one of the exposure parameters.

Table 11: Radiographic parameter limits


Information message	Description
Min kVp	Minimum kVp (generator limit)
Max kVp	Maximum kVp (generator limit)
Max kVp Tube	Maximum kVp (limited by the X-ray tube load curves or limited during the generator configuration)
Min mA	Minimum mA (generator limit configured for each focal spot)
Max mA	Maximum mA (generator limit configured for each focal spot)
Min mAs	Minimum mAs (generator limit)
Max mAs	Maximum mAs = 500 mAs (regulatory limit for AEC)
Min ms	Minimum exposure time (generator limit)
Max ms	Maximum exposure time (generator limit)
Min ms & Min mA	Minimum exposure time and minimum mA (generator limit)
Max ms & Max mA	Maximum exposure time and maximum mA (generator limit)
Max Power	Maximum power (generator limit)
Space Charge	Filament emission limit for a combination of kVp and mA in the selected focal spot.

Information message	Description
Max Energy (60kJ)	The maximum energy cannot exceed 60 kJ (kVp x mAs = 60 kJ) (regulatory limit for AEC).
Instant Power	Instantaneous power limit of the X-ray tube depending on the anode temperature, the selected exposure time and the selected focal spot.

Self-diagnosis Indicators

Self-diagnosis indicators identify a malfunction in the system alerting the operator that an error is inhibiting exposure. During normal operation of the system, these indicators are directly shown on the lower area of the console or as an error code close to the “CONT.” button.

Table 12: Self-diagnostic indicators

Indicator	Description
DOOR OPEN	Indicates that the X-ray room door is open when the X-ray equipment is in use.
GENERATOR OVERLOAD	Indicates that the exposure was interrupted because arcing or malfunctioning occurred during exposure on the HV circuitry (X-ray tube, HV transformer and/or HV cables) or a failure of IGBT module (overheated or defective IGBTs) was detected. It can also appear when making a lengthy or high powered exposure with the X-ray tube cool (when the X-ray tube has not been sufficiently warmed-up).
TUBE OVERLOAD	Indicates that either the technique selected is beyond the X-ray tube ratings or present conditions of the X-ray tube inhibit the exposure (anode overheated). Parameters for next exposure may be temporarily limited by the generator (change the exposure values or wait for the X-ray tube to cool).
ROTOR ERROR	Indicates that the X-ray tube anode is not rotating while “Prep” is active. Exposures are inhibited.
OVERHEAT	Indicates that the X-ray tube thermostat/pressurestat is open due to overheating of the X-ray tube housing (housing is too hot, wait for the housing to cool) or that there is a thermostat/pressurestat malfunction (housing is cool). Heat units may rise to any value.
TECHNIQUE ERROR	 If “TECHNIQUE ERROR” is displayed during exposure it means the exposure has been interrupted by the “Security Timer” because of a system failure. You should call field service. This error can also be shown if a failure on the automatic collimator has been detected (blades are full open or in movement during exposure, etc.). In this case the indicator lights up continuously.
PANEL OFF	Indicates that the detector panel has been automatically powered off due to: <ul style="list-style-type: none"> • a power off in the X-ray generator control. • an overheat (above 47 °C) in the detector panel. Exposure with the DR detector is inhibited except for the free exposure mode.
(G)E63	Excessive current in auxiliary winding during acceleration to 10000 RPM. Retry the exposure.

Technical Data

- [DX-D 300 technical data](#) on page 88
- [Portable DR detector technical data \(mounted fixed in the bucky\)](#) on page 89
- [Fixed DR detector technical data](#) on page 91
- [Fixed DR detector technical data](#) on page 93
- [U-Arm technical data](#) on page 95
- [Portable DR detector technical data](#) on page 96
- [Dose Area Product Meter \(IBA DAP\) Technical Data](#) on page 97

DX-D 300 technical data

The technical data are provided in this chapter or in the user manual of the component.

Table 13: Electrical connection

Rated power supply	230 / 240 V# Single Phase 50/60 Hz
Minimum input power required	2.5 kVA

Table 14: Environmental conditions for the U-arm

Environmental conditions (during storage and transport)	
Temperature (ambient)	between -20 and 70 degrees Celsius
Humidity (non condensing)	between 10 and 90 % relative humidity
Atmospheric pressure	between 50 and 106 kPa
Environmental conditions (during normal operation)	
Temperature (ambient)	between 10 and 35 degrees Celsius
Humidity (non condensing)	between 30 and 75 % relative humidity
Atmospheric pressure	between 70 and 106 kPa

For overall system environmental conditions, the environmental conditions of the DR detector or image plate should be taken into account. Refer to the related user manual for environmental conditions for the DR detector or image plate. When using the DR detector or image plate inside the bucky, take into account that the temperature inside the bucky can be up to 5°C higher than the temperature in the X-ray room.

Portable DR detector technical data (mounted fixed in the bucky)

Manufacturer	
Manufacturer DR detector	Vieworks Co., Ltd. (Gwanyang-dong), 41-3, Burim-ro 170beon-gil, Dongan-gu, Anyang-si, Gyeonggi-do, Korea
Distributor DR detector	Agfa NV Septestraat 27, B-2640 Mortsel - Belgium
Original manufacturer model name	
XD 17	FXRD-4343VAW
XD*17	FXRD-4343VAW PLUS
Electrical connection	
Power adapter with USB Type-C cable	DC 18 V, max. 2.78 A
Power consumption	max. 24 W
Network connection	
Wireless connection	IEEE 802.11n/ac (2.4 GHz/5 GHz)
Environmental conditions (during normal operation)	
Room temperature	between 0 °C and +40 °C
Humidity (non condensing)	between 5% and 90% RH (non-condensing)
Atmospheric pressure	between 700 hPa and 1060 hPa
Environmental conditions (during storage and transport)	
Temperature (ambient)	between -15 °C and +55 °C
Humidity (non condensing)	between 5% and 90% (non-condensing)
Atmospheric pressure	between 500 and 1060 hPa
Image acquisition	
Image acquisition time (minimum cycle time)	4 s
Conversion screen	CsI
Pixel size	140 µm
Active pixel matrix	3072 x 3072
Effective pixel matrix	3048 x 3048
Detector type	amorphous silicium
Active area size	430 mm x 430 mm

Effective area size	426.7.0 mm x 426.7 mm
---------------------	-----------------------

Fixed DR detector technical data

Manufacturer	
Manufacturer DR detector	Varex Imaging Corporation, 1678 So. Pioneer Rd, Salt Lake City, UT 84104, USA
Supported models	
4343R (part number 7965)	CsI conversion screen
4343R (part number 7964)	GOS conversion screen
Electrical connection	
Operating voltage	90-240 V (AC)
Mains fuse protection	6A
Mains frequency	47-63 Hz
Power consumption	
Maximum power consumption	45 W
Warming-up time	
	1 hour
Throughput	
Maximum number of image acquisitions	150 acquisitions per hour
Pixel matrix	
Pixel size	139 μm (H,V)
Pixel matrix	3072(H) x 3072(V)
Active pixel matrix	3056(H) x 3056(V)
Fill factor	100 %
Detector type	Amorphous Silicon
Active area size	42,7 cm (H) x 42,7 cm (V)
Reliability	

Estimated product life (if regularly serviced and maintained according to Agfa instructions)
--

100 000 RAD

Fixed DR detector technical data

Manufacturer	
Manufacturer DR detector	THALES AVS FRANCE SAS 460 Rue du Pommarin – BP122 38430 MOIRANS France
Supported models	
Pixium RAD 4343 C-E	CsI conversion screen
Pixium RAD 4343 G-E	GOS conversion screen
Electrical connection	
Operating voltage	+24V 3.5A DC
Warming-up time	
	5 minutes
Throughput	
Maximum number of image acquisitions	150 acquisitions per hour
Reliability	
Estimated product life (if regularly serviced and maintained according to Agfa instructions)	100 Gy

Pixel matrix	
Pixel size	148 µm (H,V)
Pixel matrix	2880(H) x 2880(V)
Active pixel matrix	2860(H) x 2874(V)
Fill factor	100 %
Detector type	Amorphous Silicon
Active area size	426.24 mm (H) x 426.24 mm (V)

- [Fixed DR detector environmental conditions](#) on page 94

Fixed DR detector environmental conditions

Pixium RAD 4343 C-E

Environmental conditions (during normal operation)	
Temperature (ambient)	between 15° and 35° Celsius
Humidity Atmospheric pressure Maximum altitude	Refer to environmental conditions of the X-ray system

	minimum	maximum
Distance to calibration temperature	-10 °C	+10 °C
Distance to calibration pressure	-100 mbar	+100 mbar

Pixium RAD 4343 G-E

Environmental conditions (during normal operation)	
Temperature (ambient)	between 15° and 40° Celsius
Humidity Atmospheric pressure Maximum altitude	Refer to environmental conditions of the X-ray system

	minimum	maximum
Distance to calibration temperature	-10 °C	+10 °C
Distance to calibration pressure	-100 mbar	+100 mbar

U-Arm technical data

Manufacturer	
Manufacturer U-Arm	Sedecal S.A. Polígono Ind. Rio de Janeiro 9-13 28110 Algete - Madrid Spain

Refer to the DX-D 300 U-arm User Manual (document 0174) for technical data of the U-arm.

Portable DR detector technical data

Refer to the DR Detector User Manual.

Dose Area Product Meter (IBA DAP) Technical Data

Manufacturer	IBA Dosimetry GmbH Bahnhofstrasse 5 DE-90592 Schwarzenbruck
Supported Type	120-131 HS/RS485
Dose area product range	(0.1...99999999.99) cGy x cm ²
DAP resolution	0.01 cGy x cm ²
Active area	14.0 cm x 14.0 cm
Dimensions	17.9 cm x 16.6 cm x 1.7 cm (WxDxH)
Weight	approx. 220 g
Equivalent filtration of the ionization chamber at 70 kV	0.31 mm Al

Correction factors for using the DAP meter on high altitude	
Environmental conditions	Correction factor
75 kPa (ca. 2500 m) 0° Celcius	1.26
75 kPa (ca. 2500 m) 20° Celcius	1.35
70 kPa (ca. 3000 m) 0° Celcius	1.35
70 kPa (ca. 3000 m) 20° Celcius	1.45

Remarks for HF-emission and immunity

This equipment generates, uses, and can radiate radio frequency energy.



Caution: The equipment may cause radio frequency interference to other medical or non medical devices and to radio communications.

To provide reasonable protection against such interference, this equipment complies with emissions limits for a Group 1-Class A Medical Devices Directive as stated in IEC 60601-1-2. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment is found to cause interference (which may be determined by turning the equipment on and off), the operator (or qualified service personnel) should attempt to correct the problem by one or more of the following measures:

- reorient or relocate the affected device,
- increase the separation between the equipment and the affected device,
- power the equipment from a source different from that of the affected device,
- consult the service engineers for further suggestions.

To comply with the regulations applicable to an electromagnetic interference for a Group 1 -- Class A Medical Device, all interconnect cables to peripheral devices must be shielded and properly grounded. Use of cables not properly shielded and grounded may result in the equipment causing radio frequency interference in violation of the European Union Medical Device Directive and of Federal Communications Commission regulations.



Caution: Before using this equipment make sure that all requirements about EMC included in this manual are accomplished.



Caution: Should any interference (EMC) be detected with other equipment, please position other equipment away from this one.



Caution: It is customer responsibility to assure that this equipment and vicinity equipment complies the value of radio frequency interferences shown in General Regulation for safety according to IEC 60601-1-2 tables as described in this section.



Caution: The manufacturer is not responsible for any interference caused by using other than recommended interconnect cables, accessories and transducers or by unauthorized changes or modifications to this equipment.

The DX-D 300 system is intended for use in the electromagnetic environment specified below. The customer or the user of this DX-D 300 system should assure that it is used in such an environment.

Table 15: Guidance and manufacturer's declaration on electromagnetic emissions (IEC 60601-1-2)

Emission test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The device is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	
Note: In accordance with Standard IEC 61601-1-2, the emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 Class A. If it is used in a residential environment (for which CISPR 11 Class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orientating the equipment.		

The DX-D 300 system is intended for use in the electromagnetic environment specified below. The customer or the user of this DX-D 300 system should assure that it is used in such an environment.

Table 16: Guidance and manufacturer's declaration on electromagnetic immunity

Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should consist of wood, concrete or ceramic tiles. The relative humidity must be at least 30%, if the floor is made of synthetic material.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines (100 kHz repetition frequency)	± 2 kV for power supply lines ± 1 kV for input/output lines (100 kHz repetition frequency)	The quality of the voltage supplied should correspond to a typical commercial or clinical environment.
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV line(s) to line(s) ± 0.5 kV, ± 1 kV, ± 2 kV line(s) to earth	± 0.5 kV, ± 1 kV line(s) to line(s) ± 0.5 kV, ± 1 kV, ± 2 kV line(s) to earth	The quality of the voltage supplied should correspond to that of a typical commercial or clinical environment.
Voltage dips, short interruptions and voltage variations on power supply input lines. IEC 61000-4-11	0% U_T for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U_T for 1 cycle at 0° 70% U_T for 25/30 cycles at 0° 0% 250/300 cycles	0% U_T for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U_T for 1 cycle at 0° 70% U_T for 25/30 cycles at 0° 0% 250/300 cycles	The quality of the voltage supply should correspond to that of a typical commercial or clinical environment. If the user wants the device to work continuously, even when the energy supply is interrupted, it is recommended to use an energy supply free of interruptions or a battery.
Magnetic field at the supply frequency (50/60 Hz) IEC 61000-4-8	30 A/m	30 A/m	Magnetic field at the network frequency should correspond to the typical values as they are in a commercial and clinical environment.
<ul style="list-style-type: none"> Note: U_T is the alternating current in the network before the application of the test level. 			

Table 17: Guidance and manufacturer's declaration on electromagnetic immunity

Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic environment - guidance
---------------	--------------------------	------------------	--

Radiated RF EM fields IEC 61000-4-3	3 Vrms from 80 MHz to 2.7 GHz (80% AM at 1 kHz)	3 Vrms from 80 MHz to 2.7 GHz (80% AM at 1 kHz)	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm to any part of the equipment, including cables specified by manufacturer. Otherwise, degradation of the performance of this equipment could result.
Proximity fields from RF wireless communications equipment IEC 61000-4-3	Refer to the section "Immunity to RF wireless communication equipment"	Refer to the section "Immunity to RF wireless communication equipment"	
Conducted disturbances induced by RF fields IEC 61000-4-6	3 Vrms from 150 kHz to 80 Mhz 6 Vrms in ISM bands from 150 kHz to 80 MHz (80% AM at 1 kHz)	3 Vrms from 150 kHz to 80 Mhz 6 Vrms in ISM bands from 150 kHz to 80 MHz (80% AM at 1 kHz)	
<p>Note: The ISM (industrial, scientific and medical) bands between 0.15 MHz and 80 MHz are 6.765 MHz to 6.795 MHz; 13.553 MHz to 13.567 MHz; 26.957 MHz to 27.283 MHz; and 40.66 MHz to 40.70 MHz. The amateur radio bands between 0.15 MHz and 80 MHz are 1.8 MHz to 2.0 MHz; 3.5 MHz to 4.0 MHz; 5.3 MHz to 5.4 MHz; 7 MHz to 7.3 MHz; 10.1 MHz to 10.15 MHz; 14 MHz to 14.2 MHz; 18.07 MHz to 18.17 MHz; 21.0 MHz to 21.4 MHz; 24.89 MHz to 24.99 MHz; 28.0 MHz to 29.7 MHz; and 50.0 MHz to 54.0 MHz.</p>			

- [Immunity to RF wireless communication equipment](#) on page 100

Immunity to RF wireless communication equipment

ISM Band (MHz)	Modulation	Distance (m)	Immunity test level (V/m)
300-390	Pulse modulation 18 Hz	0.3	27
430-470	FM ± 5 kHz deviation 1 kHz sine	0.3	28
704-787	Pulse modulation 217 Hz	0.3	9
800-960	Pulse modulation 18 Hz	0.3	28

ISM Band (MHz)	Modulation	Distance (m)	Immunity test level (V/m)
1700-1990	Pulse modulation 217 Hz	0.3	28
2400-2570	Pulse modulation 217 Hz	0.3	28
5100-5800	Pulse modulation 217 Hz	0.3	9